# REAL TIME PCR DETECTION KIT

## *Mayaro Virus*

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VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 6 x 8-well strips, low profile	<b>VS-MYV106L</b>
VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 6 x 8-well strips, high profile	<b>VS-MYV106H</b>
VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 12 x 8-well strips, low profile	<b>VS-MYV112L</b>
VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 12 x 8-well strips, high profile	<b>VS-MYV112H</b>
VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 96-well plate, low profile	<b>VS-MYV113L</b>
VIASURE <i>Mayaro Virus</i> Real Time PCR Detection Kit 96-well plate, high profile	<b>VS-MYV113H</b>

VIASURE

## ENGLISH

---

### 1. Intended of use

VIASURE *Mayaro Virus* Real Time PCR Detection Kit is designed for specific identification of Mayaro Virus in clinical samples from patients with signs and symptoms of Mayaro Virus infection. This test is intended for use as an aid in the diagnosis of the Mayaro Virus in combination with clinical and epidemiological risk factors. RNA is extracted from specimens, amplified using RT-PCR and detected using fluorescent reporter dye probes specific for Mayaro Virus.

### 2. Summary and Explanation

Mayaro virus (MAYV) is an enveloped, single-stranded positive RNA virus which belongs to the genus *Alphavirus*, family *Togaviridae*. Its transmission cycle is thought to occur mainly through mosquito vectors, especially those of genus *Haemagogus*, but *Aedes* spp. mosquitoes may also be competent vectors. These vectors transmit the virus to primates and birds and sometimes to humans.

The genome has 11.5 kb and comprises two coding regions; the first encodes a nonstructural polyprotein (nsP) which is proteolytically cleaved into four non-structural (NSP1-4) peptides. The second region encodes a polyprotein cleaved by autocatalysis and cellular proteases into six structural peptides: capsid, envelope 3 (E3), E2, 6K, transframe (TF) and E1. Separating these two coding regions it is found an internal ribosome entry site.

Phylogenetic studies using a fragment of the structural polyprotein open reading frame suggest the classification of MAYV in 2 different genotypes: D (includes isolates from all countries) and L (contains strains detected only in Brazil).

The disease caused by Mayaro lasts for 3-5 days and includes fever, headache, rash, myalgia, arthralgia and, occasionally, arthritis, which can be very debilitating and can persist for months. This symptomatology is quite similar to other arthropod-borne virus, being sometimes clinically indistinguishable from dengue, chikungunya fever, malaria, rabies, measles or other arboviral diseases, so it is required to use a reliable and specific test to avoid this misdiagnosis.

This Real Time PCR Detection Kit enable a reliable diagnosis of Mayaro virus by the amplification of the *nsp1* gene, which has been shown to be the best target for detection due to its highly conserved sequences.

### 3. Principle of the procedure

VIASURE *Mayaro Virus* Real Time PCR Detection Kit is designed for the diagnosis of the Mayaro Virus in clinical samples. The detection is done in one step real time RT format where the reverse transcription and the subsequent amplification of specific target sequence occur in the same reaction well. The isolated RNA target is transcribed generating complementary DNA by reverse transcriptase which is followed by the amplification of the *nsp1* gene using specific primers and a fluorescent-labelled probe.

VIASURE *Mayaro Virus* Real Time PCR Detection Kit is based on 5' exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal which is proportional to the quantity of target template. This fluorescence could be measured on Real Time PCR platforms.

VIASURE *Mayaro Virus* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPS, buffer, polymerase, retrotranscriptase) in a stabilized format, as well as an internal control to monitor PCR inhibition. *Mayaro Virus* RNA targets are amplified and detected in FAM channel and the internal control (IC) in HEX, VIC or JOE channel (depending on the equipment used).

#### 4. Reagents provided

VIASURE *Mayaro Virus* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table 1 and Table 2:

Reference	Reagent/Material	Description	Colour	Amount
VS-MYV1SL/ VS-MYV1SH	<i>Mayaro Virus</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	6/12 X 8-well strip
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-MYV1C	<i>Mayaro Virus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	6/12 X 8-cap strip

Table 1. Reagents and materials provided in VIASURE *Mayaro Virus* Real Time PCR Detection Kit with Ref. VS-MYV106L, VS-MYV106H, VS-MYV112L and VS-MYV112H.

Reference	Reagent/Material	Description	Colour	Amount
VS-MYV1PL/ VS-MYV1PH	<i>Mayaro Virus</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-MYV1C	<i>Mayaro Virus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 X 8-cap strip

Table 2. Reagents and materials provided in VIASURE *Mayaro Virus* Real Time PCR Detection Kit with Ref VS-MYV113L and VS-MYV113H.

## 5. Reagents and equipment to be supplied by the user

---

The following list includes materials that are required for using but not included in the VIASURE *Mayaro Virus* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler) (to check compatibility see Annex I).
- RNA extraction kit.
- Centrifuge for 1.5 mL tubes.
- Vortex.
- Micropipettes (0.5-20  $\mu$ L, 20-200  $\mu$ L).
- Filter tips.
- Powder-free disposal gloves.

## 6. Transport and storage conditions

---

- The kits can be shipped and stored at 2-40°C until expiration date stated in the label.
- Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.
- Keep components away from sunlight.

## 7. Precautions for users

---

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- Design a unidirectional workflow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area. Once you finish the test wash your hands.
- Specimens must be treated as potentially infectious as well as all reagents and materials that have been exposed to the samples and must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

## 8. Test procedure

### 8.1. RNA EXTRACTION

Perform the sample preparation according to the recommendations appearing in the instructions for use of extraction kit used.

For RNA extraction from blood and serum samples you can use your manually or automatic routine optimized system. Also, you can use any commercially available RNA extraction kit and follow the manufacturer's instructions for use.

### 8.2. LYOPHILIZED POSITIVE CONTROL

*Mayaro Virus* Positive Control contains high copies template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Mayaro Virus* Positive Control (red vial) adding 100 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.

### 8.3. PCR PROTOCOL

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run. Peel off protective aluminium seal from plates or strips.

#### 1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

#### 2) Adding samples and controls.

Add 5 µL of RNA sample, reconstituted *Mayaro Virus* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the wells with the caps provided. Centrifuge briefly.

Load the plate or the strips in the thermocycler.

#### 3) Set up your thermocycler.

Program your thermocycler following the conditions below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 seg	95°C
	Annealing/Extension (Data collection*)	50 seg	60°C

Table 3. PCR protocol

Fluorogenic data should be collected during the extension step (\*) through the FAM (*Mayaro Virus*) and HEX, JOE or VIC channels (Internal Control (IC)). In Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System and Stratagene Mx3005P™ Real Time PCR System check that passive reference option ROX is none.

## 9. Result interpretation

The use of positive and negative controls in each run, validate the reaction by checking the absence of signal in negative control well and the presence of signal for Mayaro Virus positive control well. Check Internal Control signal to verify the correct functioning of the amplification mix. The analysis of the samples is done by the software itself of the used real time PCR equipment according to manufacturer’s instructions.

Using the following table read and analyse the results:

Mayaro Virus	Internal control	Negative control	Positive control	Interpretation
+	+/-	-	+	Mayaro Virus Positive
-	+	-	+	Mayaro Virus Negative
+	+	+	+	Experiment fail
-	-	-	-	Experiment fail


Table 4. Sample interpretation

- +: Amplification curve
- : No amplification curve

A sample is considered positive if the Ct value obtained is less than 40 and the internal control shows or not an amplification signal. Sometimes, the detection of internal control is not necessary because a high copy number of target can cause preferential amplification of target-specific nucleic acids.

A sample is considered negative, if the sample shows no amplification signal in the detection system but the internal control is positive. An inhibition of the PCR reaction can be excluded by the amplification of internal control.

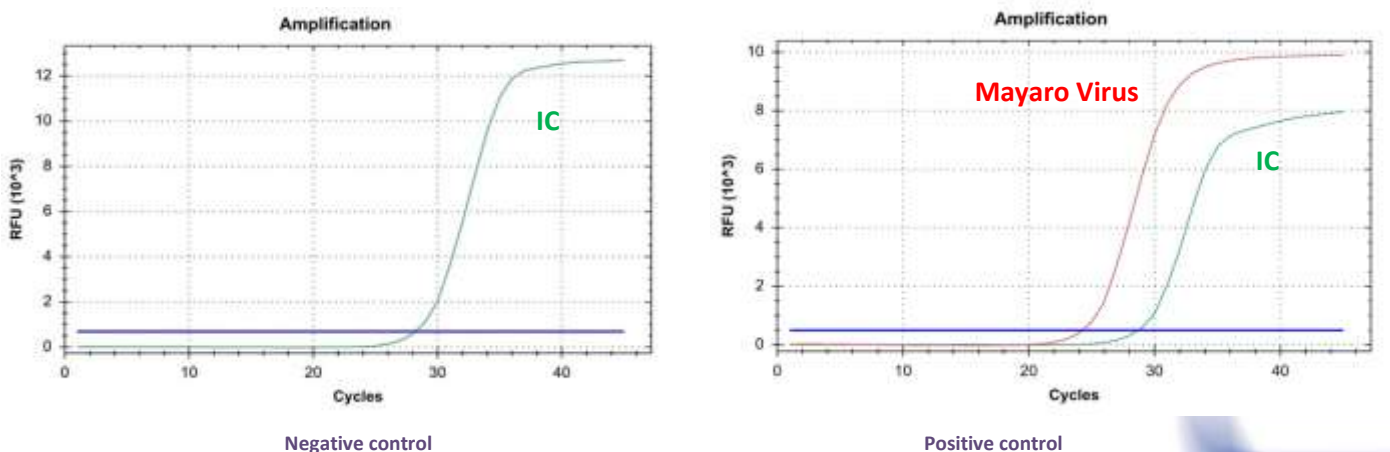Figure 1. Correct run of negative and positive control run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

The result is considered invalid if there is signal of amplification in negative control or absence of signal in the positive well. We recommend to repeat the assay again.

In case of absence of internal control signal in sample wells we recommend to repeat the assay diluting the sample 1:10 or to repeat the extraction to check for possible problems of inhibition.

## 10. Limitations of the test

- The result of the test should be evaluated by a health care professional and evaluated in the context of medical history, clinical symptoms and other diagnostic tests.
- The quality of the test depends on the quality of the sample; proper RNA from clinical samples must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Mayaro Virus, either samples containing high concentrations of target RNA or contamination due to PCR products from previous reactions.

## 11. Quality control


VIASURE *Mayaro Virus* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) in each well confirms the correct performance of the technique.

## 12. Performance characteristics

### 12.1. ANALYTICAL SENSITIVITY

VIASURE *Mayaro Virus* Real Time PCR Detection Kit has a detection limit of  $\geq 10$  RNA copies per reaction (Figure 2).

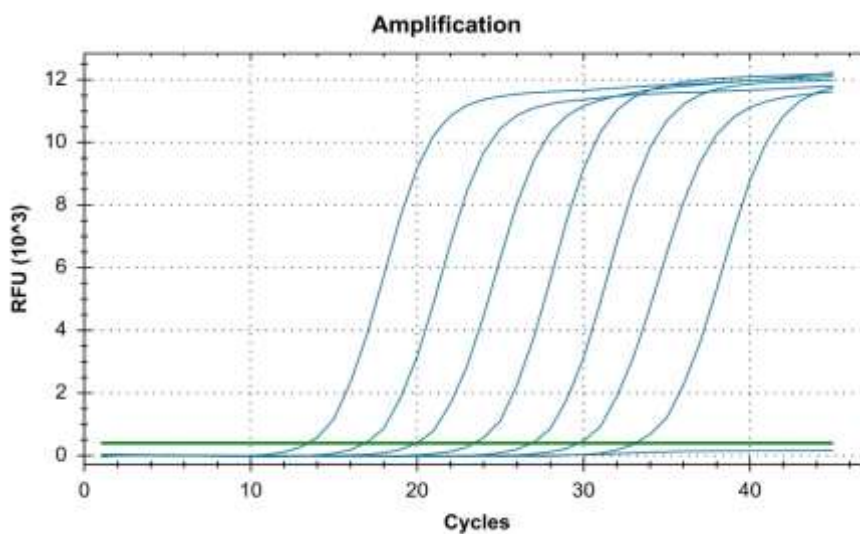Figure 2. Dilution series of Mayaro Virus ( $10^7$ - $10^1$  copies/rxn) template run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

**12.2. ANALYTICAL SPECIFICITY**

The specificity of the Mayaro Virus assay was confirmed by testing a panel consisting of 11 microorganisms representing the most common arboviruses. No cross-reactivity was detected against any of the following microorganisms tested:

Cross-reactivity testing			
Chikungunya virus strain S27 Petersfield	-	Yellow Fever virus strain 17D	-
Zika virus strain MR 766	-	Dengue 1 virus strain Hawaii	-
St Louis Encephalitis virus strain 17D	-	Dengue 2 virus strain New Guinea C	-
West Nile virus strain H160/99	-	Dengue 3 virus strain H87	-
West Nile virus Heja	-	Dengue 4 virus strain H241	-
West Nile virus Ug37	-		

Table 5. Reference pathogenic microorganisms used in this study.

**12.3. ANALYTICAL REACTIVITY**

The reactivity of VIASURE *Mayaro Virus* Real Time PCR Detection Kit was evaluated against Mayaro virus showing positive results.

**ANNEX 1:**

**COMPATIBILITY OF THE MOST COMMON REAL TIME PCR EQUIPMENT**

Low profile strips can be used in all PCR thermocyclers equipped with low profile block, like systems listed in table A.1. High profile strips can be used in all PCR thermocyclers equipped with high or regular profile block, like systems listed in table A.2. If you do not find your thermocycler in the list below, please contact with your supplier.

Table A.1 LOW PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7500 Fast Real-Time PCR System
Applied Biosystems	7500 Fast Dx Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well Fast
Applied Biosystems	QuantStudio™ 6 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 7 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 3 Real-Time PCR System
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	StepOne Plus™ Real-Time PCR System
Applied Biosystems	StepOne™ Real-Time PCR System
Applied Biosystems	ViiA™ 7 Fast Real-Time PCR System
Bio-Rad	CFX96 Touch™ Real-Time PCR Detection System
Bio-Rad	Mini Opticon™ Real-Time PCR Detection System
Roche	LightCycler® 480 Real-Time PCR System
Roche	LightCycler® 96 Real-Time PCR System
Agilent Technologies	AriaMx Real-Time PCR System
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cyclers
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®*

Table A.2 HIGH PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7300 Real-Time PCR System
Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	7900 HT Real-Time PCR System
Applied Biosystems	ABI PRISM 7000
Applied Biosystems	ABI PRISM 7700
Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	QuantStudio™ 7 Flex 96-well
Applied Biosystems	QuantStudio™ 3 Real-Time PCR System
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	ViiA™ 7 Real-Time PCR System
Bio-Rad	CFX96 Touch™ Deep Well Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Bio-Rad	iCycler iQ™5 Real-Time PCR Detection System
Bio-Rad	MyiQ™ Real-Time PCR Detection System
Bio-Rad	MyiQ™2 Real-Time PCR Detection System
Eppendorf	Mastercycler™ep realplex
Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
Analytik Jena Biometra	TOptical
Analytik Jena Biometra	qTOWER 2.0
Abbott	Abbott m2000 RealTime System
BIONEER	Exicycler™ 96
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cyclers
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®*

Table A1/A2. Compatible low and high profile Real Time PCR systems.

\* The product should be reconstituted following the appropriate procedure (see Test Procedure) and transferred into specific Rotor-Gene® Q or SmartCycler® tubes.

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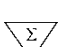
### 13. Bibliography/Bibliografía

1. M.T. Oliveira Mota et al. Complete Genome Sequence of Mayaro Virus Imported from the Amazon Basin to Sao Paulo State, Brazil. *Genome Announcements* 2015; 3(6): e01341-15.
2. J. Lednicky et al. Mayaro Virus in Child with Acute Febrile Illness, Haiti, 2015. *Emerging infectious diseases* 2016; 22(11): 2000-2002.
3. M. Llagone-Barets et al. A case of Mayaro virus infection imported from French Guiana. *Journal of Clinical Virology* 2016; 77: 66-68.
4. M. L. Garcia de Figueiredo et al. Emerging alphaviruses in the Americas: Chikungunya and Mayaro. *Revista de Sociedade Brasileira de Medicina Tropical* 2014; 47(6): 677-683.
5. B. Friedrich-Jänicke et al. Genome analysis of Mayaro virus imported to Germany from French Guiana. *Emerging infectious diseases* 2014; 20(7): 1255-1257.

### 14. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico *in vitro*

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VIASURE *Mayaro Virus* Real Time PCR Detection Kit has been validated on the following equipments: Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System, Agilent Technologies AriaMx Real-Time PCR System, DNA-Technology DTprime Real-time Detection Thermal Cycler, Rotor-Gene® Q (Qiagen) and SmartCycler® (Cepheid). When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder to reduce the risk of crushed tube (Ref. PN 4388506).

---

VIASURE *Mayaro Virus* Real Time PCR Detection Kit ha sido validado en los siguientes equipos: Applied Biosystems 7500 Fast Real-Time PCR System, Applied Biosystems StepOne™ Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System, Agilent Technologies AriaMx Real-Time PCR System, DNA-Technology DTprime Real-time Detection Thermal Cycler, Rotor-Gene® Q (Qiagen) y SmartCycler® (Cepheid). Cuando se utiliza el equipo Applied Biosystems 7500 Fast con tiras, se recomienda colocar el soporte adecuado para reducir el riesgo de aplastar el tubo (Ref. PN 4388506).

- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.
- ABI®, QuantStudio™, StepOnePlus™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registered trademark of Roche.
- Mx3000P™, Mx3005™ and AriaMx are registered trademarks of Agilent Technologies.
- Mastercycler™ is a registered trademark of Eppendorf.
- Rotor-Gene®Q is a registered trademark of Qiagen.
- SmartCycler® is a registered trademark of Cepheid.

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