REAL TIME PCR DETECTION KIT

Norovirus GI + GII

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ENGLISH

1. Intended of use

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit is designed for specific detection and differentiation of Norovirus Genogroup I (GI) and Norovirus Genogroup II (GII) in human stool samples from patients with signs and symptoms of gastrointestinal infection. This test is intended for use as an aid in the diagnosis of Norovirus GI and/or Norovirus GII in combination with clinical and epidemiological risk factors. RNA is extracted from stool specimens, amplified using RT-PCR and detected using fluorescent reporter dye probes specific for Norovirus GI and/or Norovirus GII.

2. Summary and Explanation

Norovirus (NoV) belongs to the *Caliciviridae* family and is considered to be the major cause of acute nonbacterial gastroenteritis in all age groups, worldwide. NoVs are classified into six distinct genogroups (GI-GVI) based on phylogenetic clustering of the complete VP1 amino acid sequence. In particular, genogroups GI, GII, and GIV infects humans. They are further divided into genotypes defined on the basis of either the RNA polymerase and/or the capsid protein VP1 sequences. To date, nine capsid genotypes have been recognized in GI and twenty two in GII, and three genotypes of GII (GII.11, GII.18, and GII.19) have been uniquely detected in swine. Among them, GII.4 is responsible for the majority of the norovirus outbreaks worldwide.

After an incubation period of 12 to 48 h, norovirus illness is characterized by projectile vomiting, nonbloody diarrhea, nausea, abdominal cramps, and low-grade fever. Noroviruses can infect humans via multiple routes, including the oral route, transmitted through contact with fecal matter or aerosolized vomitus from infected people, as well as contaminated surfaces, food, or water.

The Norovirus genome consists of a single-stranded, positive-sense RNA containing three open reading frames (ORFs). The ORF1 encodes a polypeptide of six non-structural proteins, including RNA dependent RNA polymerase (RdRp), the ORF2 encodes the major capsid protein, VP1, whereas the ORF3 encodes the minor capsid protein, VP2. Most real-time RT-PCR assays for GI and GII Norovirus target the RNA polymerase-capsid junction (ORF1-ORF2 junction), which is the most highly conserved region of the Norovirus genome.

3. Principle of the procedure

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit is designed for the diagnosis of Norovirus GI and/or Norovirus GII in human stool samples. The detection of Norovirus GI and Norovirus GII is done in one step real time RT format where the reverse transcription and the subsequent amplification of specific target sequence occur in the same reaction well. The isolated RNA targets are transcribed generating complementary DNA by reverse transcriptase which is followed by the amplification of a conserved region of the ORF1-ORF2 junction for Norovirus GI and GII using specific primers and a fluorescent-labelled probe.

REAL TIME PCR DETECTION KIT

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit is based on 5' exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal which is proportional to the quantity of the target template. This fluorescence could be measured on Real Time PCR platforms.

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPS, buffer, polymerase, retrotranscriptase) in a stabilized format, as well as an internal control to monitor PCR inhibition. Norovirus GII RNA targets are amplified and detected in FAM channel, Norovirus GI RNA targets are amplified and detected in Cy5 channel and the internal control (IC) in HEX, VIC or JOE channel (depending on the equipment used select the proper detection channel, see Annex 2).

4. Reagents provided

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table 1 and Table 2:

Reference	Reagent/Material	Description	Colour	Amount
VS-NOR1SL/ VS-NOR1SH	<i>Norovirus GI + GII</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	6/12 X 8-well strip
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-NOR1C	<i>Norovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	6/12 X 8-cap strip

Table 1. Reagents and materials provided in VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit with Ref. VS-NOR106L, VS-NOR106H, VS-NOR112L and VS-NOR112H.

Reference	Reagent/Material	Description	Colour	Amount
VS-NOR1PL/ VS-NOR1PH	<i>Norovirus GI + GII</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-NOR1C	<i>Norovirus</i> Positive Control	Non-infectious synthetic lyophilized cDNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNAse/DNAse free	Water RNAse/DNAse free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 X 8-cap strip

Table 2. Reagents and materials provided in VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit with Ref. VS-NOR113L and VS-NOR113H.

5. Reagents and equipment to be supplied by the user

The following list includes materials that are required for using but not included in the VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler) (to check compatibility see Annex I).
- RNA extraction kit.
- Centrifuge for 1.5 mL tubes.
- Vortex.
- Micropipettes (0.5-20 µL, 20-200 µL).
- Filter tips.
- Powder-free disposal gloves.

6. Transport and storage conditions

- The kits can be shipped and stored at 2-40°C until expiration date stated in the label.
- Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.
- Keep components away from sunlight.

7. Precautions for users

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- Design a unidirectional workflow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area. Once you finish the test wash your hands.
- Specimens must be treated as potentially infectious as well as all reagents and materials that have been exposed to the samples and must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

8. Test procedure

8.1. SAMPLE PREPARATION

Stool samples should be collected in clean containers and processed as soon as possible to guarantee the quality of the test. We recommend to use fresh samples.

For longer storage, the samples must be frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation. Freezing and thawing cycles are not recommended.

Perform the sample preparation according to the recommendations appearing in the instructions for use of extraction kit used.

8.1.1. RNA EXTRACTION

For RNA extraction from human stool samples you can use your manually or automatic routine optimized system. Also, you can use any commercially available RNA extraction kit and follow the manufacturer's instructions for use. We have validated the following extraction kits:

- Viasure RNA-DNA Extraction kit (VIASURE), recommended.
- QIAamp MinElute Virus Spin Kit (QIAGEN)
- QIAamp Viral RNA Mini Kit (QIAGEN)
- Invisorb® Spin Universal Kit (Strattec).
- RIDA® Xtract (R-biopharm).
- Maxwell® RSC Blood DNA Kit, using the Maxwell® 16 instrument (Promega).
- NucliSENS® EasyMAG™ platform (bioMérieux).

8.2. LYOPHILIZED POSITIVE CONTROL

Norovirus Positive Control contains high copies template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Norovirus* Positive Control (red vial) adding 100 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.

8.3. PCR PROTOCOL

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run. Peel off protective aluminium seal from plates or strips.

1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

2) Adding samples and controls.

Add 5 µL of RNA sample, reconstituted *Norovirus* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the wells with the caps provided. Centrifuge briefly.

Load the plate or the strips in the thermocycler.

3) Set up your thermocycler.

Program your thermocycler following the conditions below and start the run:

Cycles	Step	Time	Temperature
1	Reverse transcription	15 min	45°C
1	Initial denaturation	2 min	95°C
45	Denaturation	10 seg	95°C
	Annealing/Extension (Data collection*)	50 seg	60°C

Table 3. PCR protocol

Fluorogenic data should be collected during the extension step (*) through the FAM (Norovirus GII), Cy5 (Norovirus GI) and HEX, JOE or VIC channels (Internal Control (IC)). Depending on the equipment used select the proper detection channel (see Annex 2). In Applied Biosystems 7500 Fast Real-Time PCR System and Stratagene Mx3005P™ Real Time PCR System check that passive reference option ROX is none.

9. Result interpretation

The use of positive and negative controls in each run, validate the reaction by checking the absence of signal in negative control well and the presence of signal for *Norovirus* positive control well. Check Internal Control signal to verify the correct functioning of the amplification mix. The analysis of the samples is done by the software itself of the used real time PCR equipment according to manufacturer's instructions.

Using the following table read and analyse the results:

Norovirus GII (FAM)	Norovirus GI (Cy5)	Internal control (HEX)	Negative control	Positive control	Interpretation
+	+	+/-	-	+	Norovirus GI and GII Positive
-	-	+	-	+	Norovirus GI and GII Negative
+	-	+/-	-	+	Norovirus GII Positive, Norovirus GI Negative
-	+	+/-	-	+	Norovirus GI Positive, Norovirus GII Negative
+	+	+	+	+	Experiment fail
-	-	-	-	-	Experiment fail

Table 4. Sample interpretation


+: Amplification curve
-: No amplification curve

A sample is considered positive if the Ct value obtained is less than 40 and the internal control shows or not an amplification signal. Sometimes, the detection of internal control is not necessary because a high copy number of target can cause preferential amplification of target-specific nucleic acids.

REAL TIME PCR DETECTION KIT

A sample is considered negative, if the sample shows no amplification signal in the detection system but the internal control is positive. An inhibition of the PCR reaction can be excluded by the amplification of internal control.

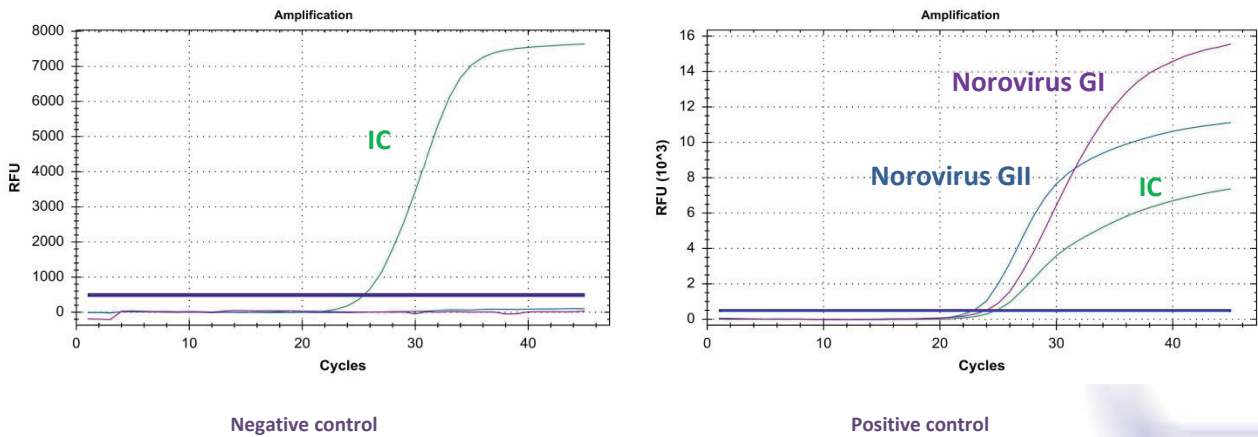Figure 1. Correct run of negative and positive control run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

The result is considered invalid if there is signal of amplification in negative control or absence of signal in the positive well. We recommend to repeat the assay again.

In case of absence of internal control signal in sample wells we recommend to repeat the assay diluting the sample 1:10 or to repeat the extraction to check for possible problems of inhibition.

10. Limitations of the test

- The result of the test should be evaluated by a health care professional and evaluated in the context of medical history, clinical symptoms and other diagnostic tests.
- Although this assay can be used with other types of samples it has been validated with human faecal samples.
- The quality of the test depends on the quality of the sample; proper RNA from faecal specimens must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by Norovirus GI and/or GII, either samples containing high concentrations of target RNA or contamination due to PCR products from previous reactions.

11. Quality control

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) in each well confirms the correct performance of the technique.

12. Performance characteristics

12.1. CLINICAL SENSITIVITY AND SPECIFICITY

The clinical performance of VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit was tested using 79 faecal samples from symptomatic patients. These results were compared with those obtained by molecular detection method (“RIDA®GENE *Norovirus I&II*” (r-Biopharm)).

The results were as follows:

VIASURE <i>Norovirus GI + GII</i> Real Time PCR Detection Kit	RIDA®GENE <i>Norovirus I&II</i> (r-Biopharm)			Total
		+	-	
	+	15	5*	20
	-	0	59	59
Total	15	64	79	

Table 5. Comparative results for *Norovirus GI*.

*The low amount of template RNA in this respiratory sample is below the detection limit of the method used.

VIASURE <i>Norovirus GI + GII</i> Real Time PCR Detection Kit	RIDA®GENE <i>Norovirus I&II</i> (r-Biopharm)			Total
		+	-	
	+	47	3*	50
	-	0	29	29
Total	47	32	79	

Table 6. Comparative results for *Norovirus GII*.

*The low amount of template RNA in this respiratory sample is below the detection limit of the method used.

The results show a high sensitivity and specificity to detect *Norovirus GI* and *Norovirus GII* using VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit.

12.2. ANALYTICAL SENSITIVITY

VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit has a detection limit of ≥ 10 viral RNA copies per reaction for *Norovirus GI* and *Norovirus GII* (Figure 2 and 3).

Figure 2. Dilution series of Norovirus GII (10^7 - 10^1 copies/rxn) template run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System (FAM channel).


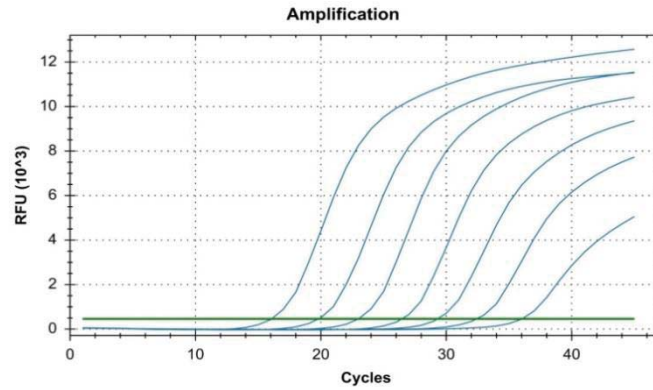
Figure 3. Dilution series of Norovirus GI (10^7 - 10^1 copies/rxn) template run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System (Cy5 channel).

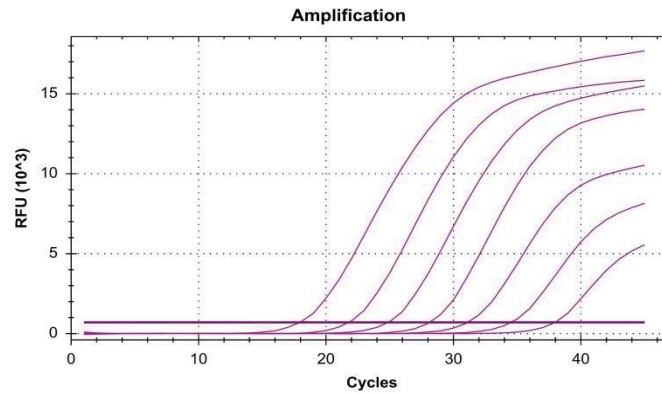12.3. ANALYTICAL SPECIFICITY

The specificity of the Norovirus GI and GII assays were confirmed by testing a panel consisting of different microorganisms representing the most common enteric pathogens or flora present in the intestine. No cross-reactivity of Norovirus was detected against any of the following microorganisms tested.

Cross-reactivity testing					
<i>Helicobacter pylori</i>	-	<i>Campylobacter coli</i>	-	<i>Enterococcus faecalis</i>	-
<i>Helicobacter hepaticus</i>	-	<i>Campylobacter jejuni subsp. jejuni</i>	-	<i>Yersinia enterocolitica</i> O:3	-
<i>Helicobacter cinaedi</i>	-	<i>Campylobacter upsaliensis</i>	-	<i>Yersinia enterocolitica</i> O:9	-
<i>Helicobacter heilmannii</i>	-	<i>Proteus vulgaris</i>	-	<i>Bacteroides fragilis</i>	-
<i>Shigella flexneri</i>	-	<i>Citrobacter freundii</i>	-	<i>Aeromonas hydrophila subsp. hydrophila</i>	-
<i>Shigella dysenteriae</i>	-	<i>Staphylococcus aureus subsp. aureus</i>	-	<i>Aeromonas caviae</i>	-
<i>Salmonella typhi</i>	-	<i>Serratia liquefaciens</i>	-	Adenovirus serotypes 40/41	-
<i>Salmonella paratyphi A</i>	-	<i>Vibrio parahaemolyticus</i>	-	Rotavirus A	-
<i>Salmonella paratyphi B</i>	-	<i>Clostridium difficile</i>	-	Astrovirus Genotype I-VIII	-
<i>Salmonella typhimurium</i>	-	<i>Clostridium perfringens</i>	-	Sapovirus	-
<i>Salmonella bongori</i>	-	<i>Enteropathogenic Escherichia coli</i>	-	<i>Entamoeba histolytica</i>	-
<i>Salmonella enteritidis</i>	-	<i>Enterotoxigenic Escherichia coli</i>	-	<i>Entamoeba dispar</i>	-
<i>Salmonella enterica subsp. enterica</i>	-	<i>Klebsiella oxytoca</i>	-	<i>Dientamoeba fragilis</i>	-
<i>Salmonella pullorum</i>	-	<i>Listeria monocytogenes</i>	-	<i>Cryptosporidium parvum/hominis</i>	-
<i>Salmonella gallinarum</i>	-	<i>Candida albicans</i>	-	<i>Giardia intestinalis</i>	-
<i>Campylobacter lari</i>	-	<i>Arcobacter butzleri</i>	-	<i>Blastocystis hominis</i>	-
<i>Campylobacter fetus</i>	-	<i>Pseudomonas aeruginosa</i>	-		

Table 7. Reference pathogenic microorganisms used in this study.

12.4. ANALYTICAL REACTIVITY

The reactivity of VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit for Norovirus GI was evaluated against genotypes GI.1, GI.3, GI.7, GI.8 and GI.Pb, showing positive results.

The reactivity of VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit for Norovirus GII was evaluated against genotypes GII.1, GII.2, GII.3, GII.4, GII.4 New Orleans 2009, GII.4 Sydney 2012, GII.5, GII.6, GII.7, GII.10, GII.12, GII.13, GII.16, GII.17, GII.21, GII.b, GII.P4, GII.P17, GII.P21, GII.Pe and GII.Pg, showing positive results.

ANNEX 1:

COMPATIBILITY OF THE MOST COMMON REAL TIME PCR EQUIPMENT

Low profile strips can be used in all PCR thermocyclers equipped with low profile block, like systems listed in table A.1. High profile strips can be used in all PCR thermocyclers equipped with high or regular profile block, like systems listed in table A.2. If you do not find your thermocycler in the list below, please contact with your supplier.

Table A.1 LOW PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7500 Fast Real-Time PCR System
Applied Biosystems	7500 Fast Dx Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well Fast
Applied Biosystems	QuantStudio™ 6 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 7 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	ViiA™ 7 Fast Real-Time PCR System
Bio-Rad	CFX96 Touch™ Real-Time PCR Detection System
Roche	LightCycler® 480 Real-Time PCR System
Roche	LightCycler® 96 Real-Time PCR System
Agilent Technologies	AriaMx Real-Time PCR System
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cyclers
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®**

Table A.2 HIGH PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	QuantStudio™ 7 Flex 96-well
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	ViiA™ 7 Real-Time PCR System
Bio-Rad	CFX96 Touch™ Deep Well Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ 5 Real-Time PCR Detection System
Eppendorf	Mastercycler™ep <i>realplex</i>
Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
Analytik Jena Biometra	TOptical
Analytik Jena Biometra	qTOWER 2.0
Abbott	Abbott m2000 RealTime System
BIONEER	Exicycler™ 96
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cyclers
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®**

Table A1/A2. Compatible low and high profile Real Time PCR systems.

* The product should be reconstituted following the appropriate procedure (see Test Procedure) and transferred into specific Rotor-Gene® Q or SmartCycler® tubes.

ANNEX 2:

DETECTION CHANNELS OF MOST COMMON REAL TIME PCR EQUIPMENT

The fluorescence detection channels of some of most common Real Time PCR Thermocyclers are specified in Table A3.

REAL-TIME PCR THERMOCYCLER	VIASURE CHANNEL	DETECTION CHANNEL	OBSERVATIONS
Bio-Rad CFX96™	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
ABI 7500 Applied Biosystems	FAM	FAM	Passive reference option ROX is none
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
Roche Lightcycler®480II	FAM	465/510	Colour Compensation required
	HEX	533/580	
	ROX	533/610	
	Cy5	618/660	
Smartcycler® Cepheid	FAM	Channel 1	
	HEX	Channel 2	
	ROX	Channel 3	
	Cy5	Channel 4	
Abbott m2000rt	FAM	FAM	
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
Mx3000P™ Mx 3005P™ Stratagene	FAM	FAM	Passive reference option ROX is none
	HEX	VIC	
	ROX	ROX	
	Cy5	Cy5	
AriaMx Agilent	FAM	FAM	
	HEX	HEX	
	ROX	ROX	
	Cy5	Cy5	
Rotor-Gene®Q Qiagen	FAM	Green	
	HEX	Yellow	
	ROX	Orange	
	Cy5	Red	

Table A3: Detection fluorescence channels of different Real Time PCR systems.

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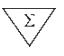
13. Bibliography/Bibliografía

1. C. Fuentes *et al.* Standardized multiplex one-step qRT-PCR for hepatitis A virus, norovirus GI and GII quantification in bivalve mollusks and water. *Food Microbiology* 2014; 40: 55-63.
2. T. Kageyama *et al.* Broadly reactive and highly sensitive assay for Norwalk-like viruses based on real-time quantitative reverse transcription-PCR. *Journal of Clinical Microbiology* 2003; 41(4): 1548-1557.
3. N. Jothikumar *et al.* Rapid and sensitive detection of noroviruses by using TaqMan-based one-step reverse transcription-PCR assays and application to naturally contaminated shellfish samples. *Applied and Environmental Microbiology* 2005; 71(4): 1870-1875.

4. J. Vinjé. Advances in laboratory methods for detection and typing of norovirus. *Journal of Clinical Microbiology* 2015; 53(2): 373-381.

14. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico *in vitro*

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VIASURE *Norovirus GI + GII* Real Time PCR Detection Kit has been validated on the following equipments: Applied Biosystems 7500 Fast Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System, Agilent Technologies AriaMx Real-Time PCR System, DNA-Technology DTprime Real-time Detection Thermal Cycler, Rotor-Gene® Q (Qiagen) and SmartCycler® (Cepheid). When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder to reduce the risk of crushed tube (Ref. PN 4388506).

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- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.
- ABI®, QuantStudio™ is registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registered trademark of Roche.
- Mx3000P™, Mx3005™ and AriaMx are registered trademarks of Agilent Technologies.
- Mastercycler™ is a registered trademark of Eppendorf.
- Rotor-Gene®Q is a registered trademark of Qiagen.
- SmartCycler® is a registered trademark of Cepheid.

CERTEST BIOTEC S.L.
Pol. Industrial Río Gállego II, Calle J, Nº 1,
50840, San Mateo de Gállego, Zaragoza (SPAIN)

www.certest.es 