CERTEST

Campylobacter

ONE STEP *Campylobacter* CARD
TEST

CERTEST BIOTEC S.L.

CERTEST *Campylobacter*

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ENGLISH

INTENDED USE

CerTest *Campylobacter* one step card test is a coloured chromatographic immunoassay for the qualitative detection of *Campylobacter* in stool samples and *Campylobacter* suspected colonies in stool culture.

CerTest *Campylobacter* card test offers a simple and a highly screening assay to make a presumptive diagnosis of *Campylobacter* infection (campylobacteriosis) and it could be used to identify of suspected isolates of *Campylobacter* from selective media (stool culture).

INTRODUCTION

Campylobacteriosis is the disease caused by the presence of *Campylobacter spp.* The common routes of transmission are faecal-oral, person-to-person sexual contact, ingestion of contaminated food or water, and the eating of raw meat. The onset of disease symptoms usually occurs two to five days after infection, but can range from one to ten days.

There are 16 species and 6 subspecies assigned to the genus *Campylobacter*, of which the most frequently reported in human disease are *C. jejuni* (subspecies *jejuni*) and *C. coli* (99% *C. jejuni*). *C. laridis* and *C. upsaliensis* are also regarded as primary pathogens, but are generally reported far less frequently in cases of human disease.

The most common clinical symptoms of *Campylobacter* infections include diarrhoea (frequently with blood in the faeces), abdominal pain, fever, headache, nausea, and/or vomiting. The symptoms typically last three to six days.

TEST PRINCIPLE

CerTest *Campylobacter* is based on the principle of a qualitative immunochromatographic assay for the determination of *Campylobacter* in stool samples and *Campylobacter* suspected colonies in stool culture.

The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Campylobacter* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Campylobacter*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Campylobacter* monoclonal antibodies-red polystyrene microspheres) which was previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Campylobacter* antibodies present on the membrane (test line) capture the coloured conjugate and the red line will be visible. This band is used to interpret the result.

If the sample is negative, there is no *Campylobacter* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with the red-coloured conjugate complex. The anti-*Campylobacter* antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appear. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest *Campylobacter* test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.


SPECIMEN COLLECTION AND PREPARATION

Stool samples

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):


1. Take out the cap of the collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the collection tube. For liquid samples, add approx. 125µL in the collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).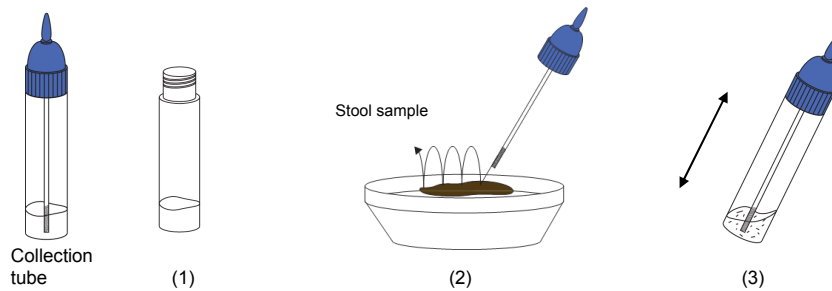


Suspected *Campylobacter* colonies in stool culture

The Butzler or Skirrow media with antibiotics are the methods of choice for the isolation of *Campylobacter* (microaerobic atmosphere, 48 hours/42°C). After 48 hours of incubation in Butzler or Skirrow media the typical *Campylobacter* colonies will be flat, moist, grey-white colonies with irregular spreading margins.

1. Examine Butzler or Skirrow plates after 2 days incubation. Select *Campylobacter* typical colonies. Take out the cap of the collection tube (1). Use the stick or an inoculating needle to pick up 3 or 4 suspected *Campylobacter* colonies (2) and add them to the collection tube.
2. Close the tube with the diluent and suspected colonies. Shake the tube in order to assure good sample dispersion (3).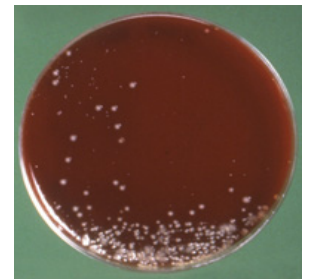

Campylobacter colonies in Butzler or Skirrow media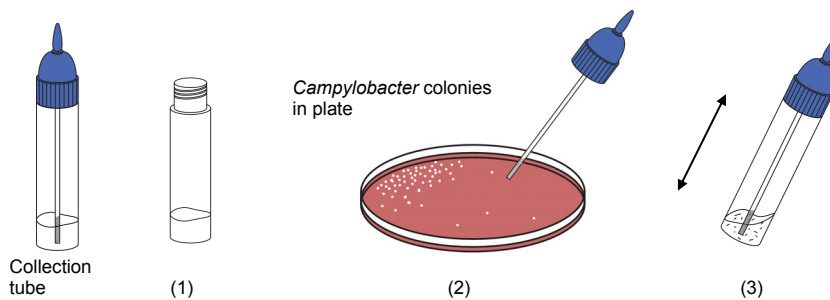

MATERIALS

MATERIALS PROVIDED

- CerTest *Campylobacter* card tests
- Instructions for use
- Collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED


- Specimen collection container
- Disposable gloves
- Timer
- Butzler media/Skirrow media

TEST PROCEDURE


Allow tests, samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the collection tube in order to assure good sample dispersion.
2. Remove the CerTest *Campylobacter* card test from its sealed bag just before using it.
3. Take the collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.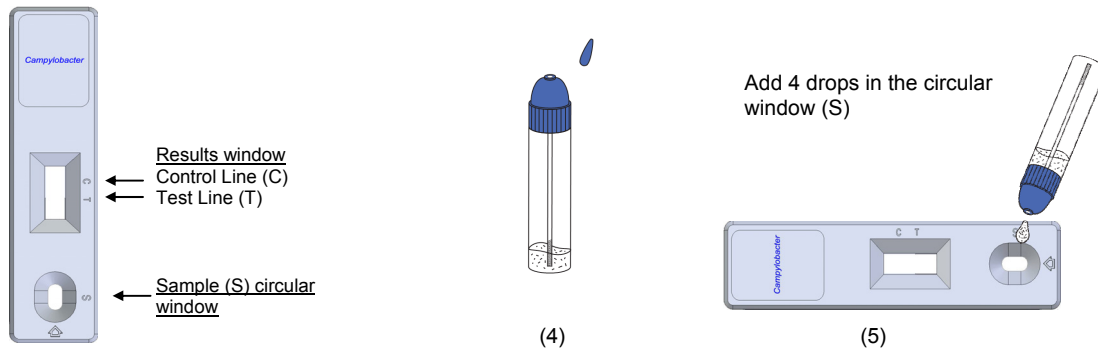

INTERPRETATION OF RESULTS (please refer to the illustration below)

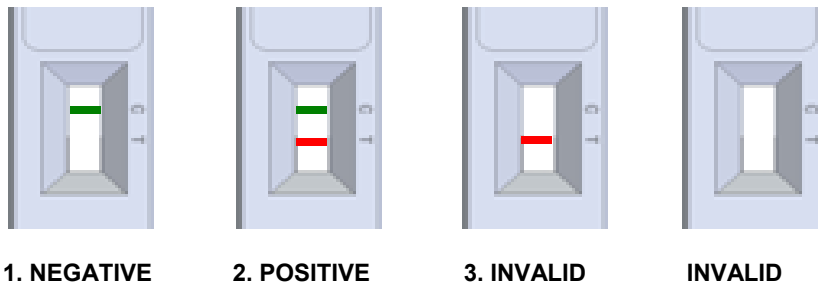	<i>Campylobacter</i>	Interpretation of the results
1.	- GREEN	There is no <i>Campylobacter</i> presence. No infection caused by <i>Campylobacter</i>
2.	+ GREEN-RED	There is <i>Campylobacter</i> presence. <i>Campylobacter</i> infection, include diarrhoea (frequently with blood in the faeces), abdominal pain, fever, headache, nausea and/or vomiting.
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

NEGATIVE: Only the GREEN line appears across the results window in the site marked with the letter C (control line).

POSITIVE: In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

INVALID: Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

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NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentrations to faint when the antigen concentration is closed to the detection limit value of the test.
4. CerTest *Campylobacter* should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
5. Positive results determine the presence of *Campylobacter* in faecal samples; nevertheless, it can be due to several causes and/or species. A positive result should be followed up with additional laboratory techniques (biochemical and serological methods or by PCR) to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
6. A negative result is not meaningful because of it is possible the antigens concentration in the stool sample is lower than the detection limit value. If the symptoms or situation still persist, a *Campylobacter* determination should be carried out on a sample from an enrichment culture.

EXPECTED VALUES

Campylobacter are bacteria that are a major cause of diarrhoeal illness in humans and are generally regarded as the most common bacterial cause of gastroenteritis worldwide. In developed and developing countries, they cause more cases of diarrhoea than, for example, foodborne *Salmonella* bacteria. In developing countries, *Campylobacter* infections in children under the age of two years are especially frequent, sometimes resulting in death. In almost all developed countries, the incidence of human *Campylobacter* infections has been steadily increasing for several years. The reasons for this are unknown.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity (detection limit)

Detection limit values for the different species are 1.07×10^5 CFU/mL for *Campylobacter jejuni* and 1.07×10^8 CFU/mL for *Campylobacter coli*.

Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed comparing the results obtained by an immunochromatographic test (CerTest *Campylobacter*, CerTest) and another immunochromatographic test (ImmunoCard STAT! CAMPY, Meridian Bioscience). The samples were directly taken from patients suffering diarrhoea. The results were as follows:

		IC test: ImmunoCard STAT! CAMPY			
		+	-	Total	
IC test: CerTest <i>Campylobacter</i>	+	42	1	43	
	-	0	44	44	
Total		42	45	87	
		Sensitivity	Specificity	PPV	NPV
CerTest <i>Campylobacter</i> vs ImmunoCard STAT! CAMPY		>99%	98%	98%	>99%

The results showed a high sensitivity and specificity to detect *Campylobacter* using CerTest *Campylobacter*.

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Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest *Campylobacter*; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

<i>Escherichia coli</i> O157:H7	<i>Salmonella typhi</i>	<i>Shigella sonnei</i>
<i>Helicobacter pylori</i>	<i>Salmonella typhimurium</i>	<i>Staphylococcus aureus</i>
<i>Listeria monocytogenes</i>	<i>Shigella boydii</i>	<i>Yersinia enterocolitica</i>
<i>Salmonella enteritidis</i>	<i>Shigella dysenteriae</i>	
<i>Salmonella paratyphi</i>	<i>Shigella flexneri</i>	

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