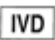
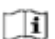
3. L. Ercoli, *et al.* Prevalence and characteristics of verotoxigenic *Escherichia coli* strains isolated from pigs and pork products in Umbria and Marche regions of Italy. *International Journal of Food Microbiology* 2016; 232: 7-14.
4. T.E. Grys *et al.* Rapid and sensitive detection of Shiga toxin-producing *Escherichia coli* from nonenriched stool specimens by real-time PCR in comparison to enzyme immunoassay and culture. *Journal of Clinical Microbiology* 2009; 47(7): 2008-12.
5. J. Madic *et al.* Detection of Shiga toxin-producing *Escherichia coli* serotypes O26:H11, O103:H2, O111:H8, O145:H28, and O157:H7 in raw-milk cheeses by using multiplex real-time PCR. *Applied Environmental Microbiology* 2011; 77(6): 2035-41.
6. K. Verstraete *et al.* A qPCR assay to detect and quantify Shiga toxin-producing *E. coli* (STEC) in cattle and on farms: a potential predictive tool for STEC culture-positive farms. *Toxins (Basel)* 2014; 6(4): 1201-21.
7. Q. Chen *et al.* Rapid genetic typing of diarrheagenic *Escherichia coli* using a two-tube modified molecular beacon based multiplex real-time PCR assay and its clinical application. *Annals of Clinical Microbiology and Antimicrobials* 2014;13:30.
8. J.B. Kape. Pathogenic *Escherichia coli*. *Nature Reviews Microbiology* 2004; 2(2): 123-40.
9. M.A. Croxen *et al.* Recent advances in understanding enteric pathogenic *Escherichia coli*. *Clinical Microbiology Reviews* 2013; 26(4): 822-80.
10. L.E. Bruijnesteijn van Coppenraet *et al.* Case-control comparison of bacterial and protozoan microorganisms associated with gastroenteritis: application of molecular detection. *Clinical Microbiology and Infection* 2015; 21(6): 592.
11. V. D. Thiem *et al.* Detection of Shigella by a PCR assay targeting the *ipaH* gene suggests increased prevalence of shigellosis in Nha Trang, Vietnam. *Journal of Clinical Microbiology* 2004; 42(5): 2031-2035.
12. E.J. Won *et al.* Multiplex Real-Time PCR Assay Targeting Eight Parasites Customized to the Korean Population: Potential Use for Detection in Diarrheal Stool Samples from Gastroenteritis Patients. *PLoS One* 2016;11(11): e0166957.

14. Symbols for IVD components and reagents/Símbolos para reactivos y productos para diagnóstico in vitro

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Tabla A.2 TERMOCICLADORES CON BLOQUE DE PERFIL ALTO	
Fabricante	Modelo
Abbott	Abbott m2000 RealTime System
Applied Biosystems	7300 Real-Time PCR System
Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	7900 HT Real-Time PCR System ⁽⁵⁾
Applied Biosystems	ABI PRISM 7000 ⁽⁶⁾
Applied Biosystems	ABI PRISM 7700 ⁽⁵⁾
Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	QuantStudio™ 7 Flex 96-well
Applied Biosystems	QuantStudio™ 3 Real-Time PCR System ⁽⁵⁾
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	ViiA™ 7 Real-Time PCR System
Analytik Jena Biometra	TOptical
Analytik Jena Biometra	qTOWER 2.0
BIONEER	Exicycler™ 96
Bio-Rad	CFX96™ Deep Well / CFX96™ Deep Well IVD Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Bio-Rad	iCycler iQ™5 Real-Time PCR Detection System
Bio-Rad	MyiQ™ Real-Time PCR Detection System ⁽⁶⁾
Bio-Rad	MyiQ™2 Real-Time PCR Detection System ⁽⁶⁾
Cepheid	SmartCycler® ⁽³⁾
DNA-Technology	DTprime Real-time Detection Thermal Cycler ⁽²⁾
DNA-Technology	DTlite Real-Time PCR System ⁽²⁾
Eppendorf	Mastercycler™ep realplex
Qiagen	Rotor-Gene® Q ⁽³⁾
Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
VIASURE	VIASURE 48 Real Time PCR System ⁽²⁾
VIASURE	VIASURE 96 Real Time PCR System ⁽²⁾

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For information purposes only

- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories.
- ABI®, QuantStudio™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registered trademark of Roche.
- Mx3000P™, Mx3005™ and AriaMx are registered trademarks of Agilent Technologies.
- Mastercycler™ is a registered trademark of Eppendorf.
- Rotor-Gene®Q is a registered trademark of Qiagen.
- SmartCycler® is a registered trademark of Cepheid.

Revision: November 2018

For information purposes only

For information purposes only

For information purposes only

For information purposes only

CerTest Biotec, S.L.
Pol. Industrial Río Gállego II · Calle J, Nº1
50840, San Mateo de Gállego, Zaragoza (Spain)
www.cerTEST.es

VIASURE online

F-362 rev01

VIASURE

Real Time PCR Detection Kits

CerTest
BIOTEC