---

# CERTEST

## *Enterovirus*

---

ONE STEP *Enterovirus*  
CARD TEST

---

CERTEST BIOTEC S.L.

---

For Information Only

Purposes

For Information Purposes Only

**ENGLISH****INTENDED USE**

CerTest *Enterovirus* one step card test is a coloured chromatographic immunoassay for the qualitative detection of *Enterovirus* in stool samples. CerTest *Enterovirus* card test offers a simple and a highly sensitive screening assay to make a presumptive diagnosis of *Enterovirus* infection.

**INTRODUCTION**

*Enterovirus* belongs to the *Picornaviridae* family and are the smallest, non-enveloped viruses known to infect both humans and animals. The spread of enteroviral infections is mainly by the faecal-oral and oral-oral route, but also through direct contact with secretions from ophthalmic and dermal lesions. *Enteroviruses* are single-stranded RNA viruses. Water, food and soil contaminated by infected faeces are an exogenous infection source which creates many opportunities for the transfer of the infection, and cause an epidemic outbreak in a short period of time. The average incubation period for enteroviral contagious is from 3–10 to 30 days.

The virus, after replicating and breaking the gastrointestinal tract barrier, is transmitted via the blood stream to every organ of the body. *Enterovirus* reveals tropism towards organs like the heart, skin, and in particular the central nervous system. It has been proved that infected people excrete large quantities of the virus in faeces for a period of even 16 weeks.

Traditional division organizes this taxonomic group into the subgenera polioviruses, coxsackieviruses (group A, B), echoviruses and a group of *Enterovirus* marked according to their serotype number (66–71 and newly identified 73–75, 77, 78). In 2003, the International Committee on Taxonomy of Viruses created a new taxonomy classification. *Enterovirus* henceforth was divided into 5 groups of species based on their molecular properties (HEV-A, HEV-B, HEV-C, HEV-D and poliovirus).

**TEST PRINCIPLE**

CerTest *Enterovirus* is based on the principle of a qualitative immunochromatographic assay for the determination of *Enterovirus* (VP1 peptide) in stool samples.

The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Enterovirus* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Enterovirus*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Enterovirus* monoclonal antibodies-red polystyrene microspheres), which was previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Enterovirus* antibodies present on the membrane (test line) capture the coloured conjugate and the red line will be visible. This band is used to interpret the result.

If the sample is negative, there is no *Enterovirus* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with the red-coloured conjugate complex. The anti-*Enterovirus* antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appears. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

**STORAGE AND STABILITY**

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

**PRECAUTIONS**


- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest *Enterovirus* test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

## SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

### Specimen preparation (see illustration):

1. Take out the cap of the stool collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the stool collection tube. For liquid samples, add approx. 125µL in the stool collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).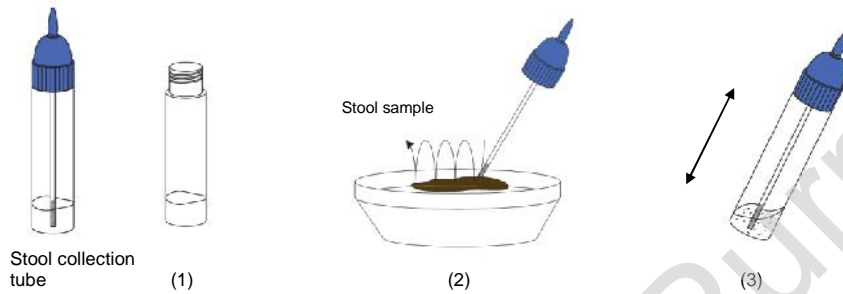

## MATERIALS

### MATERIALS PROVIDED

- CerTest *Enterovirus* card tests
- Instructions for use
- Stool collection tubes with diluent

### MATERIALS REQUIRED BUT NO PROVIDED


- Specimen collection container
- Disposable gloves
- Timer

## TEST PROCEDURE


Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove the CerTest *Enterovirus* card test from its sealed bag just before using it.
3. Take the stool collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.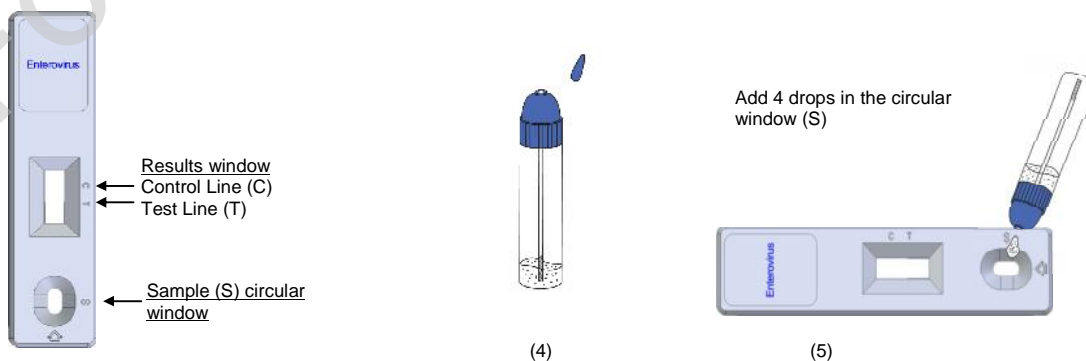

## INTERPRETATION OF RESULTS (please refer to the illustration below)

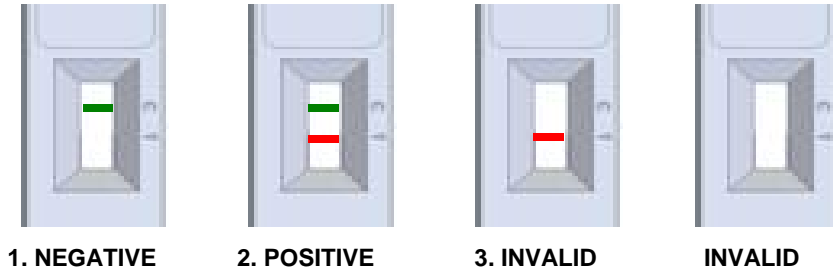	<i>Enterovirus</i>	Interpretation of the results
1.	- GREEN	There is no <i>Enterovirus</i> presence. No viral infection caused by <i>Enterovirus</i> .
2.	+ GREEN-RED	There is <i>Enterovirus</i> presence. Viral infection caused by <i>Enterovirus</i> .
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

**NEGATIVE:** Only the GREEN line appears across the results window in the site marked with the letter C (control line).

**POSITIVE:** In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

**INVALID:** Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

### NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

### QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

### LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. After one month of infection, the number of viruses in faeces is decreasing, making the sample less reactive. Stool samples could be collected previously to the onset of symptoms or also at 24-48 hours.
5. If the patient has been recently vaccinated (for example against *Poliovirus*), it could appear a positive result.
6. CerTest *Enterovirus* should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
7. Positive results determine the presence of *Enterovirus* in faecal samples; nevertheless, it can be due to several species. A positive result should be followed up with diagnostic procedures to confirm the results and the species. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
8. A negative result is not meaningful because of it is possible the antigens concentration in the stool sample is lower than the detection limit value. If the symptoms or situation still persist, an *Enterovirus* determination should be carried out with another technique (for example microscopy, biochemical method).

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## EXPECTED VALUES

There is worldwide circulation of *Enteroviruses*. Seasonal aseptic meningitis outbreaks due to non-polio *Enteroviruses* have been noted to peak in summer till early autumn in the Northern Hemisphere.

While the prevalent serotype varies from year to year, with cocirculation of several serotypes a common observation, recent epidemics have been attributed primarily to Echovirus 30 (E30).

## PERFORMANCE CHARACTERISTICS

### Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed comparing the results obtained by an immunochromatographic test (CerTest *Enterovirus*, CerTest) and another commercial *Enterovirus* assay (IDEIA *Enterovirus* assay, Dako and IMAGEN™ *Enterovirus*, Oxoid). The results were as follows:

		IDEIA <i>Enterovirus</i> assay and IMAGEN™ <i>Enterovirus</i>		
		+	-	Total
IC test: CerTest <i>Enterovirus</i>	+	3	0	3
	-	0	32	32
Total		3	32	35

	Sensitivity	Specificity	PPV	NPV
CerTest <i>Enterovirus</i> vs IDEIA <i>Enterovirus</i> assay and IMAGEN™ <i>Enterovirus</i>	>99%	>99%	>99%	>99%

The results showed a high sensitivity and specificity to detect *Enterovirus* using CerTest *Enterovirus*.

### Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest *Enterovirus*; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

- |                                 |                               |                                |
|---------------------------------|-------------------------------|--------------------------------|
| <i>Astrovirus</i>               | <i>Helicobacter pylori</i>    | <i>Shigella boydii</i>         |
| <i>Campylobacter coli</i>       | <i>Listeria monocytogenes</i> | <i>Shigella dysenteriae</i>    |
| <i>Campylobacter jejuni</i>     | <i>Norovirus</i>              | <i>Shigella flexneri</i>       |
| <i>Clostridium difficile</i>    | <i>Rotavirus</i>              | <i>Shigella sonnei</i>         |
| <i>Cryptosporidium parvum</i>   | <i>Salmonella enteritidis</i> | <i>Staphylococcus aureus</i>   |
| <i>Entamoeba histolytica</i>    | <i>Salmonella paratyphi</i>   | <i>Yersinia enterocolitica</i> |
| <i>Escherichia coli</i> O157:H7 | <i>Salmonella typhimurium</i> |                                |
| <i>Giardia lamblia</i>          | <i>Salmonella typhi</i>       |                                |

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1. RAJTAR, B. et al. "Enteroviruses in water environment-a potential threat to public health". *Ann. Agric. Environ. Med.*, 2008, No 15, p. 199-203.
2. BAEK, K. et al. "Molecular and Epidemiological Characterization of *Enteroviruses* Isolated in Chungnam, Korea from 2005-2006." *J. Microbiol. Biotechnol.*, 2009, 19(9), p. 1055-1064.
3. FONG, T. et al. "Enteric Viruses of Humans and Animals in Aquatic Environments: Health Risks, Detection, and Potential Water Quality Assessment Tools". *Microbiology and Molecular Biology Reviews*, June 2005, Vol. 69, No. 2: p. 357-371.
4. AFFFI, S. et al. "Isolation and Identification of Non-Polio *Enteroviruses* from Children in Different Egyptian Governorates", *Australian Journal of Basic and Applied Sciences*, 2009, Vol. 3, No. 4: pp. 3230-3238.

## SYMBOLS FOR IVD COMPONENTS AND REAGENTS/SÍMBOLOS PARA REACTIVOS Y PRODUCTOS PARA DIAGNÓSTICO IN VITRO

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