CERTEST ROTA+ ADENO+NORO

ONE STEP *Rotavirus*,
Adenovirus and *Norovirus*
COMBO CARD TEST

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ENGLISH

INTENDED USE

CerTest ROTA+ADENO+NORO one step combo card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of *Rotavirus*, *Adenovirus* and *Norovirus* in stool samples.

CerTest ROTA+ADENO+NORO combo card test offers a simple and highly sensitive screening assay to make a presumptive diagnosis of *Rotavirus*, *Adenovirus* and/or *Norovirus* infection.

INTRODUCTION

Rotavirus, *Adenovirus* and *Astrovirus* are most common and major causes of severe gastroenteritis in infants and young children. Pattern also observed in adults. They are transmitted by faecal-oral contact. The main symptoms of viral gastroenteritis are watery diarrhoea and vomiting. The affected person may also have headache, fever and abdominal cramps ("stomach ache"). In general, the symptoms begin 1 to 2 days following infection with a virus that causes gastroenteritis and may last for 1 to 10 days, depending on which virus causes the illness (*Rotavirus* 3 days, *Adenovirus* 5-8 days and *Astrovirus* 3 days).

Noroviruses are the leading cause of epidemic gastroenteritis, including foodborne outbreaks, in the United States. *Noroviruses* belong to the family Caliciviridae and can be grouped into five genogroups (GI through GV), which are further divided into at least 34 genotypes. Genotypes GI, GII and GIV infect humans, causing gastroenteritis, whereas GIII and GV typically infect animals.

The symptoms of *Norovirus* related diseases are those typical of gastroenteritis, that is, vomiting, watery diarrhea and abdominal cramps. Vomiting is a characteristic symptom in the majority of *Norovirus* infections (64% of adults and 81% of children). Other symptoms such as general malaise, low grade fever, nausea and fatigue are also present in over 90% of cases. The incubation period of the disease is generally between 12 and 48 hours, while infection lasts between 12 and 60 hours. Infection may also be asymptomatic, and thus contribute to the spread of the virus in the community. As a rule, the disease does not have serious consequences, and most patients recover within 1-2 days without complications. Debilitated patients and persons with weaker immune systems such as children, elderly or chronic patients may be affected by more serious forms of the disease. Specially, dehydration may represent a serious complication for children, the elderly and persons with a precarious metabolic balance or cardio circulatory instability.

TEST PRINCIPLE

CerTest ROTA+ADENO+NORO is based on the principle of a qualitative immunochromatographic assay for the determination of *Rotavirus*, *Adenovirus* and *Norovirus* (genogroups I and II) in stool samples.

Strip A consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Rotavirus* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Rotavirus*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

Strip B consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Adenovirus* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Adenovirus*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

Strip C consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Norovirus* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Norovirus*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

If the sample is *Rotavirus* positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Rotavirus* monoclonal antibodies-red polystyrene microspheres) in the strip A, if the sample is *Adenovirus* positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Adenovirus* monoclonal antibodies-red polystyrene microspheres) in the strip B and if the sample is *Norovirus* positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Norovirus* monoclonal antibodies-red polystyrene microspheres) in the strip C, which were previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Rotavirus* antibodies present on the membrane of strip A (test line), the anti-*Adenovirus* antibodies present on the membrane of strip B (test line) and the anti-*Norovirus* antibodies present on the membrane of strip C (test line) capture the coloured conjugate and the red lines will be visible in the three strips. These bands are used to interpret the result.

If the sample is negative, there is no *Rotavirus*, *Adenovirus* and *Norovirus* presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with any red-coloured conjugate complex.

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The anti-*Rotavirus*, anti-*Adenovirus* and anti-*Norovirus* antibodies present on the membranes (test lines) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red lines will not appear.

Whether the sample is positive or not, in the three strips, the mixture continues to move across the membranes to the immobilized specific antibodies placed in the control lines. The anti-specific protein antibodies present on three membranes will capture control green-conjugate complex and three control lines will always appear. The presence of these green lines serve as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS


- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest ROTA+ADENO+NORO combo card test. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool sample as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

1. Take out the cap of the stool collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2), to collect faecal sample (approx. 125mg) and add it to the stool collection tube. For liquid samples, add approx. 125µL in the collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).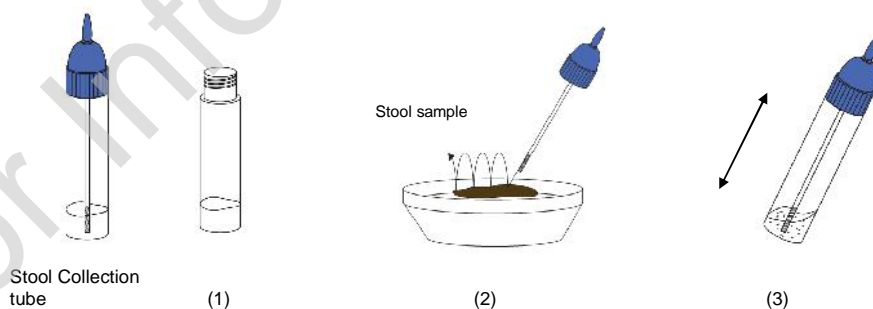

MATERIALS

MATERIALS PROVIDED

- CerTest ROTA+ADENO+NORO combo card tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED

- Specimen collection container
- Disposable gloves
- Timer


TEST PROCEDURE

Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.


1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove CerTest ROTA+ADENO+NORO combo card test from its sealed bag just before using it.
3. Take the stool collection tube, cut the end of the cap (4) and dispense 4 drops in the circular window marked with the letter A (5), 4 drops, using the same tube, in the circular window marked with the letter B (6), and 4 drops, using the same tube, in the circular window marked with the letter C (7). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of extraction diluent until seeing the liquid running through the reaction zone.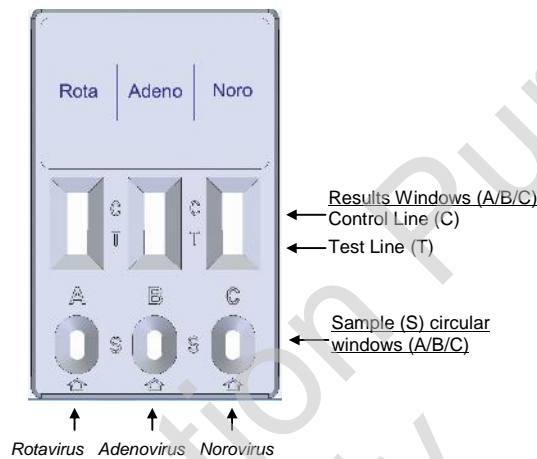


Rotavirus Strip A-procedure

Add 4 drops in A-circular window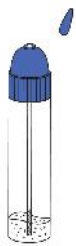


(4)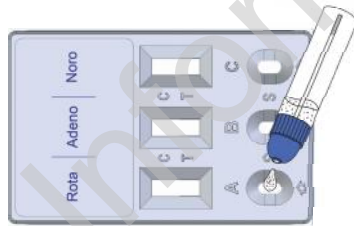

(5)

Adenovirus Strip B-procedure

Add 4 drops in B-circular window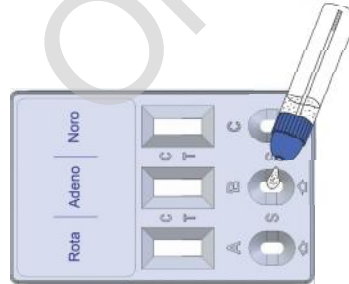

(6)


Norovirus Strip C-procedure

Add 4 drops in C-circular window


(7)


INTERPRETATION OF THE RESULTS (please refer to the illustration below)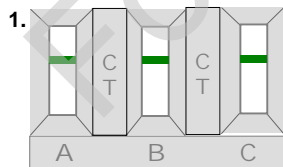


A: Green → Negative Rotavirus
B: Green → Negative Adenovirus
C: Green → Negative Norovirus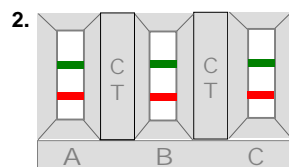


A: Green/Red → Positive Rotavirus
B: Green/Red → Positive Adenovirus
C: Green/Red → Positive Norovirus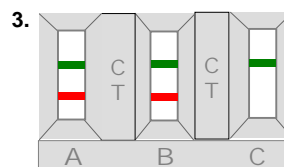


A: Green/Red → Positive Rotavirus
B: Green/Red → Positive Adenovirus
C: Green → Negative Norovirus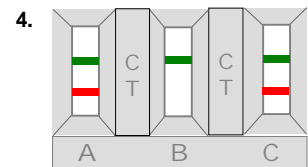


A: Green/Red → Positive Rotavirus
B: Green → Negative Adenovirus
C: Green/Red → Positive Norovirus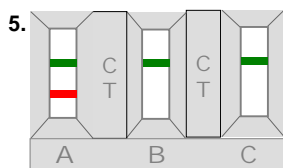


A: Green/Red → Positive Rotavirus
B: Green → Negative Adenovirus
C: Green → Negative Norovirus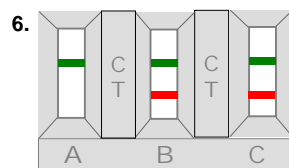

A: Green → Negative Rotavirus
B: Green/Red → Positive Adenovirus
C: Green/Red → Positive Norovirus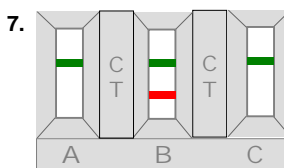

A: Green → Negative Rotavirus
B: Green/Red → Positive Adenovirus
C: Green → Negative Norovirus

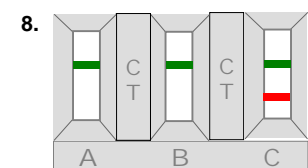A: Green → Negative Rotavirus
B: Green → Negative Adenovirus
C: Green/Red → Positive Norovirus

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	A (Rotavirus)	B (Adenovirus)	C (Norovirus)	Interpretation of the results
1.	- GREEN	- GREEN	- GREEN	There are no <i>Rotavirus</i> , <i>Adenovirus</i> and <i>Norovirus</i> . No viral infection caused by <i>Rotavirus</i> , <i>Adenovirus</i> or <i>Norovirus</i> .
2.	+ GREEN-RED	+ GREEN-RED	+ GREEN-RED	There are <i>Rotavirus</i> , <i>Adenovirus</i> and <i>Norovirus</i> presence. Viral infection caused by <i>Rotavirus</i> , <i>Adenovirus</i> and <i>Norovirus</i> .
3.	+ GREEN-RED	+ GREEN-RED	- GREEN	There is <i>Rotavirus</i> and <i>Adenovirus</i> presence. Viral infection caused by <i>Rotavirus</i> and <i>Adenovirus</i> .
4.	+ GREEN-RED	- GREEN	+ GREEN-RED	There is <i>Rotavirus</i> and <i>Norovirus</i> presence. Viral infection caused by <i>Rotavirus</i> and <i>Norovirus</i> .
5.	+ GREEN-RED	- GREEN	- GREEN	There is <i>Rotavirus</i> presence. Viral infection caused by <i>Rotavirus</i> .
6.	- GREEN	+ GREEN-RED	+ GREEN-RED	There is <i>Adenovirus</i> and <i>Norovirus</i> presence. Viral infection caused by <i>Adenovirus</i> and <i>Norovirus</i> .
7.	- GREEN	+ GREEN-RED	- GREEN	There is <i>Adenovirus</i> presence. Viral infection caused by <i>Adenovirus</i> .
8.	- GREEN	- GREEN	+ GREEN-RED	There is <i>Norovirus</i> presence. Viral infection caused by <i>Norovirus</i> .
9.	Any other result			Invalid result either A, B or C, we recommend repeating the assay using the same sample with another test.

INVALID: Total absence of any control coloured line (GREEN) regardless the appearance or not of the test lines (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control lines failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured bands in the test lines (T) in the results windows will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural controls are included in the test. The green lines appearing in the control lines (C) in the results windows are internal controls, which confirm sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. After one week of infection, the number of viruses in faeces is decreasing, making the sample less reactive. Stool samples should be collected within one week of the onset of symptoms.
5. CerTest ROTA+ADENO+NORO should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
6. Positive results determine the presence of *Rotavirus*, *Adenovirus* and/or *Norovirus* in faecal samples; nevertheless, a positive result should be followed up with additional laboratory techniques to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
7. A negative result is not meaningful because of it is possible the antigens concentration in the stool samples is lower than the detection limit value. If the symptoms or situation still persist, a *Rotavirus*, *Adenovirus* and/or *Norovirus* determination should be carried out with another technique.

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EXPECTED VALUES

Acute gastroenteritis is a common disorder in young children; moreover, the associated dehydration is a leading cause of admission to hospital in industrialized countries. Acute diarrhea is a major health problem throughout the world and main source of mortality in developing countries.

Enteric viruses have been recognized as the most significant etiological agents of the disease and yet, four categories of viruses are being considered clinically relevant: group A *Rotavirus* (family *Reoviridae*), *Norovirus* (family *Caliciviridae*), *Adenovirus* and *Astrovirus*. Several studies proved co-infections in infants 46% with acute watery diarrhea.

In recent years, global public health concern caused by *Norovirus* was raised referring to significant morbidity and occasional mortality, which frequently occurred in schools, hospitals, cruises and other semi-closed institutions. The pathogen is highly contagious with a low infectious dose and spreads by ingestion of contaminated food and/or water or inhaling the agent particle by contact with patients. It has been deduced that approximately 90% of the outbreaks of acute nonbacterial gastroenteritis in the United States were caused by *Norovirus* and 23 million cases were affected annually. In addition to human cost, *Norovirus* infections cause economic losses that approach or exceed \$650,000 in supplies and staff time off. Although previous studies showed the disease was mild and self-limiting, recent studies have showed its ability in causing more severe complications than previously expected.

PERFORMANCE CHARACTERISTICS

Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed using an immunochromatographic test (CerTest ROTA+ADENO+NORO, CerTest) and these results were compared with a commercial available test (Ridascreen®*Rotavirus* ELISA Test, r-Biopharm) for *Rotavirus*, another commercial ELISA assay (Simple Norovirus, Operon) for *Norovirus* and the results for *Adenovirus* were confirmed using PCR. The results were as follows:

IC Test: CerTest ROTA+ADENO+NORO (<i>Rotavirus</i>)	Ridascreen® <i>Rotavirus</i> ELISA Test			Total
		+	-	
+	18	1	19	
-	0	43	43	
Total	18	44	62	

	Sensitivity	Specificity	PPV	NPV
CerTest ROTA+ADENO+NORO (<i>Rotavirus</i>) vs Ridascreen® <i>Rotavirus</i> ELISA Test	>99%	98%	>94%	>99%

IC test: CerTest ROTA+ADENO+NORO (<i>Adenovirus</i>)	PCR			Total
		+	-	
+	7	0	7	
-	0	52	52	
Total	7	52	59	

	Sensitivity	Specificity	PPV	NPV
CerTest ROTA+ADENO+NORO (<i>Adenovirus</i>) vs PCR	>99%	>99%	>99%	>99%

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The results showed a high sensitivity and specificity to detect *Rotavirus*, *Adenovirus* and *Norovirus* using CerTest ROTA+ADENO+NORO.

Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest ROTA+ADENO+NORO; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

- | | | |
|-----------------------------------|----------------------------------|--------------------------------|
| <i>Adenovirus</i> (Strip A and C) | <i>Giardia lamblia</i> | <i>Salmonella typhi</i> |
| <i>Astrovirus</i> | <i>Helicobacter pylori</i> | <i>Shigella boydii</i> |
| <i>Campylobacter coli</i> | <i>Hepatitis A</i> | <i>Shigella dysenteriae</i> |
| <i>Campylobacter jejuni</i> | <i>Listeria monocytogenes</i> | <i>Shigella flexneri</i> |
| <i>Clostridium difficile</i> | <i>Norovirus</i> (Strip A and B) | <i>Shigella sonnei</i> |
| <i>Cryptosporidium parvum</i> | <i>Rotavirus</i> (Strip B and C) | <i>Staphylococcus aureus</i> |
| <i>Entamoeba histolytica</i> | <i>Salmonella enteritidis</i> | <i>Yersinia enterocolitica</i> |
| <i>Enterovirus</i> | <i>Salmonella paratyphi</i> | |
| <i>Escherichia coli</i> O157:H7 | <i>Salmonella typhimurium</i> | |

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