REAL TIME PCR DETECTION KIT

Dientamoeba fragilis

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ENGLISH

1. Intended of use

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit is designed for specific identification of *Dientamoeba fragilis* in human stool samples from patients with signs and symptoms of *Dientamoeba fragilis* infection. This test is intended for use as an aid in the diagnosis of *Dientamoeba fragilis* infection in combination with clinical and epidemiological risk factors. DNA is extracted from stool specimens, multiplied using Real Time amplification and detected using specific primers and a fluorescent reporter dye probe for *Dientamoeba fragilis*.

2. Summary and Explanation

Initially classify as an amoeba, *Dientamoeba fragilis* is actually a protozoan parasite that belongs to the *Trichonemad* order. It has been associated with gastrointestinal disease worldwide showing symptoms as diarrhea, loose stools and abdominal pain with a highly variable incidence, ranging from 1.1 to 20%. Its transmission mode remains unclear, and both faecal-oral route and vector transmission have been suggested.

Despite the frequency of *D. fragilis* presence, it is still neglected as a significant pathogen due to the lack of a routinely diagnostic in many laboratories. Detection of *D. fragilis* trophozoites on the permanent stained smear is the current standard. Trophozoites range from 5 to 15 µm in length, 9 to 12 µm in width, and contain 1 to 2 characteristically fragmented nuclei, but because of the “fragile” nature of the organism diagnosis is often only possible by experienced technologists, leading to many potentially missed infections. Recently, PCR-based diagnostic methods have been developed for diagnosis of *D. fragilis*, showing at least a three-fold sensitivity increase compared to permanent stained smears.

3. Principle of the procedure

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit is designed for the diagnosis of *Dientamoeba fragilis* in human stool samples. After DNA isolation, the identification of *Dientamoeba fragilis* is performed by the amplification of a conserved region of the 5.8S rRNA gene using specific primers and a fluorescent-labeled probe. VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit is based on 5' exonuclease activity of DNA polymerase. During DNA amplification, this enzyme cleaves the probe bound to the complementary DNA sequence, separating the quencher dye from the reporter. This reaction generates an increase in the fluorescent signal which is proportional to the quantity of the target template. This fluorescence could be measured on real time PCR platforms.

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit contains in each well all the components necessary for real time PCR assay (specific primers/probes, dNTPS, buffer, polymerase) in a stabilized format, as well as an internal control to discard the inhibition of the polymerase activity. *Dientamoeba fragilis* DNA targets are amplified and detected in FAM channel and the internal control (IC) in HEX, VIC or JOE channel (depending on the equipment used).

4. Reagents provided

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit includes the following materials and reagents detailed in Table 1 and Table 2:

Reference	Reagent/Material	Description	Color	Amount
VS-DIE1SL/ VS-DIE1SH	<i>Dientamoeba fragilis</i> 8-well strips	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	6/12 X 8-well strip
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-DIE1C	<i>Dientamoeba fragilis</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNase/DNase free	Water RNase/DNase free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing wells during thermal cycling	Transparent	6/12 X 8-cap strip

Table 1. Reagents and materials provided in VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit with Ref. VS-DIE106L, VS-DIE106H, VS-DIE112L and VS-DIE112H.

Reference	Reagent/Material	Description	Color	Amount
VS-DIE1PL/ VS-DIE1PH	<i>Dientamoeba fragilis</i> 96-well plate	A mix of enzymes, primers probes, buffer, dNTPs, stabilizers and Internal control in stabilized format	White	1 plate
VS-RB02	Rehydration Buffer	Solution to reconstitute the stabilized product	Blue	1 vial x 1.8 mL
VS-DIE1C	<i>Dientamoeba fragilis</i> Positive Control	Non-infectious synthetic lyophilized DNA	Red	1 vial
VS-NC1	Negative control	Non template control	Violet	1 vial x 1 mL
VS-H2O	Water RNase/DNase free	Water RNase/DNase free	White	1 vial x 1 mL
VS-OCS	Tear-off 8-cap strips	Optical caps for sealing plate during thermal cycling	Transparent	12 X 8-cap strip

Table 2. Reagents and materials provided in VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit with Ref. VS-DIE113L and VS-DIE113H

5. Reagents and equipment to be supplied by the user

The following list includes materials that are required for using, but not included in the VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit.

- Real Time PCR instrument (thermocycler) (to check compatibility see Annex I).
- DNA extraction kit.
- Centrifuge for 1.5 mL tubes.
- Vortex.
- Micropipettes (0.5-20 µL, 20-200 µL).
- Filter tips.
- Powder-free disposal gloves.

6. Transport and storage conditions

- The kits can be shipped and stored at 2-40°C until expiration date stated in the label.
- Once the positive control has been re-suspended, store it at -20°C. We recommend to separate in aliquots to minimize freeze and thaw cycles.
- Keep components away from sunlight.

7. Precautions for users

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- Design a unidirectional workflow. It should begin in the Extraction Area and then move to the Amplification and Detection Area. Do not return samples, equipment and reagents to the area in which the previous step was performed.
- Follow Good Laboratory Practices. Wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area. Once you finish the test wash your hands.
- Specimens must be treated as potentially infectious as well as all reagents and materials that have been exposed to the samples and must be handled according to the national safety regulations. Take necessary precautions during the collection, storage, treatment and disposal of samples.
- Regular decontamination of commonly used equipment is recommended, especially micropipettes and work surfaces.

8. Test procedure

8.1. SAMPLE PREPARATION

Stool samples should be collected in clean containers and processed as soon as possible to guarantee the quality of the test. We recommend to use fresh samples.

For longer storage, the samples must be frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenize stool sample as thoroughly as possible prior to preparation. Freezing and thawing cycles are not recommended.

Perform the sample preparation according to the recommendations appearing in the instructions for use of extraction kit used.

8.1.1. DNA EXTRACTION

For DNA extraction from human stool samples you can use your manually or automatic routine optimized system. Also, you can use any commercially available DNA extraction kit and follow the manufacturer's instructions for use. We have validated the following extraction kits:

- Viasure RNA-DNA Extraction kit (VIASURE), recommended.
- QIAamp DNA Mini kit (QIAGEN)
- QIAamp DNA Stool Mini Kit (QIAGEN)
- Invisorb® Spin Universal Kit (Stratec).
- RIDA® Xtract (R-biopharm).
- Maxwell®RSC Blood DNA Kit, using the Maxwell® 16 instrument (Promega).

8.2. LYOPHILIZED POSITIVE CONTROL

Dientamoeba fragilis Positive Control contains high copies template, the recommendation is to open and manipulate it in a separate laboratory area away from the other components. Reconstitute the lyophilized *Dientamoeba fragilis* Positive Control (red vial) adding 100 µL of Water RNase/DNase free (white vial) supplied and vortex thoroughly.

Once the positive control has been re-suspended, store it at -20°C. Recommendation is to separate it in aliquots to minimize freeze and thaw cycles.

8.3. PCR PROTOCOL

Determine and separate the number of required reactions including samples and controls. One positive and negative control must be included in each run. Peel off protective aluminum seal from plates or strips.

1) Reconstitute the number of wells you need.

Add 15 µL of Rehydration Buffer (blue vial) into each well.

2) Adding samples and controls.

Add 5 µL of DNA sample, reconstituted *Dientamoeba fragilis* Positive Control (red vial) or Negative Control (violet vial) in different wells and close the wells with the caps provided. Centrifuge briefly.

Load the plate or the strips in the thermocycler.

3) Set up your thermocycler.

Program your thermocycler following the conditions below and start the run:

Cycles	Step	Time	Temperature
1	Polymerase activation	2 min	95°C
45	Denaturation	10 seg	95°C
	Annealing/Extension (Data collection*)	50 seg	60°C

Table 3. PCR protocol.

Fluorogenic data should be collected during the extension step (*) through the FAM (*Dientamoeba fragilis*) and HEX, JOE or VIC channels (Internal Control (IC)). In Applied Biosystems 7500 Fast Real-Time PCR System and Applied Biosystems StepOne™ Real-Time PCR System check that passive reference option ROX is none.

9. Result interpretation

The use of positive and negative controls in each run, validates the reaction by checking the absence of signal in negative control well and the presence of signal for *Dientamoeba fragilis* positive control well. Check Internal Control signal to verify the correct functioning of the amplification mix. The analysis of the samples is done by the software itself of the used real time PCR equipment according to manufacturer’s instructions for use.

Using the following table, read and analyze the results:

<i>Dientamoeba fragilis</i>	Internal control	Negative control	Positive control	Interpretation
+	+/-	-	+	<i>Dientamoeba fragilis</i> Positive
-	+	-	+	<i>Dientamoeba fragilis</i> Negative
+	+	+	+	Experiment fail
-	-	-	-	Experiment fail


Table 4. Sample interpretation

- +: Amplification curve
- : No amplification curve

A sample is considered positive if the Ct value obtained is less than 40 and the internal control shows or not an amplification signal. Sometimes, the detection of internal control is not necessary because a high copy number of target can cause preferential amplification of target-specific nucleic acids.

A sample is considered negative, if the sample shows no amplification signal in the detection system but the internal control is positive. An inhibition of the PCR reaction can be excluded by the amplification of internal control.

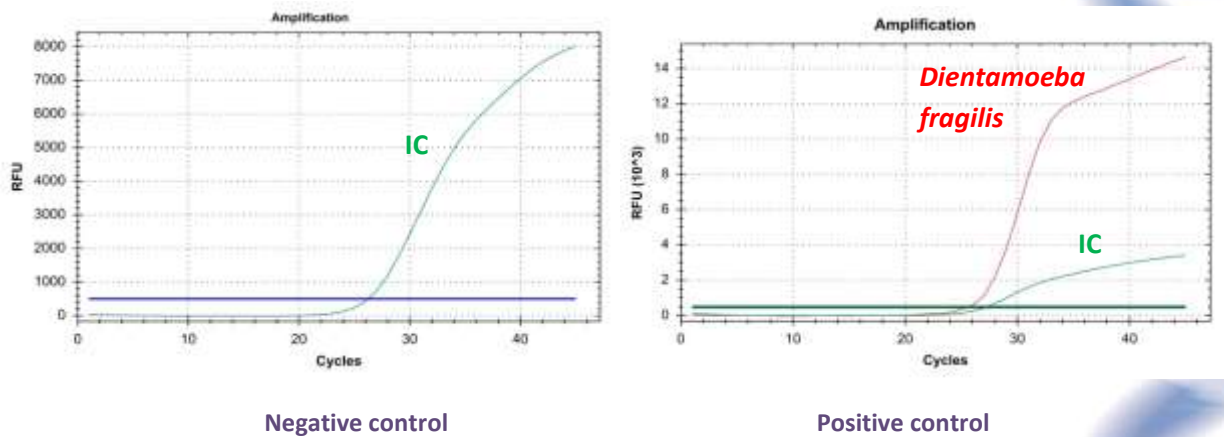Figure 1. Correct run of negative and positive control run on the Bio-Rad CFX96 Touch™ Real-Time PCR Detection System.

The result is considered invalid if there is signal of amplification in negative control or absence of signal in the positive well. We recommend to repeat the assay again.

In case of absence of internal control signal in sample wells we recommend to repeat the assay diluting the sample 1:10 or to repeat the extraction to check for possible problems of inhibition.

10. Limitations of the test

- The result of the test should be evaluated by a health care professional and evaluated in the context of medical history, clinical symptoms and other diagnostic tests.
- Although this assay can be used with other types of samples it has been validated with human faecal samples.
- The quality of the test depends on the quality of the sample; proper DNA from faecal specimens must be extracted. Unsuitable collection, storage and/or transport of specimens may give false negative results.
- Extremely low levels of target below the limit of detection may be detected, but results may not be reproducible.
- There is a possibility of false positive results due to cross-contamination by *Dientamoeba fragilis*, either samples containing high concentrations of target DNA or contamination due to PCR products from previous reactions.

11. Quality control

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit contains a positive and a negative control that must be included in each run to correctly interpret the results. Also, the internal control (IC) in each well confirms the correct performance of the technique.

12. Performance characteristics

12.1. CLINICAL SENSITIVITY AND SPECIFICITY

Overall, a total amount of 82 faecal samples from symptomatic patients were tested using *Dientamoeba fragilis* Real Time PCR Detection Kit. *Dientamoeba fragilis* was detected in 53 samples. These results were compared with those obtained by a commercial Real Time PCR Kits (RIDA[®] GENE *Dientamoeba fragilis* (r-Biopharm)).


VIASURE <i>Dientamoeba fragilis</i> Real Time PCR Detection Kit	RIDA [®] GENE <i>Dientamoeba fragilis</i> (r-biopharm)		
		+	-
+	46	7*	53
-	0	29	29
Total	46	36	82

*The low amount of template (DNA) detected in these samples are below the detection limit of the method used and so it may produce random positive results.

12.2. ANALYTICAL SENSITIVITY

VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit has a detection limit of ≥ 10 DNA copies per reaction (Figure 2).

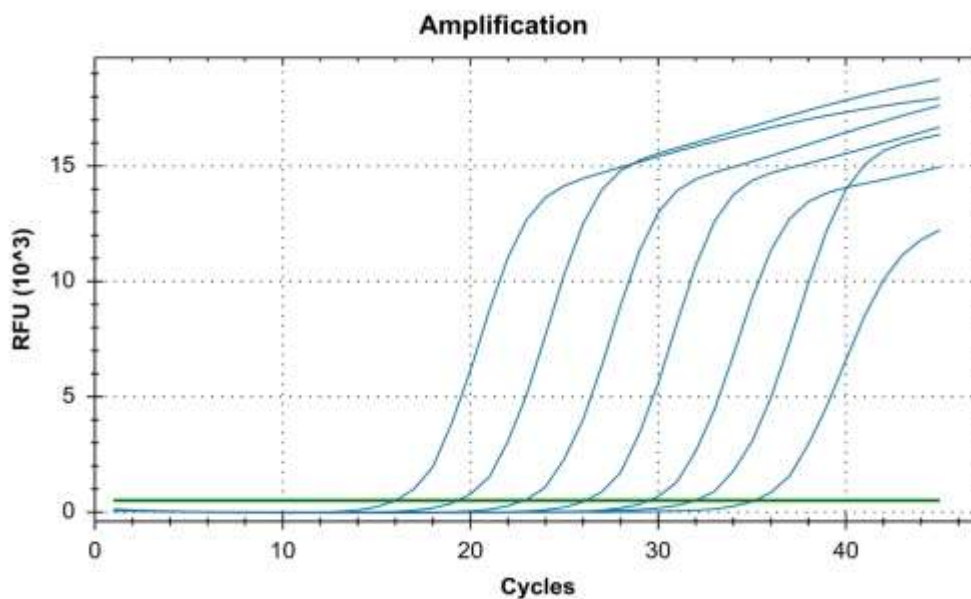Figure 2. Dilution series of *Dientamoeba fragilis* (10^7 - 10^1 copies/rxn) template run on the Bio-Rad CFX96 Touch[™] Real-Time PCR Detection System.

12.3. ANALYTICAL SPECIFICITY

The specificity of the *Dientamoeba fragilis* assay was confirmed by testing a panel consisting of 45 microorganisms representing the most common enteric pathogens or flora present in the intestine. No cross-reactivity was detected against any of the following microorganisms tested.

Cross-reactivity testing					
<i>Helicobacter pylori</i>	-	<i>Campylobacter lari</i>	-	<i>Klebsiella oxytoca</i>	-
<i>Helicobacter hepaticus</i>	-	<i>Campylobacter fetus</i>	-	<i>Listeria monocytogenes</i>	-
<i>Helicobacter cinaedi</i>	-	<i>Campylobacter coli</i>	-	<i>Candida albicans</i>	-
<i>Helicobacter heilmannii</i>	-	<i>Campylobacter jejuni subsp. jejuni</i>	-	<i>Arcobacter butzleri</i>	-
<i>Shigella flexneri</i>	-	<i>Campylobacter upsaliensis</i>	-	<i>Pseudomonas aeruginosa</i>	-
<i>Shigella dysenteriae</i>	-	<i>Proteus vulgaris</i>	-	<i>Enterococcus faecalis</i>	-
<i>Salmonella typhi</i>	-	<i>Aeromonas hydrophila subsp. hydrophila</i>	-	<i>Bacteroides fragilis</i>	-
<i>Salmonella paratyphi A</i>	-	<i>Citrobacter freundii</i>	-	<i>Yersinia enterocolitica O:3 y O:9</i>	-
<i>Salmonella paratyphi B</i>	-	<i>Staphylococcus aureus subsp. aureus</i>	-	<i>Cryptosporidium parvum</i>	-
<i>Salmonella typhimurium</i>	-	<i>Serratia liquefaciens</i>	-	<i>Giardia intestinalis</i>	-
<i>Salmonella bongori</i>	-	<i>Vibrio parahaemolyticus</i>	-	<i>Entamoeba histolytica</i>	-
<i>Salmonella enteritidis</i>	-	<i>Clostridium difficile</i>	-	Adenovirus serotypes 40/41	-
<i>Salmonella enterica subsp. entérica</i>	-	<i>Clostridium perfringens</i>	-	Rotavirus A	-
<i>Salmonella pullorum</i>	-	<i>Enterotoxigenic Escherichia coli</i>	-	Norovirus Genotypes I and II	-
<i>Salmonella gallinarum</i>	-	<i>Enteropathogenic Escherichia coli</i>	-	Astrovirus Genotype I-VIII	-

Table 6. Reference pathogenic and non-pathogenic microorganisms used in this study.

12.4. ANALYTICAL REACTIVITY

The reactivity of VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit was evaluated against *Dientamoeba fragilis* showing positive results.

ANNEX 1:

COMPATIBILITY OF THE MOST COMMON REAL TIME PCR EQUIPMENT

Low profile strips can be used in all PCR thermocyclers equipped with low profile block, like systems listed in table A.1. High profile strips can be used in all PCR thermocyclers equipped with high or regular profile block, like systems listed in table A.2. If you do not find your thermocycler in the list below, please contact with your supplier.

Table A.1 LOW PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7500 Fast Real-Time PCR System
Applied Biosystems	7500 Fast Dx Real-Time PCR System
Applied Biosystems	QuantStudio™ 12K Flex 96-well Fast
Applied Biosystems	QuantStudio™ 6 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 7 Flex 96-well Fast
Applied Biosystems	QuantStudio™ 3 Real-Time PCR System
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	StepOne Plus™ Real-Time PCR System
Applied Biosystems	StepOne™ Real-Time PCR System
Applied Biosystems	ViiA™ 7 Fast Real-Time PCR System
Bio-Rad	CFX96 Touch™ Real-Time PCR Detection System
Bio-Rad	Mini Opticon™ Real-Time PCR Detection System
Roche	LightCycler®480 Real-Time PCR System
Roche	LightCycler®96 Real-Time PCR System
Agilent Technologies	AriaMx Real-Time PCR System
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cycler
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®**

Table A.2 HIGH PROFILE BLOCK THERMOCYCLERS	
Manufacturer	Model
Applied Biosystems	7300 Real-Time PCR System
Applied Biosystems	7500 Real-Time PCR System
Applied Biosystems	7900 HT Real-Time PCR System
Applied Biosystems	ABI PRISM 7000
Applied Biosystems	ABI PRISM 7700
Applied Biosystems	QuantStudio™ 12K Flex 96-well
Applied Biosystems	QuantStudio™ 6 Flex 96-well
Applied Biosystems	QuantStudio™ 7 Flex 96-well
Applied Biosystems	QuantStudio™ 3 Real-Time PCR System
Applied Biosystems	QuantStudio™ 5 Real-Time PCR System
Applied Biosystems	ViiA™ 7 Real-Time PCR System
Bio-Rad	CFX96 Touch™ Deep Well Real-Time PCR Detection System
Bio-Rad	iCycler iQ™ Real-Time PCR Detection System
Bio-Rad	iCycler iQ™5 Real-Time PCR Detection System
Bio-Rad	MyiQ™ Real-Time PCR Detection System
Bio-Rad	MyiQ™2 Real-Time PCR Detection System
Eppendorf	Mastercycler™ep realplex
Stratagene / Agilent Technologies	Mx3000P™ Real Time PCR System
Stratagene / Agilent Technologies	Mx3005P™ Real Time PCR System
Analytik Jena Biometra	TOptical
Analytik Jena Biometra	qTOWER 2.0
Abbott	Abbott m2000 RealTime System
BIONEER	Exicycler™ 96
DNA-Technology	DTlite Real-Time PCR System
DNA-Technology	DTprime Real-time Detection Thermal Cycler
Qiagen	Rotor-Gene® Q*
Cepheid	SmartCycler®**

Table A1/A2. Compatible low and high profile Real Time PCR systems.

* The product should be reconstituted following the appropriate procedure (see Test Procedure) and transferred into specific Rotor-Gene® Q or SmartCycler® tubes.

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VIASURE *Dientamoeba fragilis* Real Time PCR Detection Kit has been validated on the following equipments: Applied Biosystems 7500 Fast Real-Time PCR System, Bio-Rad CFX96 Touch™ Real-Time PCR Detection System, Agilent Technologies AriaMx Real-Time PCR System, DNA-Technology DTprime Real-time Detection Thermal Cycler, Rotor-Gene® Q (Qiagen) and SmartCycler® (Cepheid). When using the Applied Biosystems 7500 Fast with strips it is recommend to place a plate holder to reduce the risk of crushed tube (Ref. PN 4388506).

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- CFX™ and IQ5™ are registered trademarks of Bio-Rad Laboratories
- ABI®, QuantStudio™, StepOnePlus™ and ViiA™ are registered trademarks of Thermo Fisher Scientific Inc.
- LightCycler® is a registered trademark of Roche
- Mx3000P™, Mx3005™ and AriaMx are registered trademarks of Agilent Technologies
- Mastercycler™ is a registered trademark of Eppendorf.
- Rotor-Gene®Q is a registered trademark of Qiagen.
- SmartCycler® is a registered trademark of Cepheid.

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