CERTEST

H. pylori

ONE STEP *Helicobacter pylori*
CARD TEST

CERTEST BIOTEC S.L.

ENGLISH**INTENDED USE**

CerTest *H. pylori* one step card test is a coloured chromatographic immunoassay for the qualitative detection of *Helicobacter pylori* in stool samples.

CerTest *H. pylori* card test offers a simple, a highly sensitive and non-invasive screening assay to make a presumptive diagnosis of *Helicobacter pylori* infection.

INTRODUCTION

Helicobacter pylori (*H. pylori*) is a spiral-shaped bacterium that is found in the gastric mucous layer or adherent to the epithelial lining of the stomach. *H. pylori* causes more than 90% of duodenal ulcers and up to 80% of gastric ulcers.

The importance of *Helicobacter pylori* testing has increased greatly since the strong correlation between the presence of bacteria and confirmed gastrointestinal diseases (stomach and duodenum) like gastritis, peptic ulcer disease and gastric carcinoma.

TEST PRINCIPLE

CerTest *H. pylori* is based on the principle of a qualitative immunochromatographic assay for the determination of *Helicobacter pylori* in stool samples.

The strip consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Helicobacter pylori* and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Helicobacter pylori*) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

If the sample is positive, the antigens of the diluted sample react with the red-coloured conjugate complex (anti-*Helicobacter pylori* monoclonal antibodies-red polystyrene microspheres), which was previously pre-dried on the absorbent pad. The mixture then moves upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*Helicobacter pylori* antibodies present on the membrane (test line) capture the coloured conjugate and the red line will be visible. This band is used to interpret the result.

If the sample is negative, there is no *Helicobacter pylori* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with the red-coloured conjugate complex. The anti-*Helicobacter pylori* antibodies present on the membrane (test line) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red line will not appear.

Whether the sample is positive or not, the mixture continues to move across the membrane to the immobilized specific antibodies placed in the control line. The anti-specific protein antibodies present on the membrane will capture control green-conjugate complex and the control line will always appear. The presence of this green line serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS


- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- The tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the CerTest *H. pylori* test. Do not use any other commercial kit component.
- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

SPECIMEN COLLECTION AND PREPARATION

Stool samples should be collected in clean containers. The samples can be stored in the refrigerator (2-8°C) for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at -20°C. In this case, the sample will be totally thawed and brought to room temperature before testing. Homogenise stool samples as thoroughly as possible prior to preparation.

Specimen preparation (see illustration):

1. Take out the cap of the collection tube (1) and use the stick to pick up sufficient sample quantity. Then, introduce the stick once into 4 different parts of the stool sample (2) making sure that, at each insertion, only the stick's screw is covered with the sample to take the appropriate amount of faecal sample (approx. 50mg) and add it to the collection tube. Any additional sample that exceeds the stick's screw could cause wrong results. For liquid samples, add approx. 125µL in the collection tube using a micropipette.
2. Close the tube with the diluent and stool sample. Shake the tube in order to assure good sample dispersion (3).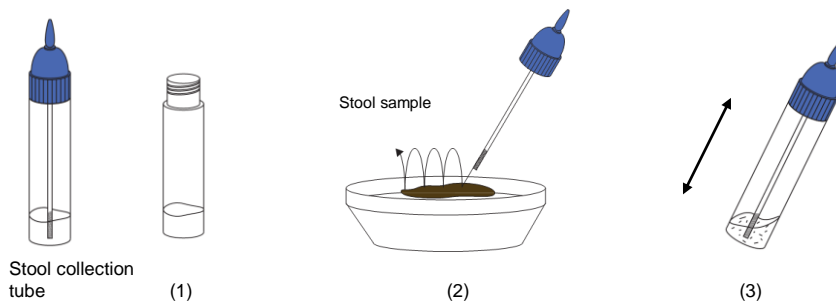

MATERIALS

MATERIALS PROVIDED

- CerTest *H. pylori* card tests
- Instructions for use
- Stool collection tubes with diluent

MATERIALS REQUIRED BUT NO PROVIDED


- Specimen collection container
- Disposable gloves
- Timer

TEST PROCEDURE


Allow tests, stool samples and controls to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Proceed to shake the stool collection tube in order to assure good sample dispersion.
2. Remove the CerTest *H. pylori* card test from its sealed bag just before using it.
3. Take the stool collection tube, cut the end of the cap (4) and dispense 3 drops in the circular window marked with the letter S (5). Avoid adding solid particles with the liquid.
4. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.

If the test does not run due to solid particles, stir the sample added in the sample window (S) with the stick. If it doesn't work, dispense a drop of diluent until seeing the liquid running through the reaction zone.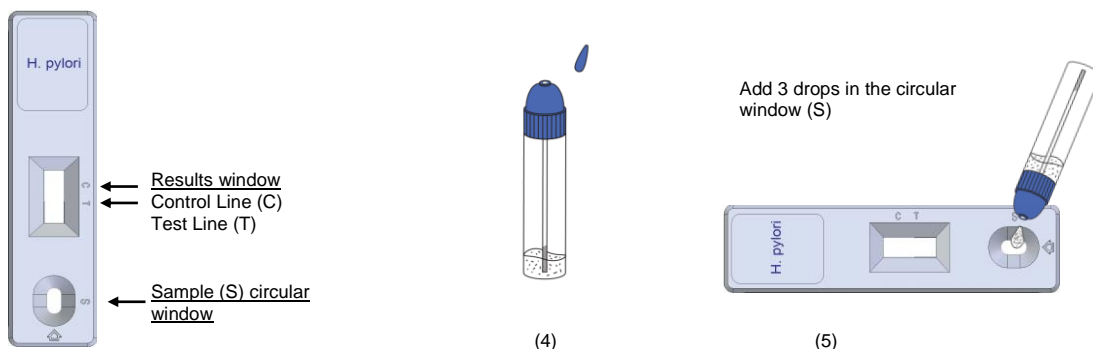

INTERPRETATION OF RESULTS (please refer to the illustration below)

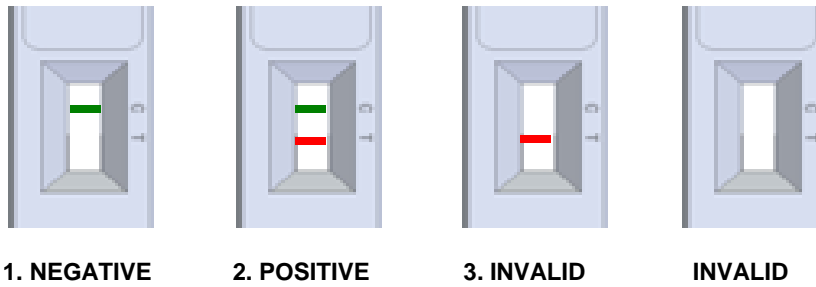	<i>H. pylori</i>	Interpretation of the results
1.	-	There is no <i>Helicobacter pylori</i> presence. No infection caused by <i>Helicobacter pylori</i> .
	GREEN	
2.	+	There is <i>Helicobacter pylori</i> presence. <i>Helicobacter pylori</i> infection, which might mean gastrointestinal diseases (stomach and duodenum) like gastritis, peptic ulcer disease or gastric carcinoma.
	GREEN-RED	
3.	Any other result	Invalid result, we recommend repeating the assay using the same sample with another test.

NEGATIVE: Only the GREEN line appears across the results window in the site marked with the letter C (control line).

POSITIVE: In addition to the GREEN control line, a RED line also appears in the site marked with the letter T (test line) in the results window.

INVALID: Total absence of the control coloured line (GREEN) regardless the appearance or not of the test line (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persist, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured band in the test line (T) in the results window will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

Internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. An excess of sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. CerTest *H. pylori* should be used only with samples from human faeces. The use of other samples has not been established. The quality of the test depends on the quality of the sample; proper faecal specimens must be obtained.
5. Positive results determine the presence of *Helicobacter pylori* in faecal samples; nevertheless, a positive result should be followed up with additional invasive techniques (endoscopy) to confirm the results. A confirmed infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
6. A negative result is not meaningful because of it is possible the antigens concentration in the stool sample is lower than the detection limit value. If the symptoms or situation still persist, a *Helicobacter pylori* determination should be carried out, on a sample from an enrichment culture or using an invasive technique.
7. Bloody stool samples can contain components that may cause non-specific reactions in the test. Every bloody stool sample whose result is positive should be followed up with other techniques of diagnosis to confirm the result.

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8. Mucous stool samples may cause non-specific reactions in the test- Every mucous stool sample whose result is positive should be followed up with other techniques of diagnosis to confirm the result.

EXPECTED VALUES

Studies have found that more than 90% of patients with duodenal ulcer and 80% of patients with gastric ulcer are infected with *Helicobacter pylori*. CerTest *H. pylori* has been compared with different methods: cultures, Urea Breath Test and Urease Test, demonstrating an overall accuracy of >92%.

PERFORMANCE CHARACTERISTICS

Analytical sensitivity (detection limit)

Detection limit range is: 0.78ng/mL – 0.09ng/mL of *H. pylori* recombinant outer membrane protein.

Clinical sensitivity and specificity

An evaluation, with faecal samples, was performed comparing the results obtained by an immunochromatographic test (CerTest *H. pylori*, CerTest) and qPCR technique (VIASURE *Helicobacter pylori* Real Time Detection kit, CerTest). The samples were taken from patients with the same as *Helicobacter pylori* infection symptoms arrived in Gastroenterology area in a Spanish hospital. The results were as follows:

		qPCR test: Viasure <i>Helicobacter pylori</i> Real Time Detection kit		
		+	-	Total
IC test: CerTest <i>H. pylori</i>	+	54	1	55
	-	1	60	61
	Total	55	61	116

CerTest <i>H. pylori</i> vs Viasure <i>Helicobacter pylori</i> Real Time Detection Kit		
	Mean Value	95% CI (confidence interval)
Sensitivity	98.2%	90.3 – 100.0%
Specificity	98.4%	91.2 – 100.0%
PPV	98.2%	90.3 - 100.0%
NPV	98.4%	91.2 –100.0%

The results showed a high sensitivity and specificity to detect *Helicobacter pylori* using CerTest *H. pylori*.

Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest *H. pylori*; no cross reactivity against gastrointestinal pathogens occasionally present in faeces:

<i>Campylobacter coli</i>	<i>Salmonella enteritidis</i>	<i>Shigella dysenteriae</i>
<i>Campylobacter jejuni</i>	<i>Salmonella paratyphi</i>	<i>Shigella flexneri</i>
<i>Clostridium difficile</i>	<i>Salmonella typhi</i>	<i>Shigella sonnei</i>
<i>Escherichia coli</i> O157:H7	<i>Salmonella typhimurium</i>	<i>Staphylococcus aureus</i>
<i>Listeria monocytogenes</i>	<i>Shigella boydii</i>	<i>Yersinia enterocolitica</i>

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