CerTest Rapid Test

CerTest
BIOTEC

CERTEST SARS-CoV-2 + Flu A + Flu B + RSV

ONE STEP SARS-CoV-2, *Influenza type A*,
Influenza type B and RSV Ag COMBO CARD
TEST

CE IVD

The test should be used by professional trained in IVD devices

These instructions for use apply to the following references:

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SV862001PC; SV862001PCC.

Initial part of the Instructions for use is common for all the references/

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Specific parts for each of the references are described in the corresponding annexes (see tables below)/

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KIT 20 CARD SARS-CoV-2 + FLU A + FLU B + RSV, QUAD CARD WITH POSITIVE CONTROL (SEE ANNEX 1) /

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KIT 20 CARD SARS-CoV-2 + FLU A + FLU B + RSV, QUAD CARD WITH POSITIVE CONTROL AND NEGATIVE CONTROL

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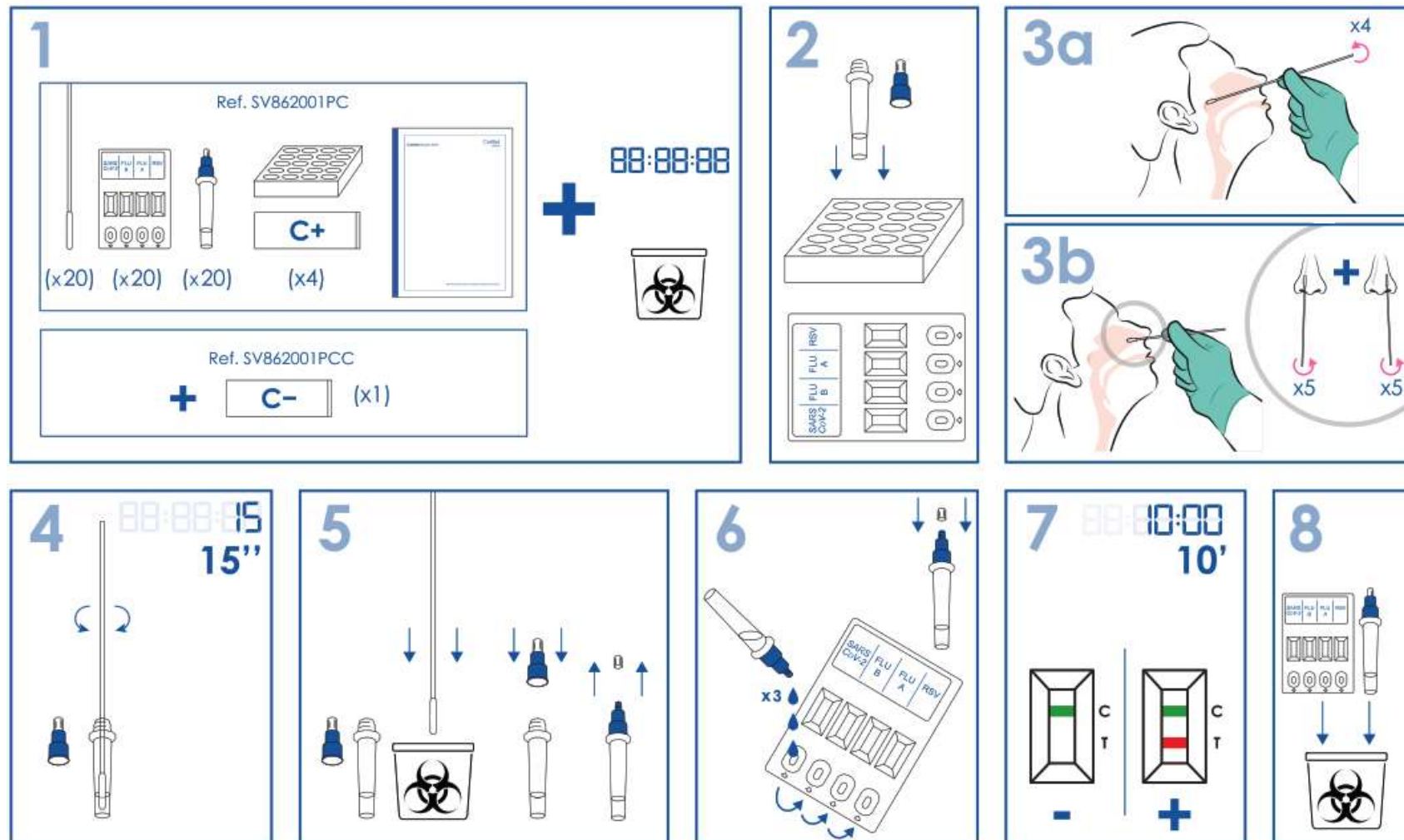
Content

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1.	Quick Guide/Guía rápida	3
ENGLISH		
2.	Intended use	7
3.	Summary and Explanation	7
4.	Test principle	8
5.	Storage and stability	9
6.	Precautions for users	9
7.	Specimen collection and preparation	10
8.	Recommendations	11
9.	Limitations of the test	12
10.	Expected values.....	12
11.	Performance characteristics.....	13
A.	Analytical sensitivity (detection limit)	13
B.	Clinical sensitivity and specificity.....	13
C.	Hook Effect	17
D.	Cross reactivity	17
E.	Interferences.....	19
F.	Repeatability and Reproducibility.....	20
ANNEX 1		21
I.	Materials.....	21
II.	Test procedures (samples)	21
III.	Interpretation of results	24
IV.	Quality Control	26
A. External Positive control		27
1.	CerTest SARS-CoV-2 Positive Control.....	27
1.1	CerTest SARS-CoV-2 Positive Control procedure	27
2.	CerTest <i>Influenza A</i> Positive Control	28
2.1	CerTest <i>Influenza A</i> Positive Control procedure.....	28
3.	CerTest <i>Influenza B</i> Positive Control.....	29
3.1	CerTest <i>Influenza B</i> Positive Control procedure	29
4.	CerTest RSV Positive Control.....	30
4.1	CerTest RSV Positive Control procedure	30
A.1	Interpretation positive control procedure	31
ANNEX 2.....		32
I.	Materials.....	32
II.	Test procedures (samples)	32

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ENGLISH

2. Intended use

CerTest SARS-CoV-2 + Flu A + Flu B + RSV one step combo card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of nucleoprotein (or N protein) antigen of SARS-CoV-2, *Influenza type A*, *Influenza B* and *Respiratory Syncytial Virus (RSV)* from nasopharyngeal swab or nasal samples from patients suspected of COVID-19 infection and/or *Influenza A* and/or *Influenza B* and/or RSV infection.

CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test offers a simple and a highly sensitive screening assay as an aid in the diagnosis of SARS-CoV-2 infection and/or *Influenza A* and/or *Influenza B* and/or RSV infection. This product is for professional *in vitro* diagnosis use only.

3. Summary and Explanation

An unseen pneumonia spread swiftly in Wuhan, China, in December 2019. Scientists sequenced and recognized a novel β -coronavirus.

The disease, caused by the Severe Acute Respiratory Syndrome Corona Virus 2 (SARS-CoV-2), is called as Coronavirus disease 2019 (COVID-19).

Coronaviruses (CoVs) are RNA, enveloped viruses, with a large genome (29.9 kb), amongst which β -CoVs and α -CoVs can infect mammals. Viruses belonging to the Coronaviridae family have the most abundant membrane protein (M) among other proteins, i.e. spike glycoprotein (S), nucleocapsid protein (N) and an envelope protein (E). Spike glycoprotein (S) is one of the targets of T cell response in the immune system. The S protein also promotes the binding of the virus envelope to the ACE2 receptor and the entry of the virus into the cell target. ACE2 receptors are present in cells of the arteries, veins, smooth muscles, small intestine, alveoli of the lungs, hair follicles, cardiac myofibroblasts, skin, brain, and kidney, thus SARS-CoV-2 could potentially infect these tissues.

Clinical forms of this disease comprise from mild to very severe symptoms: pneumonia, fever and respiratory symptoms are the most frequent. Other symptoms of the viral infection include a sore throat, aches, pains, and shortness of breath. In some cases, runny nose, nausea, and diarrhea also occur.

The outbreak of newly discovered coronavirus begins mainly via discharge from the nose or droplets of saliva, once an infected individual sneezes or coughs, and the estimated time for incubation is within 2 weeks. Due to the high infectious rate of SARS-CoV-2, detection of asymptomatic positive patients is probably one of the key points to controlling the outbreak.

Influenza is caused by a virus that attacks mainly the upper respiratory tract - the nose, throat and bronchi and rarely also the lungs. The infection usually lasts for about a week. It is characterized by sudden onset of high fever, myalgia, headache and severe malaise, non-productive cough, sore throat, and rhinitis. Most people recover within one to two weeks without requiring any medical treatment. In the very young, the elderly and people suffering from medical conditions such as lung diseases, diabetes, cancer, kidney or heart problems, Influenza poses a serious risk. In these people, the infection may lead to severe complications of underlying diseases, pneumonia and death.

The currently circulating influenza viruses that cause human disease are divided into two groups: A and B. Influenza A has 3 subtypes which are important for humans: A (H3N2), A (H1N1) and A (H5N1), of which the former is currently associated with most deaths. Influenza viruses are defined by 2 different protein components, known as antigens, on the surface of the virus. They are spike-like features called haemagglutinin (H) and neuraminidase (N) components.

Respiratory Syncytial Virus (RSV) is the most common cause of bronchiolitis and pneumonia among infants and children under 1 year of age. Illness begins most frequently with fever, runny nose, cough and sometimes wheezing. Severe lower respiratory tract disease may occur at any age, especially among the elderly or among those with compromised cardiac, pulmonary, or immune systems. RSV is spread from respiratory secretions through close contact with infected persons or contact with contaminated surfaces or objects.

4. Test principle

CerTest SARS-CoV-2 + Flu A + Flu B + RSV one step combo card is based on the principle of a qualitative immunochromatographic assay for the qualitative detection of SARS-CoV-2 and/or *Influenza* type A and/or *Influenza* type B and/or RSV from nasopharyngeal swab samples or nasal swab samples from patients suspected of COVID-19 infection and/or *Influenza A* infection and/or *Influenza* type B infection and/or RSV infection.

Strip A consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against SARS-CoV-2 and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-SARS-CoV-2) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

Strip B consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Influenza* type A and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-type A) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

Strip C consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Influenza* type B and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-*Influenza* type B) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

Strip D consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against RSV and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal

antibodies anti-RSV) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

If the sample is *SARS-CoV-2* positive, the antigens of the diluted sample react with the red-coloured conjugates complex (anti-*SARS-CoV-2* monoclonal antibodies-red polystyrene microspheres) in the strip A and if the sample is *Influenza A* positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-*Influenza* type A monoclonal antibodies-red polystyrene microspheres) in the strip B, which were previously pre-dried on the absorbent pad. If the sample is *Influenza* type B positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-*Influenza* type B monoclonal antibodies-red polystyrene microspheres) in the strip C and if the sample is *RSV* positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-*RSV* monoclonal antibodies-red polystyrene microspheres) in the strip D, which were previously pre-dried on the absorbent pad. The mixtures then move upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-*SARS-CoV-2* antibodies present on the membrane of strip A (test line), the anti-*Influenza* type A antibodies present on the membrane of strip B (test line), the anti-*Influenza* type B antibodies present on the membrane of strip C (test line) and the anti-*RSV* antibodies present on the membrane of strip D (test line) capture the coloured conjugate and a red line will be visible in the strips. These bands are used to interpret the result.

If the sample is negative, there is no *SARS-CoV-2*, *Influenza* type A, *Influenza* type B, and *RSV* antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with any red-coloured conjugate complex. The anti-*SARS-CoV-2*, the anti-*Influenza* type A, the anti-*Influenza* type B and the anti-*RSV* antibodies present on the membranes (test lines) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red lines will not appear.

Whether the sample is positive or not, in the four strips, the mixture continues to move across the membranes to the immobilized specific antibodies placed in the control lines. The anti-specific protein antibodies present on the four membranes will capture control green-conjugate complex and the four control lines will always appear. The presence of these green lines serves as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents.

5. Storage and stability

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

6. Precautions for users

- **For professional *in vitro* diagnostic use.**
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous (especially samples from patients suspected of *SARS-CoV-2* infection) and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.

- Use proper infection control practices when collecting and handling specimens. These practices should include, but are not limited to, personal protective equipment (PPE), such as a laboratory coat, surgical or appropriate mask, or face shield, disposable gloves, and eye protection. Follow local or national regulations regarding collecting and handling specimens.
- The tests (swabs, vials, devices) should be discarded in a proper biohazard container after testing. These containers should be discarded in accordance with local or national laws or regulations.
- Clean up spills thoroughly using an appropriate disinfectant.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provide in the kit are approved for use in the **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** test. Do not use any other commercial kit component.
- Sterile swabs provided in the kits should be only used for taking the nasopharyngeal or nasal sample collection. They cannot be reuse.
- Do not touch the head of the sterile swabs provided when opening their primary packaging to avoid contamination.
- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- All positive results should be processed following local or national laws and regulations.

7. Specimen collection and preparation

Samples should be processed as soon as possible after collection. Follow proper infection control practices. If this is not possible, the samples can be stored in the refrigerator (2-8 °C) for a maximum of 8 hours prior to testing. If the samples (only for nasopharyngeal samples) are preserved in validated transport media (VTM, UTM o Saline Buffer) could be preserved on it until 6 hours at room temperature or in the refrigerator (2-8°C).

Specimen preparation (see illustration):

Nasopharyngeal swab method:

1. Remove the sterile swab from its packing.
 2. Collect specimen with a sterile swab from one nostril.
 3. Insert the swab into the nostril to the nasopharynx, rotating against the wall (4 times) to ensure swab contains cells as well as mucus. Remove the swab from the nostril carefully.
 4. Consider repeating procedure using other nostril only if the protocol of the professional taking the sample required do it.
 5. Process the swab (sample) as soon as possible after collecting the specimen.
- Follow the test procedure (samples).

Nasopharyngeal Specimen collection

Direct nasopharyngeal swab is the preferred protocol.

Nasopharyngeal samples previously extracted by transport media:

Even while using direct nasopharyngeal swab is the preferred protocol, the device can be also used with samples from nasopharyngeal swabs previously diluted in transport media such as: VTM, UTM or Saline Buffer.

1. Use the minimum volume of transport media in order to avoid loss in sensitivity. Example: 1.0 mL. Using higher volumes will affect sensitivity of the system.
2. Dilute the extracted sample 1:1 in the provided sample diluent (DIL Respiratory, 0.7 mL/vial). Do not use the provided swabs for the dilution of these kind of samples in the sample diluent.

Follow immediately with the point 3 of test procedure (samples).

Nasal swab method:

The device can be also used with samples from nasal swabs.

1. Remove the sterile swab from its packing.
2. Collect specimen with a sterile swab from one nostril. Recommended blow your nose once using a tissue before collection of the specimen.

3. Insert the swab into the nostril to the nasopharynx (approx. 2cm), rotating against the nasal wall several times (5 times) to ensure swab contains cells as well as mucus. Remove the swab from the nostril carefully and repeat the procedure with the other nostril.

Nasal Specimen collection

4. Once the process for both nostrils is finished, process the swab (sample) as soon as possible.

Follow the test procedure (samples).

8. Recommendations

Recommendations World Health Organization: Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays- 11 September 2020.

- 1) To optimize performance, testing should be conducted by trained operators in strict accordance with the test procedure and within the first 5-7 days following the onset of symptoms.
- 2) Where possible, all positive samples giving positive results should be transported to laboratories with NAAT (Nucleic Acid Amplification Test) capability for confirmatory testing.
- 3) This test could be used to screen at-risk individuals and rapidly isolate positive cases in NAAT- confirmed COVID-19 outbreaks.
- 4) To monitor trends in disease incidence in communities.
- 5) For early detection and isolation of positive cases in health facilities, where there is widespread community transmission.
- 6) A negative result cannot completely exclude an active COVID-19 infection, repeat testing or preferably confirmatory testing should be performed (NAAT) whenever possible, particularly in symptomatic patients.
- 7) Even **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** test was not validated using samples from asymptomatic contacts of cases, asymptomatic cases have been demonstrated to have viral loads similar to symptomatic cases, so **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** could detect as positive.

9. Limitations of the test

- 1) The test must be carried out within 2 hours after opening the sealed bag.
- 2) **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** test should be used only with nasopharyngeal swab samples or nasal swabs samples. The use of swab specimens taken from other sites or the use of other samples such as saliva, sputum or urine has not been established. The quality of the test depends on the quality of the sample; proper nasopharyngeal or nasal specimens must be obtained.
- 3) The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
- 4) Positive results determine the presence of SARS-CoV-2 and/or *Influenza* type A and/or *Influenza* type B and/or RSV antigens. A diagnosis of SARS-CoV-2 and/or *Influenza* type A and/or *Influenza* type B infection and/or RSV infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
- 5) Positive results do not rule out co-infections with other pathogens.
- 6) Only VTM, UTM and Saline buffer transport media have been validated with **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** test. When using transport media, the test sensitivity can be reduced due to a higher dilution of sample. It is not recommended for Nasal samples.
- 7) A negative result is not meaningful because of it is possible the antigens concentration in the nasopharyngeal or nasal samples is lower than the detection limit value. If the symptoms or situation still persist, it is recommended that all negative results undergo confirmatory testing using other method and/or virus identification by cell culture, PCR or other technique.
- 8) **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV** has been used with samples from asymptomatic patients, detecting those with high viral loads.
- 9) To obtain accurate results, do not use bloody nasopharyngeal or nasal samples.
- 10) Very viscous nasal samples may cause non-specific reaction in the test.

10. Expected values

On January 30, 2020, the World Health Organization declared COVID-19 a public health emergency of international concern. Since the onset of the pandemic, SARS-CoV-2 has affected millions of people, causing thousands of victims worldwide.

Recent studies have shown that the primary viral infection found in symptomatic patients was identical to that seen in asymptomatic patients, suggesting that COVID-19 patients with asymptomatic infections are capable of transmitting SARS-CoV-2. During the incubation period, asymptomatic patients are also likely to transmit the virus to healthy people.

Although SARS-CoV-2 infection can affect people of any age, children and the elderly with asthma are in a higher risk category for the disease. Those with underlying health problems such as cancer, chronic respiratory diseases, cardiovascular diseases, and diabetes are also reported to have a higher chance of developing serious illnesses.

Yearly *Influenza* epidemics can seriously affect all age groups, but the highest risk of complications occur among children younger than age two, adults aged 65 or older and people of any age with certain medical

conditions, such as chronic heart, lung, kidney, liver, blood or metabolic diseases (such as diabetes) or weakened immune systems.

Influenza affects between 20% and 30% of the United States population annually. Three fourths of the infected individuals develop an acute respiratory illness and one third of these seek medical attention. In a typical year, more than 20,000 to 40,000 Americans die and more than 100,000 are hospitalized because of complications related to influenza.

11. Performance characteristics

A. Analytical sensitivity (detection limit)

Detection limit value (typical) of **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** (strip A: SARS-CoV-2) is 1.25×10^2 PFU/mL nCoV-2019 D614G(S).

Detection limit value (typical) of **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** (strip B: *Influenza A*) is 12.5 ng/mL of *Influenza A* recombinant nucleoprotein.

Detection limit value (typical) of **CerTest SARS-CoV-2 + Flu A+ Flu B +RSV** (strip C: *Influenza B*) is 50.0 ng/mL of *Influenza B* recombinant nucleoprotein.

Detection limit value (typical) of **CerTest SARS-CoV-2 + Flu A+ Flu B +RSV** (strip D: RSV) is 20.0 ng/mL of RSV recombinant protein.

B. Clinical sensitivity and specificity

Evaluations for SARS-CoV-2 + Flu A + Flu B+ RSV (strip A: SARS-CoV-2)

A multi-center evaluation with **nasopharyngeal samples** from people suspected of infection by SARS-CoV-2 virus was performed comparing the results obtained by an immunochromatographic test (**CerTest SARS-CoV-2 + Flu A+ Flu B + RSV**, CerTest (SARS-CoV-2 strip) vs qPCR technique.

The results were as follows:

IC test: CerTest SARS-CoV-2 + Flu A+ Flu B +RSV, (SARS-CoV-2 strip)	qPCR technique			Total
		+	-	
+		93	1	94
-		7	455	462
Total		100	456	556

CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV, (SARS-CoV-2 strip) vs qPCR technique		
	Mean Value	95% confidence interval
Sensitivity	93.0%	86.1–97.1%
Specificity	99.8%	98.8–100.0%
PPV	98.9%	94.2–100.0%
NPV	98.5%	96.9–99.4%

This multi-center evaluation (nasopharyngeal samples) with positive samples with Ct < 28 (*) showed the following results: sensitivity 95.1% (95% confidence interval: 88.0-98.7%) and specificity: 99.8% (95% confidence interval: 98.8-100.0%).

(*) Taking into account the recommendations for *Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays (11 September 2020)* from the WHO, the sensitivity of the test was calculated with nasopharyngeal samples with high viral load (high viral loads is expected in early symptomatic phases of the illness (with the first 5-7 days of illness) in the range of Ag-RDT test detection).

The results showed a high sensitivity and specificity to detect SARS-CoV-2 using **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV**.

An evaluation using **nasal samples** was performed comparing the results obtained by an immunochromatographic test (**CerTest SARS-CoV-2**, Certest) using nasal samples vs qPCR technique (using nasopharyngeal samples). The results were as follows:

		qPCR technique (nasopharyngeal samples)		
		+	-	Total
IC test: CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV, (SARS-CoV-2 strip) (nasal samples)	+	129	7	136
	-	27	827	854
	Total	156	834	990

CerTest SARS-CoV-2 + Flu A+ Flu B + RSV, (SARS-CoV-2 strip) vs qPCR technique		
	Mean Value	95% confidence interval
Sensitivity	82.7%	75.8 - 88.3%
Specificity	99.2%	98.3 - 99.7%
PPV	94.9%	89.7 - 97.9%
NPV	96.8%	95.4 - 97.9%

This evaluation (nasal samples for the evaluation with the rapid test) with positive samples with Ct < 28 showed the following results: sensitivity 96.9% (95% confidence interval: 91.2-99.4%) and specificity: 99.2% (95% confidence interval: 98.3-99.7%).

The results showed a high sensitivity and specificity to detect SARS-CoV-2 using **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV**.

Evaluations for SARS-CoV-2 + Flu A+ Flu B + RSV (strip B: Influenza A)

Results showed in tables below have been obtained from several evaluations developed using an immunochromatographic (**CerTest SARS-CoV-2 + Flu A+ Flu B +RSV**, CerTest (Influenza A)) and others commercial test kit.

1574 respiratory samples have been used for *Influenza A* evaluation (130 positive samples, 11 false positives, 1402 negatives samples and 31 false negatives). The results were as follows:

	Evaluation number	True Positives	False Negatives	True Negatives	False Positives
CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (Influenza A) vs XPECT FLU A&B (Remel)	I	7	0	85	1
CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (Influenza A) vs Instand EQAS program (March 2016)	II	101	26	955	3
CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (Influenza A) vs Binax NOW® Influenza A&B Card (Alere)	III	11	2	87	0
CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (Influenza A) vs SD Bioline, Influenza Ag A/B/A (H1N1) (SD Bioline)	IV	1	0	97	0
CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (Influenza A) vs MariPOC® respi (ArcDia International)	V	10	3	178	7
Total		130	31	1402	11

CerTest SARS-CoV-2 + Flu A+ Flu B + RSV, (Influenza A strip) vs other test used in evaluation (I, II, III, IV, V) for Influenza A		
	Mean Value	95% confidence interval
Sensitivity	80.7%	73.8 – 86.5%
Specificity	99.2%	98.6 – 99.6%
PPV	92.2%	86.5 – 96.0%
NPV	97.8%	96.9 – 98.5%

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip C: Influenza B)

Results showed in tables below have been obtained from several evaluations developed using an immunochromatographic (**CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza B)**, CerTest) and others commercial test kit.

1276 respiratory samples have been used for Influenza B evaluation (75 positive samples, 6 false positives, 1181 negatives samples and 14 false negatives). The results were as follows:

	Evaluation number	True Positives	False Negatives	True Negatives	False Positives
CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza B) vs XPECT FLU A&B (Remel)	I	7	2	83	1

	Evaluation number	True Positives	False Negatives	True Negatives	False Positives
CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza B) vs Instand EQAS program (March 2016)	II	63	12	1005	5
CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza B) vs SD Bioline, Influenza Ag A/B/A (H1N1) (SD Bioline)	III	5	0	93	0
	Total	75	14	1181	6

CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV, (Influenza B strip) vs other test used in evaluation (I, II, III, IV, V) for Influenza B		
	Mean Value	95% confidence interval
Sensitivity	84.3%	75.0 – 91.1%
Specificity	99.5%	98.9 – 99.8%
PPV	92.6%	84.6 – 97.2%
NPV	98.8%	98.0 – 99.4%

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip D: RSV)

The evaluation, was performed comparing the results obtained by an immunochromatographic test (**CerTest SARS-CoV-2 + Flu A + Flu B + RSV**, Certest) (RSV strip) and others commercially available immunochromatographic test BinaxNOW® RSV (Alere) The results were as follows:

IC test: CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV, (RSV strip)	IC test: BinaxNOW® RSV		
	+	-	Total
+	18	0	18
-	1	10	11
Total	19	10	29

CerTest SARS-CoV-2 + Flu A+ Flu B + RSV, (RSV strip) vs BinaxNOW® RSV		
	Mean Value	95% confidence interval
Sensitivity	94.7%	74.0–99.9%
Specificity	100.0%	69.2–100.0%
PPV	100.0%	81.5–100.0%
NPV	90.9%	58.7-99.8%

The results showed a high sensitivity and specificity to detect SARS-CoV-2, Influenza A, Influenza B and RSV using **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV**.

C. Hook Effect

CerTest SARS-CoV-2 + Flu A+ Flu B + RSV test does not show full inhibitory hook effect at concentration as high as 10⁵xLoD for Strip A: SARS-CoV-2, at concentrations higher than 10⁴xLoD for Strip B: *Influenza A*, at concentrations higher than 10⁴xLoD for Strip C: *Influenza B* and at concentrations higher than 10⁴xLoD for Strip D: RSV.

D. Cross reactivity

An evaluation was performed to determine the cross reactivity of **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV**; no cross reactivity against organism, pathogens, substances that could cause infections:

For Strip A. SARS-CoV-2:

Virus	<i>Adenovirus</i>	<i>Coronavirus Strain 229E</i>	<i>Coronavirus Strain OC43</i>	<i>Influenza B</i>	<i>Norovirus GII</i>	<i>Rhinovirus</i>
	<i>Astrovirus</i>	<i>Coronavirus Strain HKU1</i>	<i>Enterovirus</i>	<i>Metapneumovirus human (hMPV)</i>	<i>Parainfluenza virus</i>	<i>Rotavirus</i>
	<i>Bocavirus</i>	<i>Coronavirus Strain NL63</i>	<i>Influenza A</i>	<i>Norovirus GI</i>	<i>Respiratory Syncytial Virus</i>	
Bacteria	<i>Bordetella pertussis</i>	<i>Clostridium difficile Toxin B</i>	<i>Listeria Monocytogenes</i>	<i>Salmonella enteritidis</i>	<i>Shigella dysenteriae</i>	<i>Streptococcus pneumococcal</i>
	<i>Campylobacter jejuni</i>	<i>Escherichia coli 0:157</i>	<i>Mycobacterium tuberculosis</i>	<i>Salmonella paratyphi</i>	<i>Shigella flexneri</i>	<i>Streptococcus pneumoniae</i>
	<i>Chlamydia pneumoniae</i>	<i>Haemophilus influenzae</i>	<i>Mycoplasma pneumoniae</i>	<i>Salmonella typhi</i>	<i>Shigella sonnei</i>	<i>Streptococcus pyogenes</i>
	<i>Clostridium difficile antigen GDH</i>	<i>Helicobacter pylori</i>	<i>Mycoplasma tuberculosis</i>	<i>Salmonella typhimurium</i>	<i>Staphylococcus aureus</i>	<i>Yersinia enterocolitica O:3</i>
	<i>Clostridium difficile Toxin A</i>	<i>Legionella pneumophila</i>	<i>Pneumocystis jirovecii</i>	<i>Shigella boydii</i>	<i>Staphylococcus epidermis</i>	<i>Yersinia enterocolitica O:9</i>
Parasites	<i>Cryptosporidium Parvum</i>	<i>Giardia lamblia</i>	<i>Entamoeba histolytica</i>			
Others	Human calprotectin	Human haemoglobin	Human lactoferrin	Human transferrin	Pig haemoglobin	Bovine haemoglobin
	Pooled human nasal wash-representative of normal respiratory microbial flora					

CerTest SARS-CoV-2 + Flu A+ Flu B + RSV (SARS-CoV-2, strip A) showed some cross reaction with SARS and MERS.

For Strip B. Influenza A:

Virus	Adenovirus	Coronavirus Strain NL63	Coronavirus SARS-2 (SARS-CoV-2)	MERS-CoV Strain Florida/USA-2-Saudi Arabia.2014	Respiratory Syncytial Virus	
	Astrovirus	Coronavirus Strain OC43	Enterovirus	Norovirus GI	Rotavirus	
	Coronavirus Strain 229E	Coronavirus SARS-1 (SARS)	Influenza B	Norovirus GII		
Bacteria	Campylobacter jejuni	Escherichia coli 0:157	Helicobacter pylori	Salmonella paratyphi	Shigella dysenteriae	Streptococcus pyogenes
	Clostridium difficile antigen GDH	Escherichia coli 0:026	Legionella pneumophila	Salmonella typhi	Shigella flexneri	Yersinia enterocolitica O:3
	Clostridium difficile Toxin A	Escherichia coli 0:111	Listeria Monocytogenes	Salmonella typhimurium	Shigella sonnei	Yersinia enterocolitica O:9
	Clostridium difficile Toxin B	Haemophilus influenzae	Salmonella enteritidis	Shigella boydii	Streptococcus pneumococcal	
Parasites	Cryptosporidium Parvum	Giardia lamblia	Entamoeba histolytica			
Others	Human calprotectin	Human haemoglobin	Human lactoferrin	Human transferrin	Pig haemoglobin	Bovine haemoglobin

For Strip C. Influenza B:

Virus	Adenovirus	Coronavirus Strain NL63	Coronavirus SARS-2 (SARS-CoV-2)	MERS-CoV Strain Florida/USA-2-Saudi Arabia.2014	Respiratory Syncytial Virus	
	Astrovirus	Coronavirus Strain OC43	Enterovirus	Norovirus GI	Rotavirus	
	Coronavirus Strain 229E	Coronavirus SARS-1 (SARS)	Influenza A	Norovirus GII		
Bacteria	Campylobacter jejuni	Escherichia coli 0:157	Helicobacter pylori	Salmonella paratyphi	Shigella dysenteriae	Streptococcus pyogenes
	Clostridium difficile antigen GDH	Escherichia coli 0:026	Legionella pneumophila	Salmonella typhi	Shigella flexneri	Yersinia enterocolitica O:3
	Clostridium difficile Toxin A	Escherichia coli 0:111	Listeria Monocytogenes	Salmonella typhimurium	Shigella sonnei	Yersinia enterocolitica O:9
	Clostridium difficile Toxin B	Haemophilus influenzae	Salmonella enteritidis	Shigella boydii	Streptococcus pneumococcal	
Parasites	Cryptosporidium Parvum	Giardia lamblia	Entamoeba histolytica			
Others	Human calprotectin	Human haemoglobin	Human lactoferrin	Human transferrin	Pig haemoglobin	Bovine haemoglobin

For Strip D. RSV:

Virus	Adenovirus	Coronavirus Strain NL63	Coronavirus SARS-2 (SARS-CoV-2)	Influenza B	Norovirus GII	
	Astrovirus	Coronavirus Strain OC43	Enterovirus	MERS-CoV Strain Florida/USA-2-Saudi Arabia.2014	Rotavirus	
	Coronavirus Strain 229E	Coronavirus SARS-1 (SARS)	Influenza A	Norovirus GI		
Bacteria	Campylobacter jejuni	Escherichia coli 0:157	Helicobacter pylori	Salmonella paratyphi	Shigella dysenteriae	Streptococcus pyogenes
	Clostridium difficile antigen GDH	Escherichia coli 0:026	Legionella pneumophila	Salmonella typhi	Shigella flexneri	Yersinia enterocolitica O:3
	Clostridium difficile Toxin A	Escherichia coli 0:111	Listeria Monocytogenes	Salmonella typhimurium	Shigella sonnei	Yersinia enterocolitica O:9
	Clostridium difficile Toxin B	Haemophilus influenzae	Salmonella enteritidis	Shigella boydii	Streptococcus pneumococcal	
Parasites	Cryptosporidium Parvum	Giardia lamblia	Entamoeba histolytica			
Others	Human calprotectin	Human haemoglobin	Human lactoferrin	Human transferrin	Pig haemoglobin	Bovine haemoglobin

E. Interferences

An evaluation was performed to determine the possible interferences of **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV**; no interferences against the substances tested were detected:

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
Exogenous	Metronidazole	3.0 mg/mL	No interference	Loperamide hydrochloride (Fortasec)	0.15 mg/mL	No interference
	Ampicillin	3.0 mg/mL	No interference	Heparin (Hibor)	350.0 IU/mL	No interference
	Oseltamivir	3.0·10 ⁻² mg/mL	No interference	Almagato (Almax)	3.0 mg/mL	No interference
	Amantadine	0.3 mg/mL	No interference	Fosfamicin (Monurol)	3.0·10 ⁻³ mg/mL	No interference
	Ribavirin	3.0 mg/mL	No interference	Acetylcysteine (Fluimucil)	3.0·10 ⁻² mg/mL	No interference
	Codeine (Toseina)	0.2 mg/mL	No interference	Dexketoprofen trometamol (Enantyum)	0.3 mg/mL	No interference
	Benzocaine (Angileptol)	3.0·10 ⁻² mg/mL	No interference	Levofloxacin	3.0·10 ⁻² mg/mL	No interference
	Cloperastine (Flutox)	0.3 mg/mL	No interference	Ciprofloxacin	0.3 mg/mL	No interference
	Carbocisteine (Iniston mucolítico)	3.0·10 ⁻² mg/mL	No interference	Rifampicin (Rifaldin)	0.3 mg/mL	No interference
	Loratadine	0.3 mg/mL	No interference	Phenoxyethylpenicillin potassium	3.0 mg/mL	No interference
	Dexchlorpheniramine (Polaramine)	0.3 mg/mL	No interference	Ambroxol hydrochloride (Mucosan)	0.3 mg/mL	No interference
	Ebastine (Ebastel)	3.0 mg/mL	No interference	Macrogol 3350 (Movicol)	3.0 mg/mL	No interference
	Acetyl Salicylic (Adiro)	0.3 mg/mL	No interference	Lysine Carbocysteinate (Pectox)	3.0·10 ⁻² mg/mL	No interference
	Ibuprofen (Espidifen)	0.3 mg/mL	No interference	Hydroxyzine dihydrochloride	0.3 mg/mL	No interference
	Paracetamol (Dolocatil)	5.0 mg/mL	No interference	Lorazepam	3.0·10 ⁻³ mg/mL	No interference
	Metamizole (Nolotil)	5.0 mg/mL	No interference	Amoxicillin	3.0 mg/mL	No interference
	Prednisone	0.3 mg/mL	No interference	Mercaptopurine	0.3 mg/mL	No interference
	Omeprazole	2.0·10 ⁻³ mg/mL	No interference	Biotine	100.0 µg/mL	No interference
	Naso GEL	0.9 mg/mL	No interference	Sore Throat Phenol spray	0.51 mg/mL	No interference
	CVS Nasal Spray (Cromolyn)	4 mg/mL	No interference	Tobramycin	0.3 mg/mL	No interference
Afrin (Oxymetazoline)	0.05 mg/mL	No interference	Mupirocin	0.025 mg/mL	No interference	
CVS Nasal Drops (Phenylephrine)	10 mg/mL	No interference	Fluticasone Propionate	0.05 mg/mL	No interference	

	Potencial interfering substance	Concentration	Results	Potencial interfering substance	Concentration	Results
	ZICAM	0.1 mg/mL	No interference	Chloraseptic (Menthol/Benzocaine)	1.5 mg/mL	No interference
	Homeopathic	DIL 1/10	No interference			
Endogenous	Human haemoglobin	5.0 µg/mL	No interference	Human lactoferrin	5.0 µg/mL	No interference
	Human transferrin	0.5 µg/mL	No interference	Mucine	5.0 mg/mL	No interference
	Human calprotectin	5.0 µg/mL	No interference	Human blood	50.0 mg/mL	No interference
Others	HAMAs (Human anti-mouse antibodies)	10 µg/mL	No interference			

SARS-CoV-2 strip:

	Potencial interfering substance	Concentration (Ct)	Results	Potencial interfering substance	Concentration (Ct)	Results
Microbial interferences	STREP A	23.7/22.7/23.35	No interference	Influenza A	23.47/22.05/22.87	No interference
	ADENO	24.1/22.8/19.2	No interference	Influenza B	32.59/35.61/25.36	No interference
	RSV	29.28/27.78/38.82	No interference			

F. Repeatability and Reproducibility

Following the study of repeatability and reproducibility of **CerTest SARS-CoV-2+ Flu A+ Flu B+ RSV**, performed using different internal samples, negative and positive, no differences have been observed within the evaluations.

ANNEX 1

KIT 20 CARD SARS-CoV-2 + FLU A + FLU B + RSV. Catalogue reference: SV862001PC

Annex for the following reference:

PRODUCT	REFERENCE	UDI-DI
CerTest SARS-CoV-2 + Flu A + Flu B + RSV Quad Card (Ag 20 tests kit with swabs + 20 vials + PC)	SV862001PC	08435440214977

Table A 1. Reference for kit 20 card SARS-CoV-2 + Flu A + Flu B + RSV with positive control.

I. Materials

MATERIALS PROVIDED	MATERIALS REQUIRED BUT NO PROVIDED
<ul style="list-style-type: none"> - 20x CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV tests (combo card test) - 20x DIL Respiratory vials (0.7mL/vial). Sample extraction vials with diluent (sample and control diluent) - 20x Sterile swabs - 1x CerTest SARS-CoV-2 Positive Control - 1x CerTest Flu A Positive Control - 1x CerTest Flu B Positive Control - 1x CerTest RSV Positive Control - 1x Laboratory rack - 1x IFU: Instructions for use (including Quick guide use) 	<ul style="list-style-type: none"> - Disposable gloves (other personal protective equipment that will be considered necessary) - Timer - Vortex (optional)

II. Test procedures (samples)

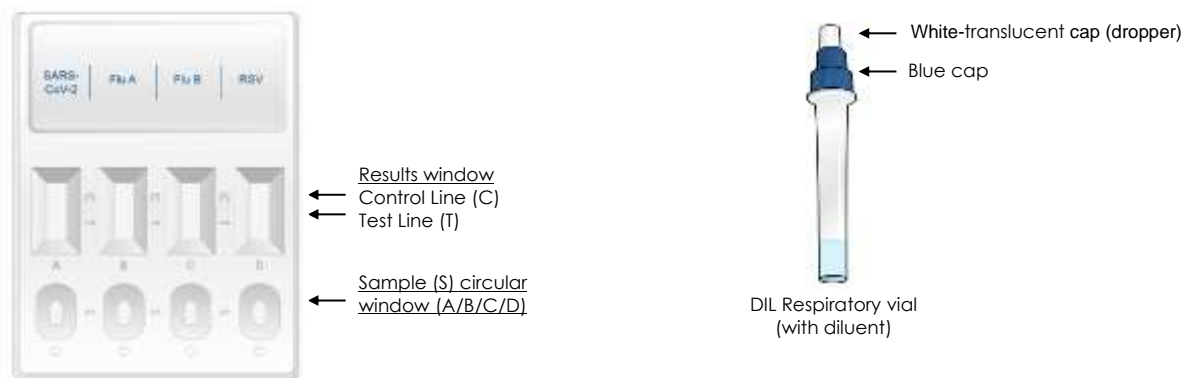
Follow proper infection control practices. Allow tests, samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Put the DIL Respiratory vials in the laboratory rack (identify properly samples from the different patients).
2. Open the blue cap of the DIL respiratory vial (prefilled with sample diluent) and immediately put the swab into the DIL Respiratory vial. Mix and homogenize the solution by rotating the swab forcefully

against the side of the DIL Respiratory vial to allow the solution to mix for 15 seconds. Best results are obtained when it is mixed to extract as much liquid as possible from the swab avoiding splashes and aerosols. Rotate the swab against the side of the DIL Respiratory vial as the swab is withdrawn to extract as much liquid as possible. Discard the swab. Close the extraction vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. The sample can be shaken using a vortex (this is optional) (1).

- Remove the **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV** combo card test from its sealed bag just before using it.
- Be sure that use a separate DIL Respiratory vial and test for each sample. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120 μ L) from the DIL Respiratory vial into the circular window marked with the letter A (3), 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter B (4), 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter C (5) and 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter D (6). After adding the drops into the sample window close the white-translucent cap (7).
- Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

The Sample diluent buffer (DIL Respiratory) has SARS-CoV-2 inactivation capability: 99.68% after 1 minute and 99.98% after 10 minutes.


(1) Put the swab into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the vial with the blue cap. Please be sure the caps are properly closed. The sample can be shaken (vortex).

(2) Open the white-translucent cap


SARS-CoV-2 Strip A-procedure

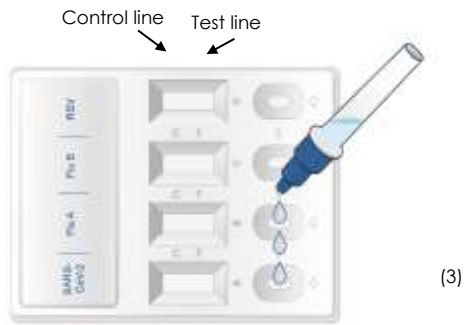(3) Add 3 full drops in A-circular window


Influenza type A Strip B-procedure

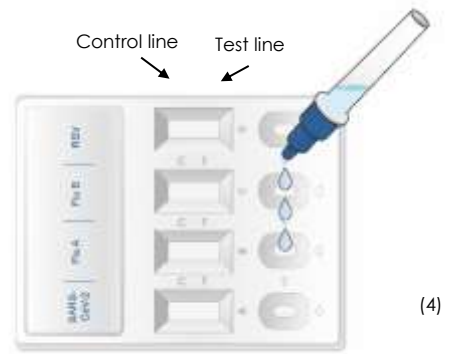(4) Add 3 full drops in B-circular window


Influenza type B Strip C-procedure

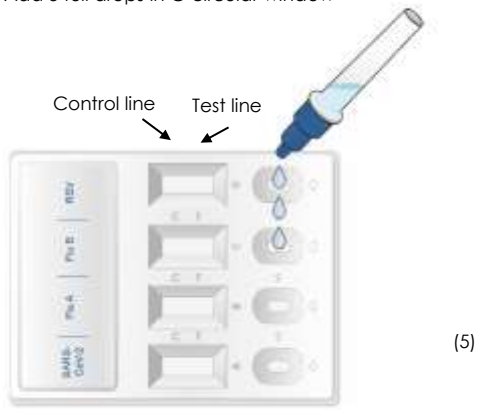(5) Add 3 full drops in C-circular window

RSV Strip D-procedure

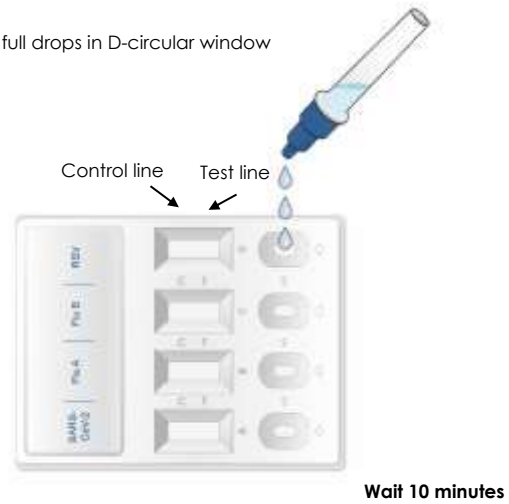(6) Add 3 full drops in D-circular window



Wait 10 minutes

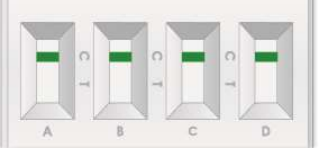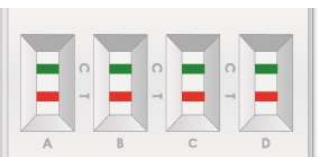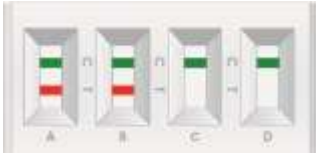
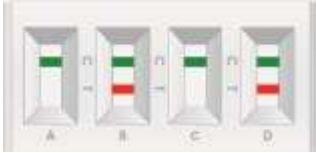
(7) Close the white-translucent cap.

III. Interpretation of results

(please refer to the illustration below)

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
1.	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE		There is no SARS-CoV-2, Influenza type A, Influenza type B, RSV antigens presence in sample.
	GREEN	GREEN	GREEN	GREEN		
2.	POSITIVE	POSITIVE	POSITIVE	POSITIVE		There is SARS-CoV-2, Influenza type A, Influenza type B and RSV antigens presence in the sample. The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
3.	POSITIVE	POSITIVE	POSITIVE	NEGATIVE		There is SARS-CoV-2, Influenza type A, Influenza type B antigens presence in sample. There is no RSV antigens presence in the sample.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN		
4.	POSITIVE	POSITIVE	NEGATIVE	POSITIVE		There is SARS-CoV-2, Influenza type A, RSV antigens presence in sample. There is no Influenza type B antigens presence in the sample.
	GREEN-RED	GREEN-RED	GREEN	GREEN-RED		
5.	POSITIVE	NEGATIVE	POSITIVE	POSITIVE		There is SARS-CoV-2, Influenza type B, RSV antigens presence in sample. There is no Influenza type A antigens presence in the sample.
	GREEN-RED	GREEN	GREEN-RED	GREEN-RED		
6.	NEGATIVE	POSITIVE	POSITIVE	POSITIVE		There is Influenza type A, Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2 antigens presence in the sample.
	GREEN	GREEN-RED	GREEN-RED	GREEN-RED		

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
7.	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE		There is SARS-CoV-2 and Influenza type A antigens presence in sample. There is no Influenza type B and RSV antigens presence in the sample.
	GREEN-RED	GREEN-RED	GREEN	GREEN		
8.	NEGATIVE	NEGATIVE	POSITIVE	POSITIVE		There is Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type A antigens presence in the sample.
	GREEN	GREEN	GREEN-RED	GREEN-RED		
9.	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE		There is influenza type A and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type B antigens presence in the sample.
	GREEN	GREEN-RED	GREEN	GREEN-RED		
10.	NEGATIVE	POSITIVE	POSITIVE	NEGATIVE		There is Influenza type A and Influenza type B antigens presence in sample. There is no SARS-CoV-2, RSV antigens presence in the sample.
	GREEN	GREEN-RED	GREEN-RED	GREEN		
11.	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE		There is SARS-CoV-2 and Influenza type B antigens presence in sample. There is no Influenza type A, RSV antigens presence in the sample.
	GREEN-RED	GREEN	GREEN-RED	GREEN		
12.	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE		There is SARS-CoV-2 and RSV antigens presence in sample. There is no Influenza type A, Influenza type B antigens presence in the sample.
	GREEN-RED	GREEN	GREEN	GREEN-RED		
13.	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE		There is SARS-CoV-2 antigens presence in sample. There is no Influenza type A, Influenza type B, RSV antigens presence in the sample.
	GREEN-RED	GREEN	GREEN	GREEN		

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
14.	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE		There is <i>Influenza</i> type A antigens presence in sample. There is no SARS-CoV-2, <i>Influenza</i> type B, RSV antigens presence in the sample.
15.	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE		There is <i>Influenza</i> type B antigens presence in sample. There is no SARS-CoV-2, <i>Influenza</i> type A, RSV antigens presence in the sample.
16.	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE		There is RSV antigens presence in sample. There is no SARS-CoV-2, <i>Influenza</i> type A, <i>Influenza</i> type B antigens presence in the sample.
17.	Any other result					Invalid result: either A, B, C or D, we recommend repeating the assay using the same sample with another test.

Whatever result detailed in the above-mentioned table (2 to 16) should be followed up with additional confirmatory diagnostic procedures.

Whatever results detailed between 2 and 6 are not very common. Dual simultaneous respiratory infections are more frequent than triple or quadruple respiratory virus infection.

INVALID: Total absence of any control coloured line (GREEN) regardless the appearance or not of the test lines (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control lines failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

Notes on the interpretation of results

The intensity of the red coloured bands in the test lines (T) in the results windows will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

IV. Quality Control

The **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** test presents five controls:

INTERNAL CONTROL: control line

Internal control: internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

EXTERNAL CONTROL, POSITIVE CONTROL SWABS:

External control: CerTest SARS-CoV-2 Positive Control, CerTest Influenza A, CerTest Influenza B and CerTest RSV Positive Control (positive control swab).

A. External Positive control**1. CerTest SARS-CoV-2 Positive Control**

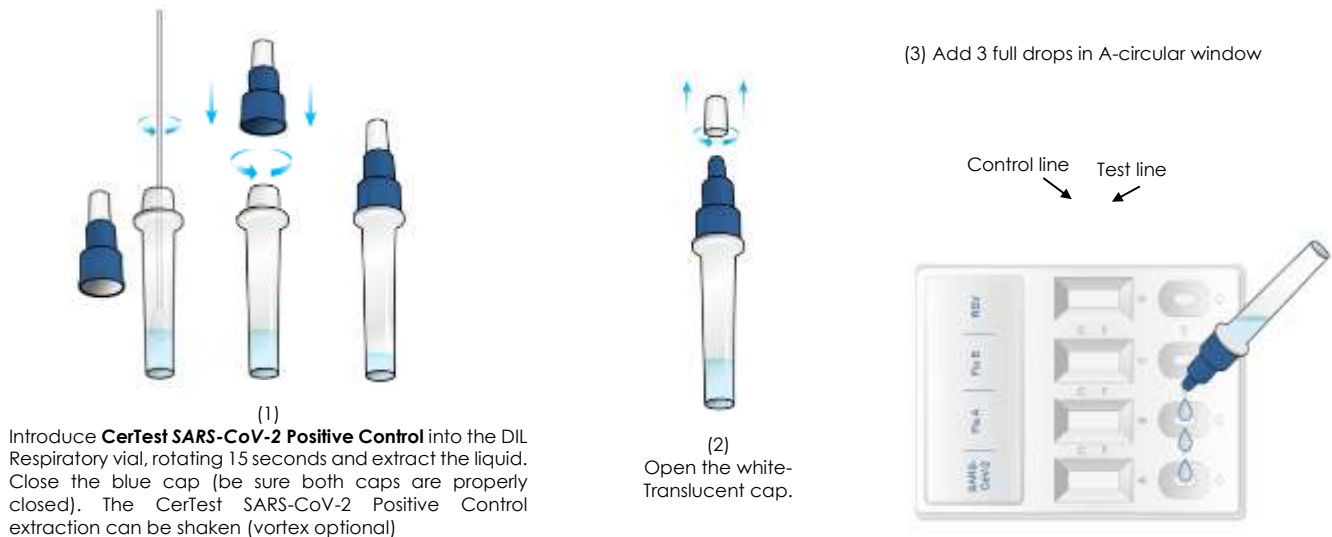
The CerTest SARS-CoV-2 Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use CerTest SARS-CoV-2 Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest SARS-CoV-2 Positive Control is stable through the expiration date printed on the sealed pouch. CerTest SARS-CoV-2 Positive Control must remain in the sealed pouch until use. Do not freeze.

1.1 CerTest SARS-CoV-2 Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest SARS-CoV-2 Positive Control into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest SARS-CoV-2 Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest SARS-CoV-2 Positive Control rotating CerTest SARS-CoV-2 Positive Control against the wall of the DIL Respiratory vial as CerTest SARS-CoV-2 Positive Control is withdrawn. Discard CerTest SARS-CoV-2 Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest SARS-CoV-2 Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter A (3). After adding the drops in the sample window, close the white-translucent cap.
5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

2. CerTest Influenza A Positive Control

The CerTest *Influenza A* Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use CerTest *Influenza A* Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest *Influenza A* Positive Control is stable through the expiration date printed on the sealed pouch. CerTest *Influenza A* Positive Control must remain in the sealed pouch until use. Do not freeze.

2.1 CerTest Influenza A Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest *Influenza A* Positive Control into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest *Influenza A* Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest *Influenza A* Positive Control rotating CerTest *Influenza A* Positive Control against the wall of the DIL Respiratory vial as CerTest *Influenza A* Positive Control is withdrawn. Discard CerTest *Influenza A* Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest *Influenza A* Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2+ Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter B (3). After adding the drops in the sample window, close the white-translucent cap.
5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

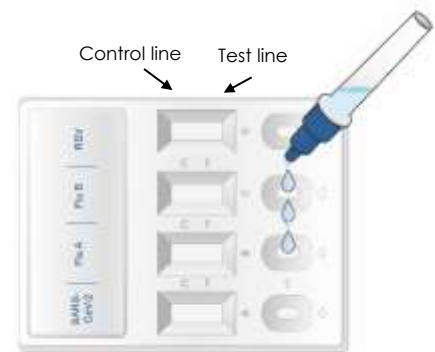
Introduce **CerTest Influenza A Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The **CerTest Influenza A Positive Control** extraction can be shaken (vortex optional)

(2)

Open the white-Translucent cap.

(3) Add 3 full drops in B-circular window

Wait 10 minutes

3. CerTest Influenza B Positive Control

The **CerTest Influenza B Positive Control** is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** triple card test. Use **CerTest Influenza B Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. **CerTest Influenza B Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest Influenza B Positive Control** must remain in the sealed pouch until use. Do not freeze.

3.1 CerTest Influenza B Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put **CerTest Influenza B Positive Control** into the DIL Respiratory vial.
2. Mix the solution by rotating **CerTest Influenza B Positive Control** forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from **CerTest Influenza B Positive Control** rotating **CerTest Influenza B Positive Control** against the wall of the DIL Respiratory vial as **CerTest Influenza B Positive Control** is withdrawn. Discard **CerTest Influenza B Positive Control**. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. **CerTest Influenza B Positive Control** extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter C (3). After adding the drops in the sample window, close the white-translucent cap.

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

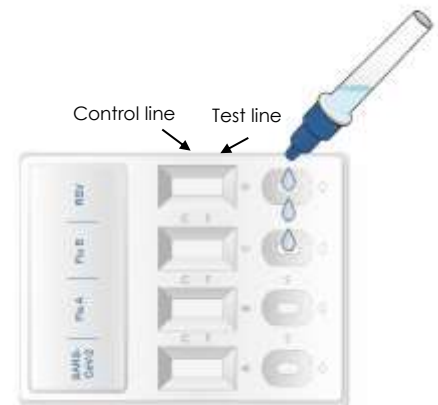
Introduce **CerTest Influenza B Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The CerTest Influenza B Positive Control extraction can be shaken (vortex optional)

(2)

Open the white-Translucent cap.

(3) Add 3 full drops in C-circular window

Wait 10 minutes

4. CerTest RSV Positive Control

The CerTest RSV Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** triple card test. Use CerTest RSV Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest RSV Positive Control is stable through the expiration date printed on the sealed pouch. CerTest RSV Positive Control must remain in the sealed pouch until use. Do not freeze.

4.1 CerTest RSV Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest RSV Positive Control (swab) into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest RSV Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest RSV Positive Control rotating the swab against the wall of the DIL Respiratory vial as CerTest RSV Positive Control is withdrawn. Discard CerTest RSV Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest RSV Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter C (3). After adding the drops in the sample window, close the white-translucent cap.

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

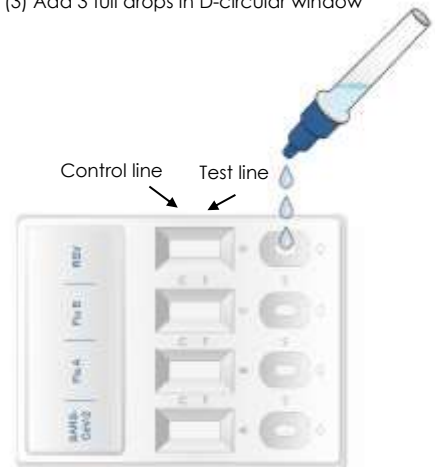
Introduce **CerTest RSV Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The CerTest RSV Positive Control extraction can be shaken (vortex optional)

(2)

Open the white-translucent cap.

(3) Add 3 full drops in D-circular window

Wait 10 minutes

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

A.1 Interpretation positive control procedure

	SARS-CoV-2 Positive Control	Influenza A Positive Control	Influenza B Positive Control	RSV Positive Control		Interpretation of the results
1.	POSITIVE					<p>SARS-CoV-2 (A) POSITIVE, Influenza A (B) POSITIVE, Influenza B (C) POSITIVE and RSV (D) POSITIVE: A GREEN line (control line (C)) and a RED line (test line (T)) in the four strips, in the results window, appear at the same time due to presence of antigens in the SARS-CoV-2 Positive Control swab in strip A, presence of antigens in Influenza A Positive Control swab in strip B, presence of antigens in Influenza B Positive Control swab in strip C and presence of antigens in RSV Positive Control swab in strip D.</p>
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
2.	INVALID	<p>INVALID: Total absence of the control coloured line (GREEN), regardless the appearance or not of the test line (RED), or only the control (C) coloured line (GREEN). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test and a new positive control swab.</p>				
	Any other result	<p>If the situation still persists, discontinue using the test kit and contact your local distributor.</p>				

ANNEX 2

KIT 20 CARD SARS-CoV-2 + FLU A + FLU B + RSV. Catalogue reference: SV862001PCC

Annex for the following reference:

PRODUCT	REFERENCE	UDI-DI
CerTest SARS-CoV-2 + Flu A + Flu B + RSV Quad Card (Ag 20 tests kit with swabs + 20 vials + PC+NC)	SV862001PCC	08435440218142

Table A 1. Reference for kit 20 card SARS-CoV-2 + Flu A + Flu B + RSV with positive control and negative control.

I. Materials

MATERIALS PROVIDED	MATERIALS REQUIRED BUT NO PROVIDED
<ul style="list-style-type: none"> - 20x CerTest SARS-CoV-2 + Flu A + Flu B + RSV tests (combo card test) - 20x DIL Respiratory vials (0.7mL/vial). Sample extraction vials with diluent (sample and control diluent) - 20x Sterile swabs - 1x CerTest SARS-CoV-2 Positive Control - 1x CerTest <i>Influenza A</i> Positive Control - 1x CerTest <i>Influenza B</i> Positive Control - 1x CerTest RSV Positive Control - 1x CerTest Negative Control - 1x Laboratory rack - 1x IFU: Instructions for use (including Quick guide use) 	<ul style="list-style-type: none"> - Disposable gloves (other personal protective equipment that will be considered necessary) - Timer - Vortex (optional)

II. Test procedures (samples)

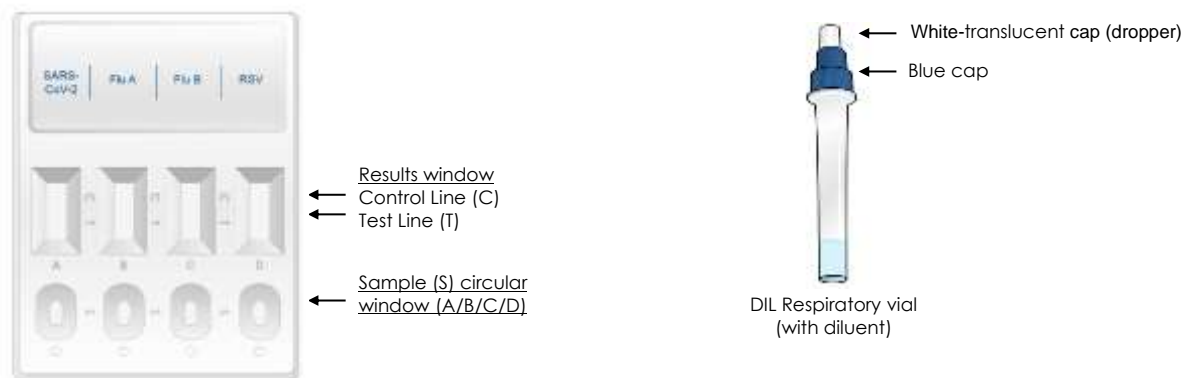
Follow proper infection control practices. Allow tests, samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Put the DIL Respiratory vials in the laboratory rack (identify properly samples from the different patients).
2. Open the blue cap of the DIL respiratory vial (prefilled with sample diluent) and immediately put the swab into the DIL Respiratory vial. Mix and homogenize the solution by rotating the swab forcefully

against the side of the DIL Respiratory vial to allow the solution to mix for 15 seconds. Best results are obtained when it is mixed to extract as much liquid as possible from the swab avoiding splashes and aerosols. Rotate the swab against the side of the DIL Respiratory vial as the swab is withdrawn to extract as much liquid as possible. Discard the swab. Close the extraction vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. The sample can be shaken using a vortex (this is optional) (1).

- Remove the **CerTest SARS-CoV-2 + Flu A+ Flu B+ RSV** combo card test from its sealed bag just before using it.
- Be sure that use a separate DIL Respiratory vial and test for each sample. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120 μ L) from the DIL Respiratory vial into the circular window marked with the letter A (3), 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter B (4), 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter C (5) and 3 full drops (approx. 100-120 μ L), using the DIL Respiratory vial, into the circular window marked with the letter D (6). After adding the drops into the sample window close the white-translucent cap (7).
- Read the results at 10 minutes.** Do not read the test result later than 10 minutes. Any result read after more than 10 minutes will be considered invalid.

The Sample diluent buffer (DIL Respiratory) has SARS-CoV-2 inactivation capability: 99.68% after 1 minute and 99.98% after 10 minutes.


(1) Put the swab into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the vial with the blue cap. Please be sure the caps are properly closed. The sample can be shaken (vortex).

(2) Open the white-translucent cap


SARS-CoV-2 Strip A-procedure

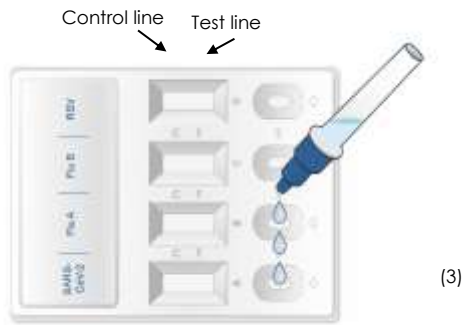(3) Add 3 full drops in A-circular window


Influenza type A Strip B-procedure

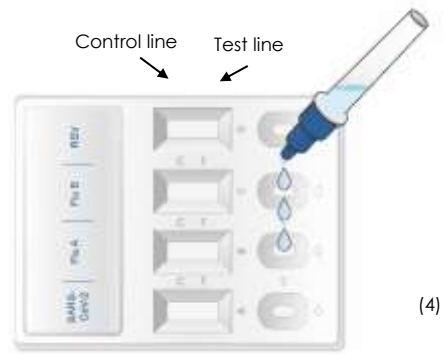(4) Add 3 full drops in B-circular window


Influenza type B Strip C-procedure

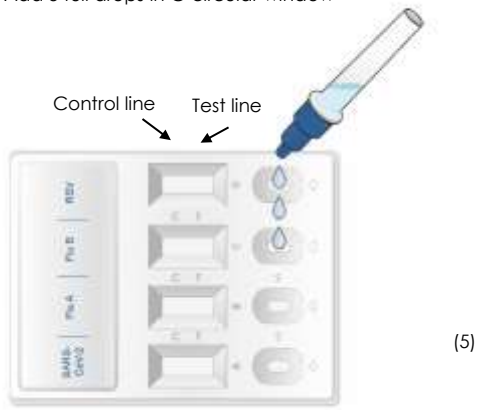(5) Add 3 full drops in C-circular window

RSV Strip D-procedure

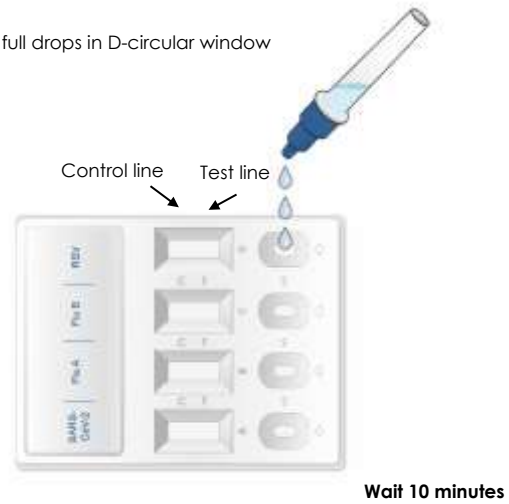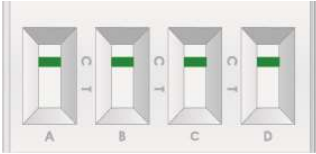
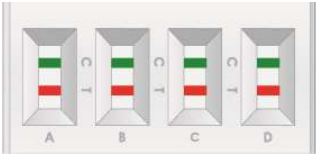
(6) Add 3 full drops in D-circular window

(7) Close the white-translucent cap.

III. Interpretation of results

(please refer to the illustration below)

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
1.	NEGATIVE	NEGATIVE	NEGATIVE	NEGATIVE		There is no SARS-CoV-2, Influenza type A, Influenza type B, RSV antigens presence in sample.
	GREEN	GREEN	GREEN	GREEN		
2.	POSITIVE	POSITIVE	POSITIVE	POSITIVE		There is SARS-CoV-2, Influenza type A, Influenza type B and RSV antigens presence in the sample. The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		
3.	POSITIVE	POSITIVE	POSITIVE	NEGATIVE		There is SARS-CoV-2, Influenza type A, Influenza type B antigens presence in sample. There is no RSV antigens presence in the sample.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN		
4.	POSITIVE	POSITIVE	NEGATIVE	POSITIVE		There is SARS-CoV-2, Influenza type A, RSV antigens presence in sample. There is no Influenza type B antigens presence in the sample.
	GREEN-RED	GREEN-RED	GREEN	GREEN-RED		
5.	POSITIVE	NEGATIVE	POSITIVE	POSITIVE		There is SARS-CoV-2, Influenza type B, RSV antigens presence in sample. There is no Influenza type A antigens presence in the sample.
	GREEN-RED	GREEN	GREEN-RED	GREEN-RED		
6.	NEGATIVE	POSITIVE	POSITIVE	NEGATIVE		There is Influenza type A, Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2 antigens presence in the sample.
	GREEN	GREEN-RED	GREEN-RED	GREEN-RED		

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
7.	POSITIVE	POSITIVE	NEGATIVE	NEGATIVE		There is SARS-CoV-2 and Influenza type A antigens presence in sample. There is no Influenza type B and RSV antigens presence in the sample.
8.	NEGATIVE	NEGATIVE	POSITIVE	POSITIVE		There is Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type A antigens presence in the sample.
9.	NEGATIVE	POSITIVE	NEGATIVE	POSITIVE		There is influenza type A and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type B antigens presence in the sample.
10.	NEGATIVE	POSITIVE	POSITIVE	NEGATIVE		There is Influenza type A and Influenza type B antigens presence in sample. There is no SARS-CoV-2, RSV antigens presence in the sample.
11.	POSITIVE	NEGATIVE	POSITIVE	NEGATIVE		There is SARS-CoV-2 and Influenza type B antigens presence in sample. There is no Influenza type A, RSV antigens presence in the sample.
12.	POSITIVE	NEGATIVE	NEGATIVE	POSITIVE		There is SARS-CoV-2 and RSV antigens presence in sample. There is no Influenza type A, Influenza type B antigens presence in the sample.
13.	POSITIVE	NEGATIVE	NEGATIVE	NEGATIVE		There is SARS-CoV-2 antigens presence in sample. There is no Influenza type A, Influenza type B, RSV antigens presence in the sample.

	SARS-CoV-2	Influenza A	Influenza B	RSV		Interpretation of the results
14.	NEGATIVE	POSITIVE	NEGATIVE	NEGATIVE		There is Influenza type A antigens presence in sample. There is no SARS-CoV-2, Influenza type B, RSV antigens presence in the sample.
15.	NEGATIVE	NEGATIVE	POSITIVE	NEGATIVE		There is Influenza type B antigens presence in sample. There is no SARS-CoV-2, Influenza type A, RSV antigens presence in the sample.
16.	NEGATIVE	NEGATIVE	NEGATIVE	POSITIVE		There is RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type A, Influenza type B antigens presence in the sample.
17.	Any other result					Invalid result: either A, B, C or D, we recommend repeating the assay using the same sample with another test.

Whatever result detailed in the above-mentioned table (2 to 16) should be followed up with additional confirmatory diagnostic procedures.

Whatever results detailed between 2 and 6 are not very common. Dual simultaneous respiratory infections are more frequent than triple or quadruple respiratory virus infection.

INVALID: Total absence of any control coloured line (GREEN) regardless the appearance or not of the test lines (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control lines failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

Notes on the interpretation of results

The intensity of the red coloured bands in the test lines (T) in the results windows will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

IV. Quality Control

The **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** test presents six controls:

INTERNAL CONTROL: control line

Internal control: internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

EXTERNAL CONTROL, POSITIVE AND NEGATIVE CONTROL SWABS:

External control: CerTest SARS-CoV-2 Positive Control, CerTest *Influenza A*, CerTest *Influenza B*, CerTest RSV Positive Control and CerTest Negative Control (negative control swab).

A. External Positive control.**1. CerTest SARS-CoV-2 Positive Control**

The CerTest SARS-CoV-2 Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use this CerTest SARS-CoV-2 Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest SARS-CoV-2 Positive Control is stable through the expiration date printed on the sealed pouch. CerTest SARS-CoV-2 Positive Control must remain in the sealed pouch until use. Do not freeze.

1. 1 CerTest SARS-CoV-2 Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest SARS-CoV-2 Positive Control into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest SARS-CoV-2 Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest SARS-CoV-2 Positive Control rotating CerTest SARS-CoV-2 Positive Control against the wall of the DIL Respiratory vial as CerTest SARS-CoV-2 Positive Control is withdrawn. Discard CerTest SARS-CoV-2 Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest SARS-CoV-2 Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter A (3). After adding the drops in the sample window, close the white-translucent cap.
5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

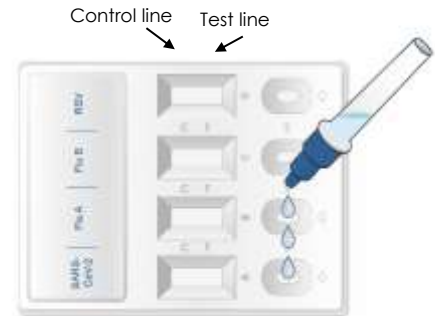
Introduce **CerTest SARS-CoV-2 Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The CerTest SARS-CoV-2 Positive Control extraction can be shaken (vortex optional)

(2)

Open the white-Translucent cap.

(3) Add 3 full drops in A-circular window

Wait 10 minutes

2. CerTest Influenza A Positive Control

The CerTest *Influenza A* Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use CerTest *Influenza A* Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest *Influenza A* Positive Control is stable through the expiration date printed on the sealed pouch. CerTest *Influenza A* Positive Control must remain in the sealed pouch until use. Do not freeze.

2.1 CerTest Influenza A Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest *Influenza A* Positive Control into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest *Influenza A* Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest *Influenza A* Positive Control rotating CerTest *Influenza A* Positive Control against the wall of the DIL Respiratory vial as CerTest *Influenza A* Positive Control is withdrawn. Discard CerTest *Influenza A* Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest *Influenza A* Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2+ Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter B (3). After adding the drops in the sample window, close the white-translucent cap.

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

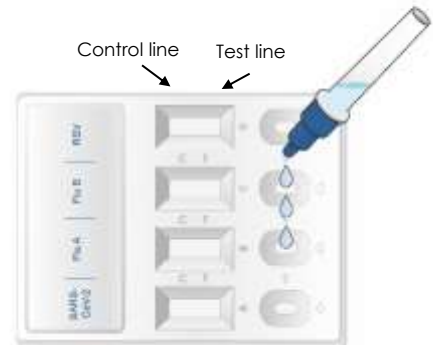
Introduce **CerTest Influenza A Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The **CerTest Influenza A Positive Control** extraction can be shaken (vortex optional)

(2)

Open the white-Translucent cap.

(3) Add 3 full drops in B-circular window

Wait 10 minutes

3. CerTest Influenza B Positive Control

The **CerTest Influenza B Positive Control** is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use **CerTest Influenza B Positive Control** to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. **CerTest Influenza B Positive Control** is stable through the expiration date printed on the sealed pouch. **CerTest Influenza B Positive Control** must remain in the sealed pouch until use. Do not freeze.

3.1 CerTest Influenza B Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put **CerTest Influenza B Positive Control** into the DIL Respiratory vial.
2. Mix the solution by rotating **CerTest Influenza B Positive Control** forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from **CerTest Influenza B Positive Control** rotating **CerTest Influenza B Positive Control** against the wall of the DIL Respiratory vial as **CerTest Influenza B Positive Control** is withdrawn. Discard **CerTest Influenza B Positive Control**. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. **CerTest Influenza B Positive Control** extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120 µL) from the DIL Respiratory vial into the

circular window marked with the letter C (3). After adding the drops in the sample window, close the white-translucent cap.

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

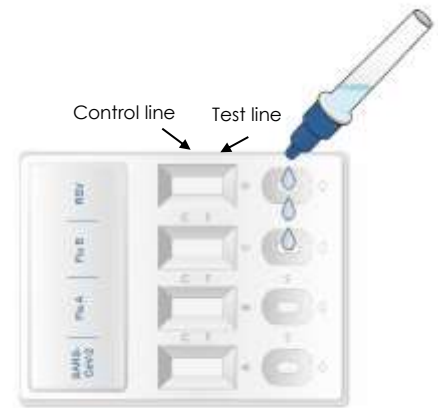
(1)
Introduce **CerTest Influenza B Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The CerTest Influenza B Positive Control extraction can be shaken (vortex optional)

(2)
Open the white-Translucent cap.

(3) Add 3 full drops in C-circular window

Wait 10 minutes

4. CerTest RSV Positive Control

The CerTest RSV Positive Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use CerTest RSV Positive Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest RSV Positive Control is stable through the expiration date printed on the sealed pouch. CerTest RSV Positive Control must remain in the sealed pouch until use. Do not freeze.

4.1 CerTest RSV Positive Control procedure

Allow tests and positive control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest RSV Positive Control into the DIL Respiratory vial.
2. Mix the solution by rotating CerTest RSV Positive Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest RSV Positive Control rotating CerTest RSV Positive Control against the wall of the DIL Respiratory vial as CerTest RSV Positive Control is withdrawn. Discard CerTest RSV Positive Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest RSV Positive Control extraction can be shaken using a vortex (this is optional) (1).
3. Remove the **CerTest SARS-CoV-2 + Flu A + Flu B + RSV** combo card test from its sealed bag just before using it.
4. Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the

circular window marked with the letter C (3). After adding the drops in the sample window, close the white-translucent cap.

5. **Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

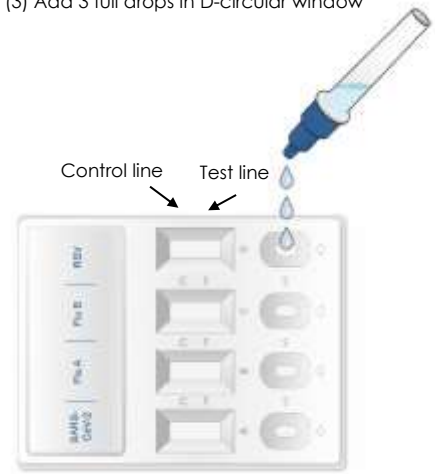
Introduce **CerTest RSV Positive Control** into the DIL Respiratory vial, rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The CerTest RSV Positive Control extraction can be shaken (vortex optional)

(2)

Open the white-Translucent cap.

(3) Add 3 full drops in D-circular window

Wait 10 minutes

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

A.1 Interpretation positive control procedure

	SARS-CoV-2 Positive Control	Influenza A Positive Control	Influenza B Positive Control	RSV Positive Control		Interpretation of the results
1.	POSITIVE					SARS-CoV-2 (A) POSITIVE , Influenza A (B) POSITIVE , Influenza B (C) POSITIVE and RSV (D) POSITIVE : A GREEN line (control line (C)) and a RED line (test line (T)) in the four strips, in the results window, appear at the same time due to presence of antigens in the SARS-CoV-2 Positive Control swab in strip A, presence of antigens in Influenza A Positive Control swab in strip B, presence of antigens in Influenza B Positive Control swab in strip C and presence of antigens in RSV Positive Control swab in strip D.
	GREEN-RED	GREEN-RED	GREEN-RED	GREEN-RED		

	SARS-CoV-2 Positive Control	Influenza A Positive Control	Influenza B Positive Control	RSV Positive Control	Interpretation of the results
2.	INVALID	INVALID: Total absence of the control coloured line (GREEN), regardless the appearance or not of the test line (RED), or only the control (C) coloured line (GREEN). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test and a new positive control swab. If the situation still persists, discontinue using the test kit and contact your local distributor.			
	Any other result				

B. External negative control. **CerTest negative control.**

The CerTest Negative Control is an external quality control for **CerTest SARS-CoV-2+ Flu A+ Flu B + RSV** combo card test. Use CerTest Negative Control to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. CerTest Negative Control is stable through the expiration date printed on the sealed pouch. CerTest Negative Control must remain in the sealed pouch until use. Do not freeze.

B. 1 Negative control procedure. CerTest Negative Control

Allow tests and negative control swab to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

- Open the blue cap of DIL Respiratory vial (prefilled with sample diluent) and immediately put CerTest Negative Control into the DIL Respiratory vial.
- Mix the solution by rotating CerTest Negative Control forcefully against the side of the DIL Respiratory vial at least 15 seconds. Extract as much liquid as possible from CerTest Negative Control rotating CerTest Negative Control against the wall of the DIL Respiratory vial as CerTest Negative Control is withdrawn. Discard CerTest Negative Control. Close the DIL Respiratory vial with its blue cap, please be sure that both caps (blue and white-translucent caps) are properly closed. CerTest Negative Control extraction can be shaken using a vortex (this is optional) (1).
- Remove the **CerTest SARS-CoV-2 + Flu A+ Flu B + RSV** combo card test from its sealed bag just before using it.
- Use a separate DIL Respiratory vial and test for each control. Open the white-translucent cap of the DIL Respiratory vial (2) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter A (3), dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter B (4), dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter C (5) and dispense 3 full drops (approx. 100-120µL) from the DIL Respiratory vial into the circular window marked with the letter D (6). After adding the drops in the sample window, close the white-translucent cap.
- Read the result at 10 minutes.** Do not read the test result later than 10 minutes.

(1)

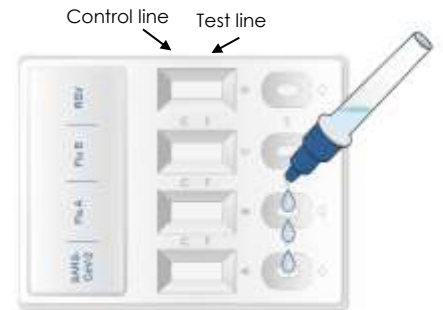
Introduce the **CerTest Negative Control** into the DL Respiratory vial rotating 15 seconds and extract the liquid. Close the blue cap (be sure both caps are properly closed). The negative control extraction can be shaken (vortex optional)


(2)

Open the white-Translucent cap.


(3) Add 3 full drops in the A-circular window

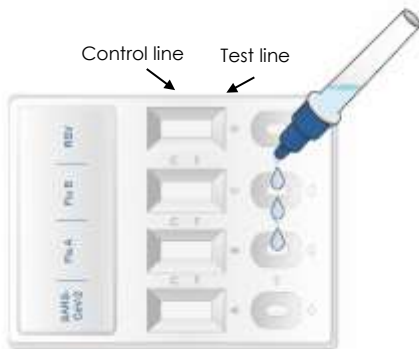
(4) Add 3 full drops in the B-circular window

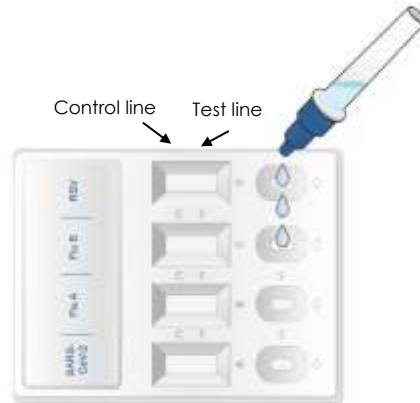(5) Add 3 full drops in the C-circular window

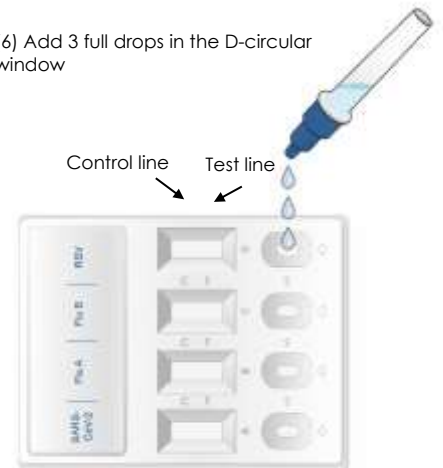(6) Add 3 full drops in the D-circular window


Wait 10 minutes

Negative controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

B.1.1. Interpretation negative control procedure

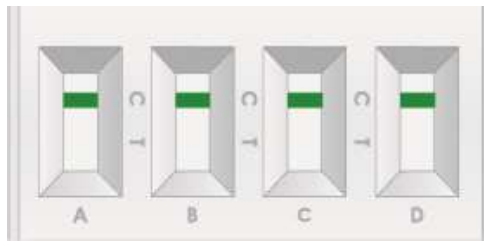Negative Control		Interpretation of the results
1.	VALID RESULT	<p>CerTest Negative Control, NEGATIVE: Only A GREEN line (control line (C)) appear across the results window during the test performance.</p>
	GREEN (only control band appear)	
2.	INVALID	<p>INVALID: Total absence of the control coloured line (GREEN), regardless the appearance or not of the test line (RED), or present of both lines (RED test and GREEN control lines). Incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the control with a new test device and a new CerTest Negative Control.</p> <p>If the situation still persists, discontinue using the test kit and contact your local distributor.</p>
	Any other result	

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