CERTEST

SARS-CoV-2 + Flu A + Flu B + RSV

ONE STEP SARS-CoV-2, *Influenza type A*, *Influenza type B* and RSV Ag COMBO CARD TEST

CERTEST BIOTEC S.L.

FOR INFORMATION PURPOSES ONLY

CERTEST SARS-CoV-2 + Flu A + Flu B + RSV

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ENGLISH

INTENDED USE

CerTest SARS-CoV-2 + Flu A + Flu B + RSV one step combo card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of nucleoprotein antigen of SARS-CoV-2, *Influenza* type A, *Influenza* type B and *Respiratory Syncytial Virus* from nasopharyngeal swab samples from patients suspected of COVID-19 infection and/or *Influenza* A and/or *Influenza* B and/or *Respiratory Syncytial Virus* (RSV) infection.

CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test offers a simple and highly sensitive screening assay as aid in the diagnosis of SARS-CoV-2 and/or *Influenza* type A and/or *Influenza* type B and/or RSV infection.

INTRODUCTION

An unseen pneumonia spread swiftly in Wuhan, China, in December 2019. Scientists sequenced and recognized a novel β-coronavirus.

The disease, caused by the Severe Acute Respiratory Syndrome Corona Virus 2 (SARS-CoV-2), is called as Coronavirus disease 2019 (COVID-19).

Coronaviruses (CoVs) are RNA, enveloped viruses, with a large genome (29.9 kb), amongst which β-CoVs and α-CoVs can infect mammals. Viruses belonging to the Coronaviridae family have the most abundant membrane protein (M) among other proteins, i.e. spike glycoprotein (S), nucleocapsid protein (N) and an envelope protein (E). Spike glycoprotein (S) is one of the targets of T cell response in the immune system. The S protein also promotes the binding of the virus envelope to the ACE2 receptor and the entry of the virus into the cell target. ACE2 receptors are present in cells of the arteries, veins, smooth muscles, small intestine, alveoli of the lungs, hair follicles, cardiac myofibroblasts, skin, brain, and kidney, thus SARS-CoV-2 could potentially infect these tissues.

Clinical forms of this disease comprise from mild to very severe symptoms: pneumonia, fever and respiratory symptoms are the most frequent. Other symptoms of the viral infection include a sore throat, aches, pains, and shortness of breath. In some cases, runny nose, nausea, and diarrhea also occur.

The outbreak of newly discovered coronavirus begins mainly via discharge from the nose or droplets of saliva, once an infected individual sneezes or coughs, and the estimated time for incubation is within 2 weeks. Due to the high infectious rate of SARS-CoV-2, detection of asymptomatic positive patients is probably one of the key points to controlling the outbreak.

Influenza is caused by a virus that attacks mainly the upper respiratory tract - the nose, throat and bronchi and rarely also the lungs. The infection usually lasts for about a week. It is characterized by sudden onset of high fever, myalgia, headache and severe malaise, non-productive cough, sore throat and rhinitis. Most people recover within one to two weeks without requiring any medical treatment. In the very young, the elderly and people suffering from medical conditions such as lung diseases, diabetes, cancer, kidney or heart problems, *Influenza* poses a serious risk. In these people, the infection may lead to severe complications of underlying diseases, pneumonia and death.

The currently circulating *Influenza* viruses that cause human disease are divided into two groups: A and B. *Influenza* A has 3 subtypes which are important for humans: A (H3N2), A (H1N1) and A (H5N1), of which the former is currently associated with most deaths. *Influenza* viruses are defined by 2 different protein components, known as antigens, on the surface of the virus. They are spike-like features called haemagglutinin (H) and neuraminidase (N) components.

Respiratory Syncytial Virus (RSV) is the most common cause of bronchiolitis and pneumonia among infants and children under 1 year of age. Illness begins most frequently with fever, runny nose, cough and sometimes wheezing. Severe lower respiratory tract disease may occur at any age, especially among the elderly or among those with compromised cardiac, pulmonary or immune systems. RSV is spread from respiratory secretions through close contact with infected persons or contact with contaminated surfaces or objects.

TEST PRINCIPLE

CerTest SARS-CoV-2 + Flu A + Flu B + RSV one step combo card test is a coloured chromatographic immunoassay for the simultaneous qualitative detection of nucleoprotein antigen of SARS-CoV-2, *Influenza* type A, *Influenza* type B and *Respiratory Syncytial Virus* from nasopharyngeal swab samples from patients suspected of COVID-19 infection and/or *Influenza* A and/or *Influenza* B and/or *Respiratory Syncytial Virus* (RSV) infection.

Strip A consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against SARS-CoV-2 and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-SARS-CoV-2) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

Strip B consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against *Influenza* type A and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is

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sprayed with test label solution (mouse monoclonal antibodies anti-Influenza type A) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming coloured conjugate complexes.

Strip C consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against Influenza type B and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-Influenza type B) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

Strip D consists of a nitrocellulose membrane pre-coated with mouse monoclonal antibodies on the test line (T), in the results window, against RSV and with rabbit polyclonal antibodies, on the control line (C), against a specific protein. The label/sample absorbent pad is sprayed with test label solution (mouse monoclonal antibodies anti-RSV) conjugated to red polystyrene latex and control label solution (specific binding protein) conjugated to green polystyrene latex, forming two coloured conjugate complexes.

If the sample is SARS-CoV-2 positive, the antigens of the diluted sample react with the red-coloured conjugates complex (anti-SARS-CoV-2 monoclonal antibodies-red polystyrene microspheres) in the strip A and if the sample is Influenza A positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-Influenza type A monoclonal antibodies-red polystyrene microspheres) in the strip B, which were previously pre-dried on the absorbent pad. If the sample is Influenza type B positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-Influenza type B monoclonal antibodies-red polystyrene microspheres) in the strip C and if the sample is RSV positive, the antigens of the diluted sample react with the red-coloured conjugated complex (anti-RSV monoclonal antibodies-red polystyrene microspheres) in the strip D, which were previously pre-dried on the absorbent pad. The mixtures then move upward on the membrane by capillary action. As the sample flows through the test membrane, the binding conjugate complexes migrate. The anti-SARS-CoV-2 antibodies present on the membrane of strip A (test line), the anti-Influenza type A antibodies present on the membrane of strip B (test line), the anti-Influenza type B antibodies present on the membrane of strip C (test line) and the anti-RSV antibodies present on the membrane of strip D (test line) capture the coloured conjugate and a red line will be visible in the strips. These bands are used to interpret the result.

If the sample is negative, there is no SARS-CoV-2, Influenza type A, Influenza type B, and RSV antigens presence and yet, the antigens may be present in a concentration lower than the detection limit value, for which the reaction will not take place with any red-coloured conjugate complex. The anti-SARS-CoV-2, the anti-Influenza type A, the anti-Influenza type B and the anti-RSV antibodies present on the membranes (test lines) will not capture the antigen-red-coloured conjugate complex (not formed), for which the red lines will not appear.

Whether the sample is positive or not, in the four strips, the mixture continues to move across the membranes to the immobilized specific antibodies placed in the control lines. The anti-specific protein antibodies present on the four membranes will capture control green-conjugate complex and the four control lines will always appear. The presence of these green lines serve as: 1) verification that sufficient volume is added, 2) that proper flow is obtained and 3) an internal control for the reagents

STORAGE AND STABILITY

Store as packaged in the sealed pouch at 2-30°C. The test is stable until the expiration date marked on its sealed pouch. The test must remain in the sealed pouch until use. Do not freeze.

PRECAUTIONS

- For professional *in vitro* diagnostic use.
- Do not use after expiration date.
- All the specimens should be considered potentially hazardous (especially samples from patients suspected of SARS-CoV-2 infection) and handled in the same manner as an infectious agent. A new test must be used for each sample to avoid contamination errors.
- Use proper infection control practices when collecting and handling specimens. These practices should include, but are not limited to, personal protective equipment (PPE), such as a laboratory coat, surgical or appropriate mask, or face shield, disposable gloves, and eye protection. Follow local or national regulations regarding collecting and handling specimens.
- The tests (swabs, testing tubes, pipettes, positive control swab and devices) should be discarded in a proper biohazard container after testing. These containers should be discarded in accordance with local or national laws or regulations.
- Clean up spills thoroughly using an appropriate disinfectant.
- Reagents contain preservatives. Avoid any contact with skin or mucous membrane. Consult safety data sheet, available on request.
- Components provided in the kit are approved for use in the CerTest SARS-CoV-2 + Flu A + Flu B + RSV test. Do not use any other commercial kit component.
- Sterile swabs provided in the kits should be only used for taking the nasopharyngeal sample collection. They cannot be reuse.
- Do not touch the head of the sterile swabs provided when opening their primary packaging to avoid contamination.

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- The presence of yellow lines in the results window (control line zone and test line zone), before using the test, is completely normal and does not imply failure of the test functionality.
- All positive results should be processed following local or national laws and regulations.

SPECIMEN COLLECTION AND PREPARATION

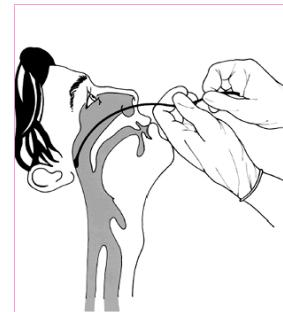
Samples should be process as soon as possible after collection. Follow proper infection control practices. If this is not possible, the samples can be stored in the refrigerator (2-8 °C) for a maximum of 8 hours prior to testing. If the samples are preserved in validated transport media (VTM, UTM o Saline Buffer) could be preserved on it until 6 hours at room temperature or in the refrigerator (2-8°C).

Specimen preparation (see illustration):

Nasopharyngeal swab method:

1. Remove the sterile swab from its packing.
2. Collect specimen with a sterile swab from one nostril.
3. Insert the swab into the nostril to the nasopharynx, rotating against the wall.
(to ensure swab contains cells as well as mucus). Remove the swab from the nostril carefully.
4. Consider repeating procedure using other nostril only if the protocol of the professional taking the sample required do it.
5. Process the swab as soon as possible after collecting the specimen.

Follow the test procedure (samples).

Specimen collection

Nasopharyngeal samples previously extracted by transport media:

Even while using direct nasopharyngeal swab is the preferred protocol, the device can be also used with samples from nasopharyngeal swabs previously diluted in transport media such as: VTM, UTM or Saline Buffer.

1. Use the minimum volume of transport media in order to avoid loss in sensitivity. Example: 1.0 mL. Using higher volumes will affect sensitivity of the system.
2. Dilute the extracted sample 1:1 in Reagent Respiratory (sample diluent) provided.

Follow immediately with the point 3 of test procedure (samples).

MATERIALS

MATERIALS PROVIDED

- CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card tests
- Reagent Respiratory (sample and controls diluent)
- Sterile Swabs
- Disposable pipettes
- Testing tubes
- Instructions for use
- Plastic caps
- Laboratory rack
- CerTest SARS-CoV-2 Positive Control
- CerTest Influenza A Positive Control
- CerTest Influenza B Positive Control
- CerTest RSV Positive Control

MATERIALS REQUIRED BUT NOT PROVIDED

- Disposable gloves (other personal protective equipment that will be considered necessary).
- Timer

TEST PROCEDURE (SAMPLES)

Follow proper infection control practices. Allow tests, samples to reach room temperature (15-30°C) prior to testing. Do not open pouches until the performance of the assay.

1. Put the tubes in the laboratory rack (identify properly samples from the different patients). Add 20 drops (approx. 670µL) (1) Reagent Respiratory and immediately put the swab into the tube.
2. Mix and homogenize the solution by rotating the swab forcefully against the side of the tube to allow the solution to mix for 15 seconds (2). Best results are obtained when it is mixed to extract as much liquid as possible from the swab avoiding splashes and aerosols. Rotate the swab against the side of the tube as the swab is withdrawn to extract as much liquid as possible. Discard the swab.
3. Remove the CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test from its sealed bag just before using it.

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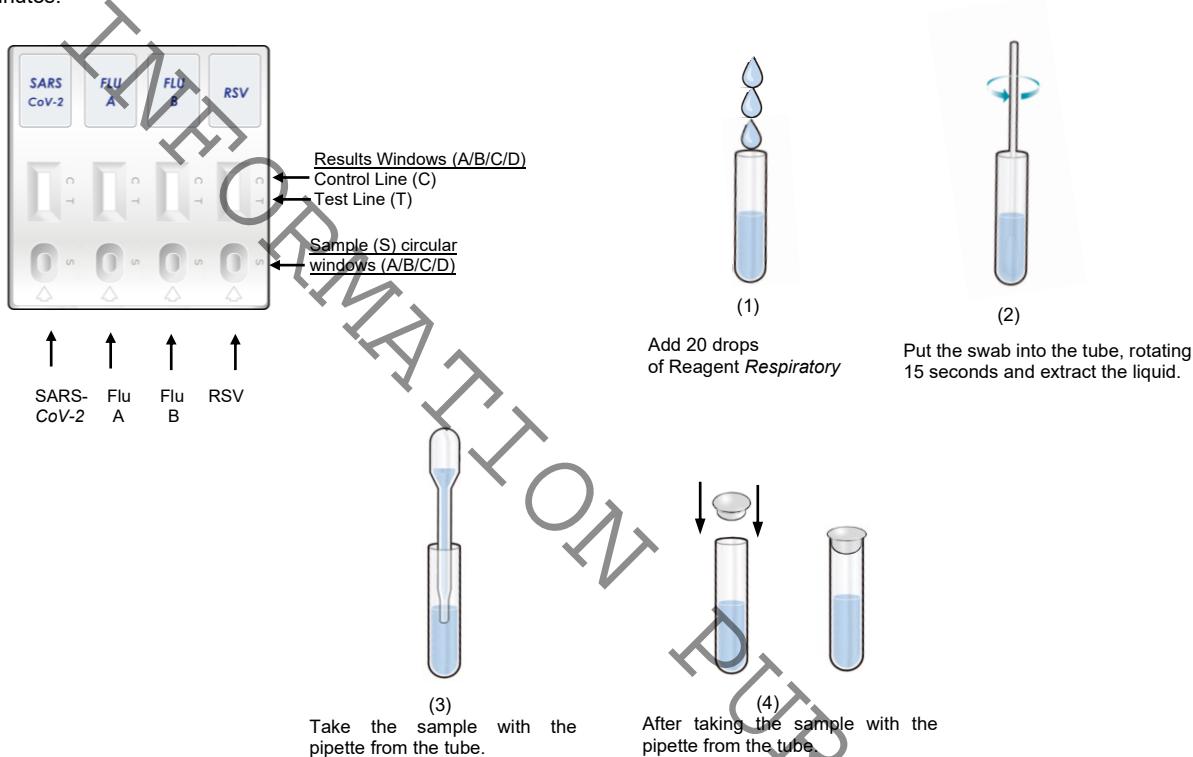
4. Take the sample from the tube with the pipette (3). Close the tube with the plastic cap (4) and dispense 3 drops (approx. 100-120µL) from the pipette, into the circular window marked with the letter A (5), 3 drops (approx. 100-120µL), using the same tube and pipette, into the circular window marked with the letter B (6), 3 drops (approx. 100-120µL), using the same tube and pipette, into the circular window marked with the letter C (7) and 3 drops (approx. 100-120µL), using the same tube and pipette, into the circular window marked with the letter D (7).

5. **Read the results at 10 minutes.** Do not read the test result later than 10 minutes.


If the test does not run (it means that liquid -sample+ diluent- does not rise through the strip -reaction zone or result window- due to the type of sample), mix the liquid added in the sample window (S) with the pipette. If it doesn't work (test does not run), dispense a drop of Reagent Respiratory until seeing the liquid running through the reaction zone.

The Sample diluent buffer (Reagent Respiratory) has SARS-CoV-2 inactivation capability: 99.68% after 1 minute and 99.98% after 10 minutes.

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
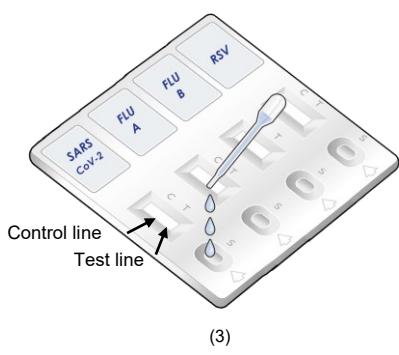
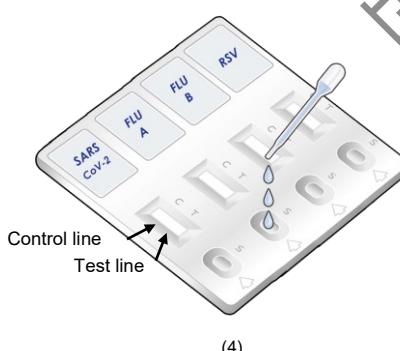
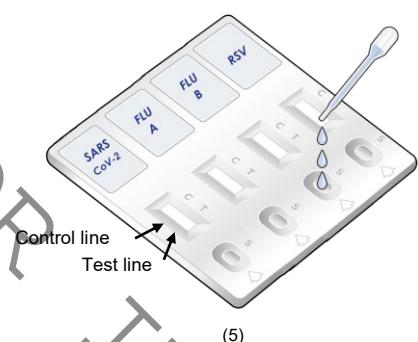
RSV Strip D procedure

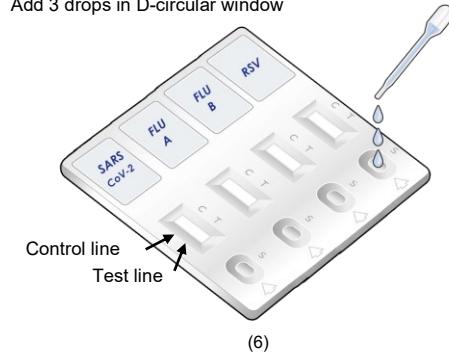Add 3 drops in D-circular window


INTERPRETATION OF THE RESULTS (please refer to the illustration below)

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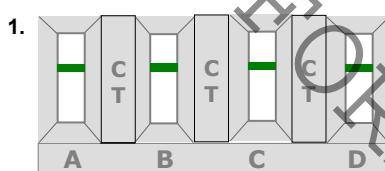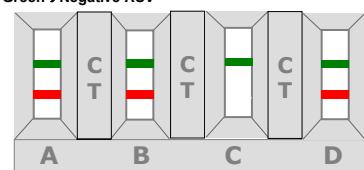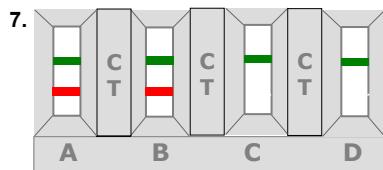
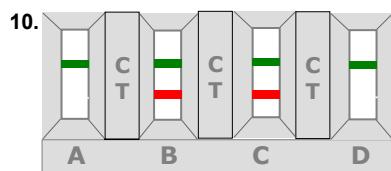
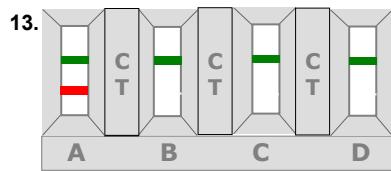
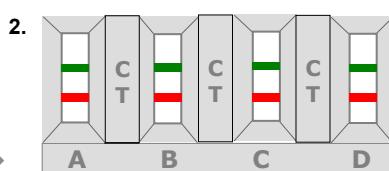
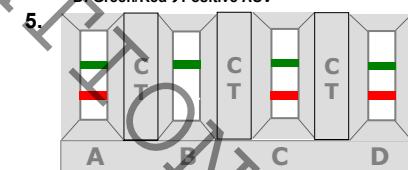
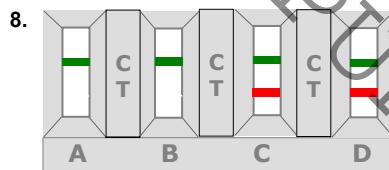
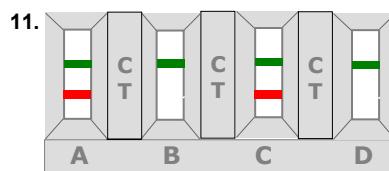
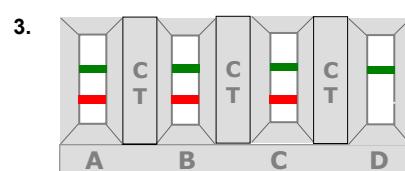
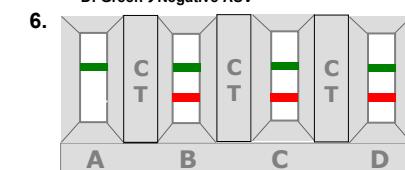
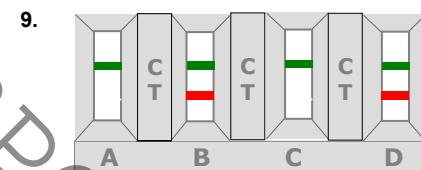
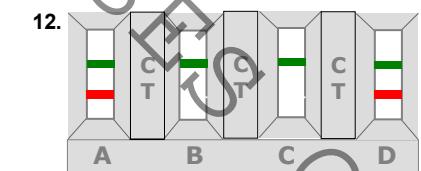
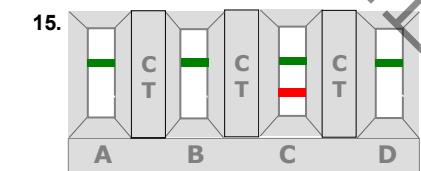
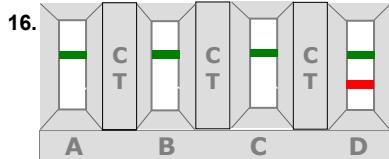
| | A (SARS-CoV-2) | B (Influenza A) | C (Influenza B) | D (RSV) | Interpretation of the results |
|-----|---------------------------|----------------------------|----------------------------|--------------------|--|
| 1. | - | - | - | - | There is no SARS-CoV-2, Influenza type A, Influenza type B, RSV antigens presence in sample. |
| | GREEN | GREEN | GREEN | GREEN | |
| 2. | + | + | + | + | There is SARS-CoV-2, Influenza type A, Influenza type B and RSV antigens presence in the sample. The presence of these lines (control green line and test red line), independent of their intensity should be considered as a positive result. |
| | GREEN-RED | GREEN-RED | GREEN-RED | GREEN-RED | |
| 3. | + | + | + | - | There is SARS-CoV-2, Influenza type A, Influenza type B antigens presence in sample. There is no RSV antigens presence in the sample. |
| | GREEN-RED | GREEN-RED | GREEN-RED | GREEN | |
| 4. | + | + | - | + | There is SARS-CoV-2, Influenza type A, RSV antigens presence in sample. There is no Influenza type B antigens presence in the sample. |
| | GREEN-RED | GREEN-RED | GREEN | GREEN-RED | |
| 5. | + | - | + | + | There is SARS-CoV-2, Influenza type B, RSV antigens presence in sample. There is no Influenza type A antigens presence in the sample. |
| | GREEN-RED | GREEN | GREEN-RED | GREEN-RED | |
| 6. | - | + | + | + | There is Influenza type A, Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2 antigens presence in the sample. |
| | GREEN | GREEN-RED | GREEN-RED | GREEN-RED | |
| 7. | + | + | - | - | There is SARS-CoV-2 and Influenza type A antigens presence in sample. There is no Influenza type B and RSV antigens presence in the sample. |
| | GREEN-RED | GREEN-RED | GREEN | GREEN | |
| 8. | - | - | + | + | There is Influenza type B and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type A antigens presence in the sample. |
| | GREEN | GREEN | GREEN-RED | GREEN-RED | |
| 9. | - | + | - | + | There is influenza type A and RSV antigens presence in sample. There is no SARS-CoV-2, Influenza type B antigens presence in the sample. |
| | GREEN | GREEN-RED | GREEN | GREEN-RED | |
| 10. | - | + | + | - | There is Influenza type A and Influenza type B antigens presence in sample. There is no SARS-CoV-2, RSV antigens presence in the sample. |
| | GREEN | GREEN-RED | GREEN-RED | GREEN | |
| 11. | + | - | + | - | There is SARS-CoV-2 and Influenza type B antigens presence in sample. There is no Influenza type A, RSV antigens presence in the sample. |
| | GREEN-RED | GREEN | GREEN-RED | GREEN | |
| 12. | + | - | - | + | There is SARS-CoV-2 and RSV antigens presence in sample. There is no Influenza type A, Influenza type B antigens presence in the sample. |
| | GREEN-RED | GREEN | GREEN | GREEN-RED | |
| 13. | + | - | - | - | There is SARS-CoV-2 antigens presence in sample. There is no Influenza type A, Influenza type B, RSV antigens presence in the sample. |
| | GREEN-RED | GREEN | GREEN | GREEN | |
| 14. | - | + | - | - | There is Influenza type A antigens presence in sample. There is no SARS-CoV-2, Influenza type B, RSV antigens presence in the sample. |
| | GREEN | GREEN-RED | GREEN | GREEN | |
| 15. | - | - | + | - | There is Influenza type B antigens presence in sample. There is no SARS-CoV-2, Influenza type A, RSV antigens presence in the sample. |
| | GREEN | GREEN | GREEN-RED | GREEN | |
| 16. | - | - | - | + | There is RSV antigens presence in sample. There is no SARS-CoV-2, |

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| | GREEN | GREEN | GREEN | GREEN-RED | Influenza type A, Influenza type B antigens presence in the sample. |
|-----|------------------|-------|-------|-----------|--|
| 17. | Any other result | | | | Invalid result: either A, B, C or D, we recommend repeating the assay using the same sample with another test. |

Whatever result detailed in the above-mentioned table (2 to 16) should be followed up with additional confirmatory diagnostic procedures.

Whatever results detailed between 2 and 6 are not very common. Dual simultaneous respiratory infections are more frequent than triple or quadruple respiratory virus infection.

INVALID: Total absence of control coloured lines (GREEN) regardless the appearance or not of the test lines (RED). Insufficient specimen volume, incorrect procedural techniques or deterioration of the reagents are mostly the main reasons for control lines failure. Review the procedure and repeat the assay with a new test. If the symptoms or situation still persists, discontinue using the test kit and contact your local distributor.

NOTES ON THE INTERPRETATION OF RESULTS

The intensity of the red coloured bands in the test lines (T) in the results windows will vary depending on the concentration of antigens present in the specimen. However, neither the quantitative value nor the rate of increase in antigens can be determined by this qualitative test.

QUALITY CONTROL

The CerTest SARS-CoV-2 + Flu A + Flu B + RSV test presents two controls:

Internal control: internal procedural control is included in the test. A green line appearing in the control line (C) in the results window is an internal control, which confirms sufficient specimen volume and correct procedural technique.

External control: Positive control swabs.

EXTERNAL CONTROL

The CerTest SARS-CoV-2 Positive Control is an external quality control for CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test. Only for the strip A (SARS-CoV-2).

The CerTest Influenza A Positive Control is an external quality control for CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test. Only for the strip B (Influenza A).

The CerTest Influenza B Positive Control is an external quality control for CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test. Only for the strip C (Influenza B).

The CerTest RSV Positive Control is an external quality control for CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test. Only for the strip D (RSV).

Use those control swabs to check that the extraction reagents and the test are working properly.

Store as packaged in the sealed pouch at 2-30°C. The positive control swab is stable through the expiration date printed on the sealed pouch. The positive control swab must remain in the sealed pouch until use. Do not freeze.

POSITIVE CONTROL PROCEDURE

Allow tests and positive control swabs to reach room temperature (15-30°C) prior to testing. Do not open the package until the performance of the assay.

1. Remove the CerTest SARS-CoV-2 + Flu A + Flu B + RSV combo card test from its sealed bag just before using.
2. The Reagent Respiratory is the same for both positive control swabs and for the samples. Use a separate pipette and testing tube for each control.

CerTest SARS-CoV-2 Positive Control swab procedure

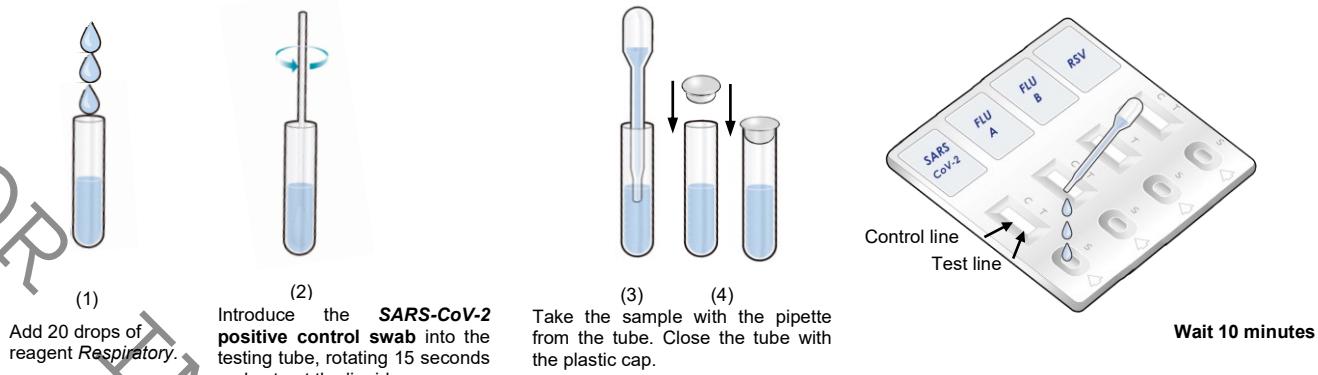
3. Add 20 drops (approx. 670µL) (1) of Reagent Respiratory and immediately introduce the SARS-CoV-2 positive control swab into the testing tube.
4. Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 15 seconds. Extract as much liquid as possible from the swab rotating the swab against the side of the tube as the swab is withdrawn (2). Discard the swab.
5. Use a separate pipette and test for each control. Take the sample from the tube with the pipette (3). Close the tube with the plastic cap (4) and dispense 3 drops (approx. 100-120µL) from the pipette into the circular window marked with the letter A (5).
6. Read the result at 10 minutes. Do not read the test result later than 10 minutes.

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

CerTest Influenza A Positive Control swab procedure

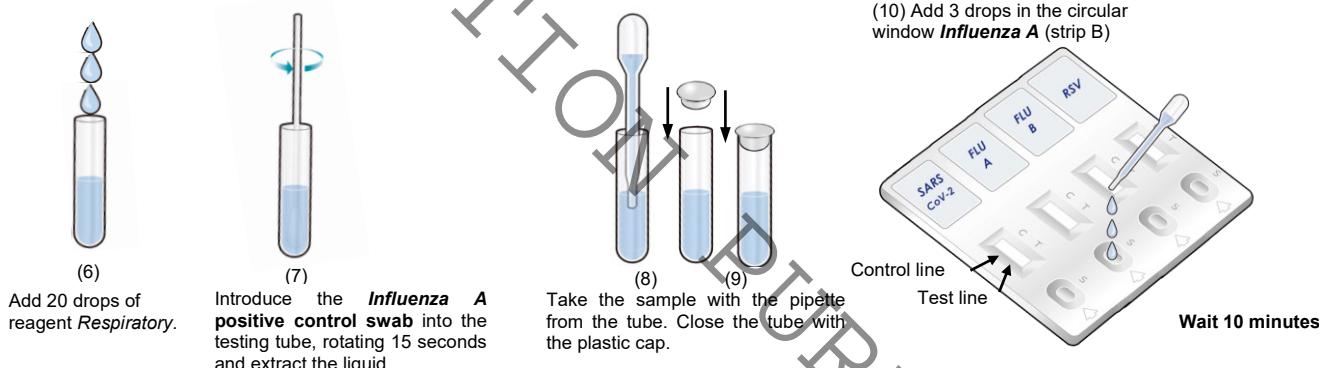- Take the Reagent Respiratory (the same used in the other Positive Control procedure and samples procedure), add 20 drops (approx. 670µL) (6) of Reagent Respiratory into a different testing tube and immediately introduce the **Influenza A Positive Control swab** into the testing tube.
- Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 15 seconds. Extract as much liquid as possible from the swab rotating the swab against the side of the tube as the swab is withdrawn (7). Discard the swab.
- Use a separate pipette and test for each control. Take the sample from the tube with the pipette (8). Close the tube with the plastic cap (9) and dispense 3 drops (approx. 100-120µL) from the pipette into the **circular window marked with the letter B** (10).
- Read the result at 10 minutes. Do not read the test result later than 10 minutes.

CerTest Influenza B Positive Control swab procedure

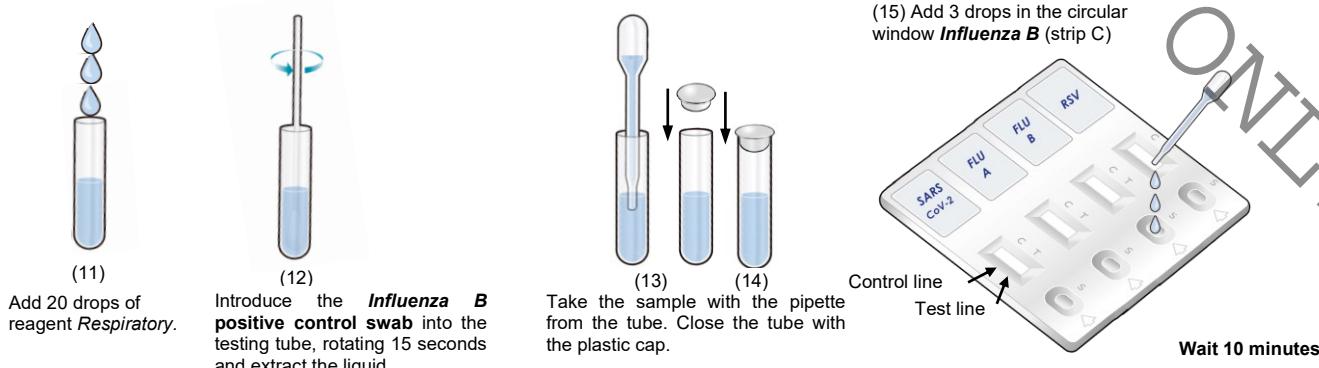- Take the Reagent Respiratory (the same used in the other Positive Control procedure and samples procedure), add 20 drops (approx. 670µL) (11) of Reagent Respiratory into a different testing tube and immediately introduce the **Influenza B Positive Control swab** into the testing tube.
- Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 15 seconds. Extract as much liquid as possible from the swab rotating the swab against the side of the tube as the swab is withdrawn (12). Discard the swab.
- Use a separate pipette and test for each control. Take the sample from the tube with the pipette (13). Close the tube with the plastic cap (14) and dispense 3 drops (approx. 100-120µL) from the pipette into the **circular window marked with the letter C** (15).
- Read the result at 10 minutes. Do not read the test result later than 10 minutes.

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
CerTest RSV Positive Control swab procedure

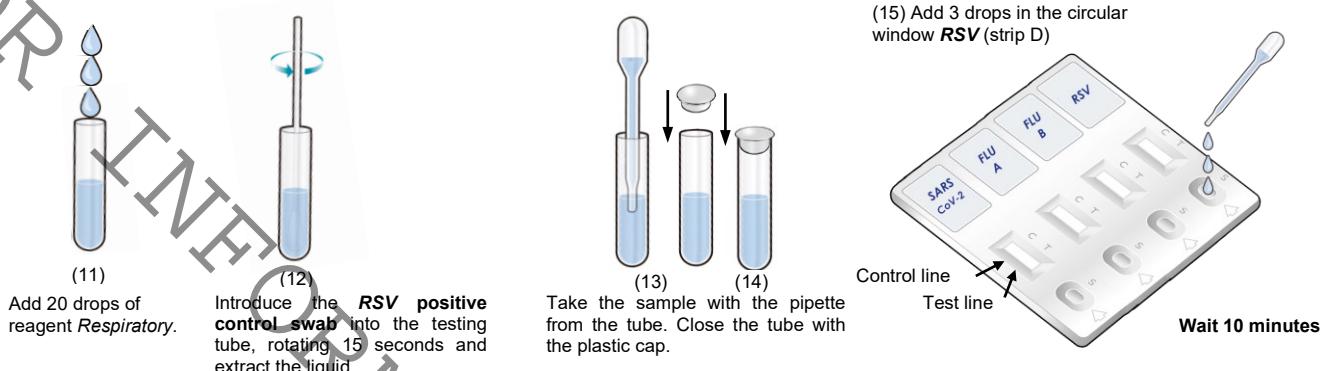15. Take the Reagent Respiratory (the same used in the other Positive Control procedure and samples procedure), add 20 drops (approx. 670µL) (11) of Reagent Respiratory into a different testing tube and immediately introduce the **RSV Positive Control swab** into the testing tube.
16. Mix the solution by rotating the positive control swab forcefully against the side of the testing tube at least 15 seconds. Extract as much liquid as possible from the swab rotating the swab against the side of the tube as the swab is withdrawn (12). Discard the swab.
17. Use a separate pipette and test for each control. Take the sample from the tube with the pipette (13). Close the tube with the plastic cap (14) and dispense 3 drops (approx. 100-120µL) from the pipette into the **circular window marked with the letter D** (15).
18. Read the result at 10 minutes. Do not read the test result later than 10 minutes.

Positive controls could be tested once for each new test kit opened and as otherwise required by your laboratory's standard quality control procedures.

INTERPRETATION POSITIVE CONTROL PROCEDURE

| SARS-CoV-2 Positive Control | Influenza A Positive Control | Influenza B Positive Control | RSV Positive Control | | Interpretation of the results |
|-----------------------------|------------------------------|------------------------------|----------------------|-----------|---|
| POSITIVE | | | | | |
| 1. | GREEN-RED | GREEN-RED | GREEN-RED | GREEN-RED | <p>SARS-CoV-2 (A) POSITIVE, Influenza B (B) POSITIVE, Influenza B (C) POSITIVE and RSV (D) POSITIVE: A GREEN line (control line (C)) and a RED line (test line (T)) in the four strips, in the results window, appear at the same time due to presence of antigens in the SARS-CoV-2 Positive Control swab in strip A, presence of antigens in Influenza A Positive Control swab in strip B, presence of antigens in Influenza B Positive Control swab in strip C and presence of antigens in RSV Positive Control swab in strip D.</p> |
| INVALID | | | | | |
| 2. | Any other result | | | | <p>Invalid result, we recommend repeating the assay with another test and positive control swabs.</p> |

RECOMMENDATIONS

Recommendations World Health Organization: Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays- 11 September 2020.

1. To optimize performance, testing should be conducted by trained operators in strict accordance with the test procedure and within the first 5-7 days following the onset of symptoms.

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2. Where possible, all positive samples giving positive results should be transported to laboratories with NAAT (Nucleic Acid Amplification Test) capability for confirmatory testing.
3. This test could be used to screen at-risk individuals and rapidly isolate positive cases in NAAT- confirmed COVID-19 outbreaks.
4. To monitor trends in disease incidence in communities.
5. For early detection and isolation of positive cases in health facilities, where there is widespread community transmission.
6. A negative result cannot completely exclude an active COVID-19 infection, repeat testing or preferably confirmatory testing should be performed (NAAT) whenever possible, particularly in symptomatic patients.
7. Even CerTest SARS-CoV-2 + Flu A + Flu B + RSV test was not validated using samples from asymptomatic contacts of cases, asymptomatic cases have been demonstrated to have viral loads similar to symptomatic cases, so CerTest SARS-CoV-2 + Flu A + Flu B + RSV could detect as positive.

LIMITATIONS

1. The test must be carried out within 2 hours after opening the sealed bag.
2. CerTest SARS-CoV-2 + Flu A + Flu B + RSV should be used only with nasopharyngeal swab samples. The use of swab specimens taken from other sites or the use of other samples such as saliva, sputum or urine has not been established. The quality of the test depends on the quality of the sample; proper nasopharyngeal specimens must be obtained.
3. The intensity of test line may vary from very strong at high antigens concentration to faint when the antigens concentration is close to the detection limit value of the test.
4. Positive results determine the presence of SARS-CoV-2 and/or *Influenza* type A and/or *Influenza* type B and/or *RSV* antigens. A diagnosis of SARS-CoV-2, *Influenza* type A, *Influenza* type B and *RSV* infection should only be made by a physician after all clinical and laboratory findings have been evaluated and must be based in the correlation of the results with further clinical observations.
5. Positive results do not rule out co-infections with other pathogens.
6. Only VTM, UTM and Saline buffer transport media has been validated with CerTest SARS-CoV-2 + Flu A + Flu B + RSV test. When using transport media the test sensitivity can be reduced due to excessive dilution of sample.
7. A negative result is not meaningful because of it is possible the antigens concentration in the nasopharyngeal samples is lower than the detection limit value. If the symptoms or situation still persist, it is recommended that all negative results undergo confirmatory testing using other method and/or virus identification by cell culture, PCR or other technique.

EXPECTED VALUES

On January 30, 2020, the World Health Organization declared COVID-19 a public health emergency of international concern. Since the onset of the pandemic, SARS-CoV-2 has affected millions of people, causing thousands of victims worldwide.

Recent studies have shown that the primary viral infection found in symptomatic patients was identical to that seen in asymptomatic patients, suggesting that COVID-19 patients with asymptomatic infections are capable of transmitting SARS-CoV-2. During the incubation period, asymptomatic patients are also likely to transmit the virus to healthy people.

Although SARS-CoV-2 infection can affect people of any age, children and the elderly with asthma are in a higher risk category for the disease. Those with underlying health problems such as cancer, chronic respiratory diseases, cardiovascular diseases, and diabetes are also reported to have a higher chance of developing serious illnesses.

Multiple viral infections are frequent in hospitalized children with respiratory tract disease (17.4%). Multiple viral infections are linked to higher fever, longer hospital stays and more frequent use of antibiotics than in the case of infants with single *RSV* infections.

Respiratory Syncytial Virus (RSV) is the leading cause of emergency visits and hospitalization for acute lower respiratory tract infections (LRTI) in infants and young children worldwide.

In adults and older, healthy children, the symptoms of *Respiratory Syncytial Virus* are mild and typically mimic the common cold. Self-care measures are usually all that's needed to relieve any discomfort.

Infection with *Respiratory Syncytial Virus* can be severe in some cases, especially in premature babies and infants with underlying health conditions. *RSV* can also become serious in older adults, adults with heart and lung diseases, or anyone with a very weak immune system (immunocompromised).

PERFORMANCE CHARACTERISTICS

Analytical sensitivity (detection limit)

Detection limit value (typical) of CerTest SARS-CoV-2 + Flu A + Flu B + RSV (strip A: SARS-CoV-2) is 1.00 ng/mL of SARS-CoV-2 recombinant protein or 1×10^3 TCID₅₀/mL of 2019nCoV/USA-WA1/2020.

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Detection limit value (typical) of CerTest SARS-CoV-2 + Flu A + Flu B + RSV (strip B: *Influenza A*) is 6.25 ng/mL of *Influenza A* recombinant nucleoprotein.

Detection limit value (typical) of CerTest SARS-CoV-2 + Flu A + Flu B + RSV (strip C: *Influenza B*) is 25.00 ng/mL of *Influenza B* recombinant nucleoprotein.

Detection limit value (typical) of CerTest SARS-CoV-2 + Flu A + Flu B + RSV (strip D: *RSV*) is 10.00 ng/mL of *RSV* recombinant protein antigen.

Clinical sensitivity and specificity

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip A: SARS-CoV-2)

An external evaluation (Hospital Universitario Principe de Asturias, Madrid (Spain)), with nasopharyngeal samples from people suspected of infection by SARS-CoV-2 virus, was performed comparing the results obtained by an immunochromatographic test (CerTest SARS-CoV-2 + Flu A + Flu B + RSV Certest (SARS-CoV-2 strip)) vs qPCR technique (VIASURE). The results were as follows

| IC test: CerTest SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) | | qPCR technique | | | Total |
|---|-------|----------------|-----|--|-------|
| | | + | - | | |
| | + | 79 | 0 | | 79 |
| | - | 5 | 150 | | 155 |
| | Total | 84 | 150 | | 234 |

CerTest SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) vs qPCR technique

| | Mean Value | 95% confidence interval |
|-----------------|------------|-------------------------|
| Sensitivity (*) | 94.0% | 86.7–98.0% |
| Specificity | 100.0% | 97.6–100.0% |
| PPV | 100.0% | 95.4–100.0% |
| NPV | 96.8% | 92.6–98.9% |

(*) Taking into account the recommendations for *Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays* (11 September 2020) from the WHO, the sensitivity of the test was calculated with nasopharyngeal samples with high viral load (high viral loads is expected in early symptomatic phases of the illness (with the first 5-7 days of illness) in the range of Ag-RDT test detection).

An external evaluation (Laboratoire de Virologie CHU Amiens-Picardie, Amiens (Francia)), with nasopharyngeal samples from people suspected of infection by SARS-CoV-2 virus, was performed comparing the results obtained by an immunochromatographic test (CerTest SARS-CoV-2 + Flu A + Flu B + RSV Certest (SARS-CoV-2 strip)) vs qPCR technique. The results were as follows:

| IC test: CerTest SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) | | qPCR technique | | | Total |
|---|-------|----------------|----|--|-------|
| | | + | - | | |
| | + | 88 | 0 | | 88 |
| | - | 8 | 62 | | 70 |
| | Total | 96 | 62 | | 158 |

SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) vs qPCR technique

| | Mean Value | 95% confidence interval |
|-----------------|------------|-------------------------|
| Sensitivity (*) | 91.7% | 84.2–96.3% |
| Specificity | 100.0% | 94.2–100.0% |
| PPV | 100.0% | 95.9–100.0% |
| NPV | 88.6% | 78.7–94.9% |

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(*) Taking into account the recommendations for *Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays* (11 September 2020) from the WHO, the sensitivity of the test was calculated with nasopharyngeal samples with high viral load (high viral loads is expected in early symptomatic phases of the illness (with the first 5-7 days of illness) in the range of Ag-RDT test detection).

An evaluation, with nasopharyngeal samples from people suspected of infection by SARS-CoV-2 virus, was performed comparing the results obtained by an immunochromatographic test (CerTest SARS-CoV-2 + Flu A + Flu B + RSV Certest (SARS-CoV-2 strip)) vs qPCR technique (VIASURE). The results were as follows:

| IC test: CerTest SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) | | qPCR technique | | | Total |
|---|-------|----------------|-----|--|-------|
| | | + | - | | |
| | + | 26 | 1 | | 27 |
| | - | 2 | 233 | | 235 |
| | Total | 28 | 234 | | 262 |

| SARS-CoV-2 + Flu A + Flu B + RSV (SARS-CoV-2) vs qPCR technique | | |
|---|------------|-------------------------|
| | Mean Value | 95% confidence interval |
| Sensitivity (*) | 92.9% | 76.5–99.1% |
| Specificity | 99.6% | 97.6–100.0% |
| PPV | 96.3% | 81.0–99.9% |
| NPV | 99.1% | 97.0–99.9% |

(*) Taking into account the recommendations for *Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays* (11 September 2020) from the WHO, the sensitivity of the test was calculated with nasopharyngeal samples with high viral load (high viral loads is expected in early symptomatic phases of the illness (with the first 5-7 days of illness) in the range of Ag-RDT test detection).

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip B: Influenza A)

Results showed in tables below have been obtained from several evaluations developed using an immunochromatographic (CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A), CerTest) and others commercial test kit.

1574 respiratory samples have been used for *Influenza A* evaluation (130 positive samples, 11 false positives, 1402 negatives samples and 31 false negatives). The results were as follows:

| | Evaluation number | True Positives | False Negatives | True Negatives | False Positives |
|--|-------------------|----------------|-----------------|----------------|-----------------|
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs XPECT FLU A&B (Remel) | I | 7 | 0 | 85 | 1 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs Instand EQAS program (March 2016) | II | 101 | 26 | 955 | 3 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs Binax NOW® Influenza A&B Card (Alere) | III | 11 | 2 | 87 | 0 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs SD Bioline, Influenza Ag A/B/A (H1N1) (SD Bioline) | IV | 1 | 0 | 97 | 0 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs MariPOC® respi (ArcDia International) | V | 10 | 3 | 178 | 7 |
| | Total | 130 | 31 | 1402 | 11 |

CerTest SARS-CoV-2 + Flu A + Flu B + RSV (Influenza A) vs other test used in evaluation (I, II, III,

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| IV, V) for Influenza A | | |
|------------------------|------------|-------------------------|
| | Mean Value | 95% confidence interval |
| Sensitivity | 80.7% | 73.8 – 86.5% |
| Specificity | 99.2% | 98.6 – 99.6% |
| PPV | 92.2% | 86.5 – 96.0% |
| NPV | 97.8% | 96.9 – 98.5% |

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip C: Influenza B)

Results showed in tables below have been obtained from several evaluations developed using an immunochromatographic (CerTest SARS-CoV-2 + Flu A + Flu B + RSV (*Influenza B*), CerTest) and others commercial test kit.

1276 respiratory samples have been used for Influenza B evaluation (75 positive samples, 6 false positives, 1181 negatives samples and 14 false negatives). The results were as follows:

| | Evaluation number | True Positives | False Negatives | True Negatives | False Positives |
|---|-------------------|----------------|-----------------|----------------|-----------------|
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (<i>Influenza B</i>) vs XPECT FLU A&B (Remel) | I | 7 | 2 | 83 | 1 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (<i>Influenza B</i>) vs Instant EQAS program (March 2016) | II | 63 | 12 | 1005 | 5 |
| CerTest SARS-CoV-2 + Flu A + Flu B + RSV (<i>Influenza B</i>) vs SD Bioline, Influenza Ag A/B/A (H1N1) (SD Bioline) | III | 5 | 0 | 93 | 0 |
| | Total | 75 | 14 | 1181 | 6 |

CerTest SARS-CoV-2 + Flu A + Flu B + RSV (*Influenza B*) vs other test used in evaluation (I, II, III) for Influenza B

| | Mean Value | 95% confidence interval |
|-------------|------------|-------------------------|
| Sensitivity | 84.3% | 75.0 – 91.1% |
| Specificity | 99.5% | 98.9 – 99.8% |
| PPV | 92.6% | 84.6 – 97.2% |
| NPV | 98.8% | 98.0 – 99.4% |

Evaluations for SARS-CoV-2 + Flu A + Flu B + RSV (strip D: RSV)

The evaluation, was performed comparing the results obtained by an immunochromatographic test (CerTest SARS-CoV-2 + Flu A + Flu B + RSV, Certest) (RSV strip) and others commercial available immunochromatographic test BinaxNOW® RSV (Alere). The results were as follows:

| IC test: CerTest SARS-CoV-2+ Flu A + Flu B + RSV (RSV) | IC test: BinaxNOW® RSV | | | | Total |
|---|------------------------|----|----|----|-------|
| | + | + | - | | |
| | | 18 | 0 | 18 | |
| | - | 1 | 10 | 11 | |
| | Total | 19 | 10 | 29 | |

CerTest SARS-CoV-2 + Flu A + Flu B + RSV (RSV) vs BinaxNOW® RSV

| | Mean Value | 95% confidence interval |
|-------------|------------|-------------------------|
| Sensitivity | 94.7% | 74.0–99.9% |
| Specificity | 100.0% | 69.2–100.0% |

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The results showed a high sensitivity and specificity to detect SARS-CoV-2, *Influenza* type A, *Influenza* type B and RSV using CerTest SARS-CoV-2 + *Flu A + Flu B + RSV*.

Hook Effect

CerTest SARS-CoV-2 + *Flu A + Flu B + RSV* test does not show inhibitory hook effect at concentration higher than 100,000xLoD for Strip A: SARS-CoV-2, at concentrations higher than 30,000xLoD for Strip B: *Influenza A*, at concentrations higher than 8,000xLoD for Strip C: *Influenza B* and at concentrations higher than 39,000xLoD for Strip D: RSV.

Cross reactivity

An evaluation was performed to determine the cross reactivity of CerTest SARS-CoV-2 + *Flu A + Flu B + RSV*; no cross reactivity against organisms, pathogens, substances that cause infections:

For Strip A. SARS-CoV-2

| | | | |
|--|-------------------------------|------------------------------------|------------------------------------|
| Adenovirus | <i>Cryptosporidium parvum</i> | <i>Influenza</i> type B | <i>Salmonella typhi</i> |
| Astrovirus | <i>Entamoeba histolytica</i> | <i>Legionella pneumophila</i> | <i>Salmonella typhimurium</i> |
| Bovine Haemoglobin | <i>Escherichia coli O157</i> | <i>Listeria monocytogenes</i> | <i>Shigella boydii</i> |
| <i>Campylobacter jejuni</i> | <i>Giardia</i> | <i>Norovirus GI</i> | <i>Shigella dysenteriae</i> |
| <i>Clostridium difficile antigen GDH</i> | <i>Helicobacter pylori</i> | <i>Norovirus GII</i> | <i>Shigella flexneri</i> |
| <i>Clostridium difficile Toxin A</i> | <i>Human Calprotectin</i> | <i>Pig Haemoglobin</i> | <i>Shigella sonnei</i> |
| <i>Clostridium difficile Toxin B</i> | <i>Human Haemoglobin</i> | <i>Rotavirus</i> | <i>Streptococcus pneumoniae</i> |
| <i>Coronavirus strain 229E</i> | <i>Human Lactoferrin</i> | <i>Respiratory Syncytial Virus</i> | <i>Streptococcus pyogenes</i> |
| <i>Coronavirus strain NL63</i> | <i>Human Transferrin</i> | <i>Salmonella enteritidis</i> | <i>Yersinia Enterocolitica O:3</i> |
| <i>Coronavirus strain OC43</i> | <i>Influenza</i> type A | <i>Salmonella paratyphi A</i> | <i>Yersinia Enterocolitica O:9</i> |

Five positive samples for Human coronavirus HKU1 were tested in an external evaluation carried out with CerTest SARS-CoV-2 + *Flu A + Flu B + RSV* (SARS-CoV-2, strip A) test and the results obtained were negative. This suggest that there is no cross reactivity with HKU1. In addition, even though the percentage identity of the nucleocapsid protein sequence of HKU1 with the nucleocapsid protein sequence of SARS-CoV-2 is about 35% that is considered as very low homology. This suggest that there is no cross reactivity with HKU1.

Three positive samples for Rhinovirus were tested in an external evaluation carried out with CerTest SARS-CoV-2 + *Flu A + Flu B + RSV* (SARS-CoV-2, strip A) test and the results obtained were negative. This suggest that there is no cross reactivity with Rhinovirus

CerTest SARS-CoV-2 + *Flu A + Flu B + RSV* (SARS-CoV-2, strip A) showed some cross reaction with SARS and very low with MERS.

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Interferences

An evaluation was performed to determine the possible interferences of CerTest SARS-CoV-2 + Flu A + Flu B + RSV; no interferences against the substances tested were detected:

| Potencial interfering substance | Concentration | Results | Potencial interfering substance | Concentration | Results |
|------------------------------------|----------------------------|----------|-------------------------------------|----------------------------|----------|
| Metronidazole | 3.0 mg/mL | Negative | Loperamide hydrochloride (Fortasec) | 0.15 mg/mL | Negative |
| Ampicillin | 3.0 mg/mL | Negative | Heparin (Hibor) | 350.0 IU/mL | Negative |
| Oseltamivir | 3.0·10 ⁻² mg/mL | Negative | Almagato (Almax) | 3.0 mg/mL | Negative |
| Amantadine | 0.3 mg/mL | Negative | Fosfamycin (Monurol) | 3.0·10 ⁻³ mg/mL | Negative |
| Ribavirin | 3.0 mg/mL | Negative | Acetylcysteine (Flumucil) | 3.0·10 ⁻² mg/mL | Negative |
| Codeine (Toseina) | 0.2 mg/mL | Negative | Dexketoprofen trometamol (Enantyum) | 0.3 mg/mL | Negative |
| Benzocaine (Angileptol) | 3.0·10 ⁻² mg/mL | Negative | Levofloxacin | 3.0·10 ⁻² mg/mL | Negative |
| Cloperastine (Flutox) | 0.3 mg/mL | Negative | Ciprofloxacin | 0.3 mg/mL | Negative |
| Carbocisteína (Iniston mucolítico) | 3.0·10 ⁻² mg/mL | Negative | Rifampicin (Rifaldin) | 0.3 mg/mL | Negative |
| Loratadine | 0.3 mg/mL | Negative | Phenoxyethylpenicillín potassium | 3.0 mg/mL | Negative |
| Dexchloropheniramine (Polaramine) | 0.3 mg/mL | Negative | Ambroxol hydrochloride (Mucosan) | 0.3 mg/mL | Negative |
| Ebastine (Ebaste) | 3.0 mg/mL | Negative | Macrogol 3350 (Movicol) | 3.0 mg/mL | Negative |
| Acetyl Salicylic (Adiro) | 0.3 mg/mL | Negative | Lysine Carbocisteína (Pectox) | 3.0·10 ⁻² mg/mL | Negative |
| Ibuprofen (Epidifen) | 0.3 mg/mL | Negative | Hydroxyzine dihydrochloride | 0.3 mg/mL | Negative |
| Paracetamol (Dolocatil) | 5.0 mg/mL | Negative | Lorazepam | 3.0·10 ⁻³ mg/mL | Negative |
| Metamizole (Nolotil) | 5.0 mg/mL | Negative | Amoxicillin | 3.0 mg/mL | Negative |
| Prednisone | 0.3 mg/mL | Negative | Mercaptopurine | 0.3 mg/mL | Negative |
| Omeprazole | 2.0·10 ⁻³ mg/mL | Negative | Biotine | 100.0 µg/mL | Negative |
| Human haemoglobin | 5.0 µg/mL | Negative | Human lactoferrin | 5.0 µg/mL | Negative |
| Human transferrin | 0.5 µg/mL | Negative | Mucine | 5.0 mg/mL | Negative |
| Human calprotectin | 5.0 µg/mL | Negative | Human blood | 50.0 mg/mL | Negative |

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1. Palestino, G.; García-Silva, I.; González-Ortega, O. and Rosales-Mendoza, S. **Can nanotechnology help in the fight against COVID-19?** *Expert Review of Anti-infective Therapy.* <https://doi.org/10.1080/14787210.2020.1776115>. Jun. 2020.
2. Moeis MR.; Rahayu AP.; Ihsani N. and Pertiwi W. **Coronavirus-SARS-CoV-2: Biology and Problems in rRT-PCR Detection.** *Borneo Journal of Pharmacy.* Vol. 3; special Issue 1:136-145. Jun. 2020. <https://doi.org/10.33084/bjop.v3iSpecial-1.1429>
3. Arnedo-Pena, A.; Sabater-Vidal, S.; Meseguer-Ferrer, N.; Pac-Sa, M. R.; Mañes-Flor, P.; Gascó-Laborda, J. C.; Larrea, R. M.; Tirado-Balaguer, M. D.; Romeu-García, M. A.; Gil-Fortuño, M.; Safont-Adsuara, L.; Blasco, A.; Gomila-Sar, B.; Moreno-Muñoz, M. R. and Bellido-Blasco, J. **COVID-19 secondary attack rate and risk factors in household contacts in Castellón (Spain): Preliminary report.** *Revista Enfermedades Emergentes.* 2020;19(2):64-70.
4. Younes, N.; Al-Sadeq, D. W.; AL-Jighefee, H.; Younes, S.; Al-Jamal, O.; Daas, H. I.; Yassine, H. M. and Nasrallah, G. K. **Challenges in Laboratory Diagnosis of the Novel Coronavirus SARS-CoV-2.** *Viruses (Multidisciplinary Digital Publishing Institute).* 12 (6): 582. <https://doi.org/10.3390/v12060582>. May. 2020.
5. Rincón, A.; Moreso, F.; López-Herradón, A.; Fernández-Robres, M.A.; Cidraque, I.; Nin, J.; Méndez, O.; López, M.; Pájaro, C.; Satorra, A.; Stuard, S. and Ramos, R. **The keys to control a coronavirus disease 2019 outbreak in a haemodialysis unit.** *Clinical Kidney Journal.* 2020: 1–8.
6. Frakou, P. C.; Papaevangelou, V.; Antoniadou, A.; Kavvatha, D.; Ploussi, A.; Pantazis N.; Sirmpilantze, T.; Psarrakis, C.; Pournaras, S. A.; Tsiodras, S. and Kelekis, A. **Preliminary Data of a Quantitative Point of Care Test for SARS-CoV-2 Antibodies From Greece.** *In vivo.* 34: xxx-xxx (2020).
7. Ezhilan, M.; Suresh, I. and Nesakumar, N. **SARS-CoV, MERS-CoV and SARS-CoV-2: A Diagnostic Challenge.** *Measurement.* 168 (2021): 108335. <https://doi.org/10.1016/j.measurement.2020.108335>. Aug. 2020.
8. Daniel E. Noyola, et al. Comparison of a New Neuraminidase Detection Assay with an Enzyme Immunoassay, Immunofluorescence, and Culture for Rapid Detection of Influenza A and B Viruses in Nasal Wash Specimens. *Journal of Clinical Microbiology*, Mar. 2000, p. 1161–1165 Vol. 38, No. 3
9. Adriana Weinberg and Miranda L. Walker. Evaluation of Three Immunoassay Kits for Rapid Detection of Influenza Virus A and B. *Clinical And Diagnostic Laboratory Immunology*, Mar. 2005, p. 367–370 Vol. 12, No. 3
10. Andreea C. Cazacu, Gail J. Demmler et al. Comparison of a New Lateral-Flow Chromatographic Membrane Immunoassay to Viral Culture for Rapid Detection and Differentiation of Influenza A and B Viruses in Respiratory Specimens. *Journal of Clinical Microbiology*, Aug. 2004, p. 3661–3664 Vol. 42, No. 8.
11. Kenneth E. Irmen and James J. Kelleher. Use of Monoclonal Antibodies for Rapid Diagnosis of Respiratory Viruses in a Community Hospital. *Clinical and Diagnostic Laboratory Immunology*, May 2000, p. 396–403 Vol. 7, No. 3.
12. 1. ANN R. FALSEY and EDWARD E. WALSH. "Respiratory Syncytial Infection in Adults", *Clinical Microbiology Reviews*, July 2000, Vol. 13, No.3, p. 371-384
13. 2. DIANE C. HALSTEAD, SANDRA TODD, and GALE FRITCH; "Evaluation of Five Methods for Respiratory Syncytial Virus Detection" *Journal of Clinical Microbiology*, May 1990, Vol. 28, No. 5; p. 1021-1025.
14. Antigen-detection in the diagnosis of SARS-CoV-2 infection using rapid immunoassays. Interim guidance. 11 September 2020. Word Health Organization.

SYMBOLS FOR IVD COMPONENTS AND REAGENTS/SÍMBOLOS PARA REACTIVOS Y PRODUCTOS PARA DIAGNÓSTICO IN VITRO

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